



Citadel Plus

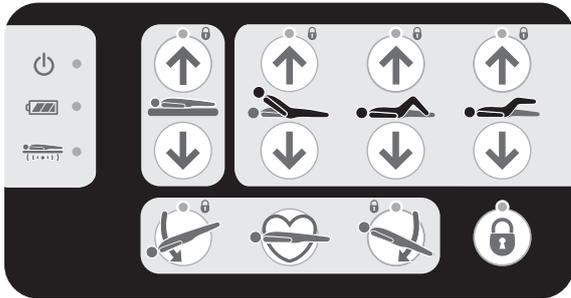

Attendant Control Panel




Nurse Control Panel

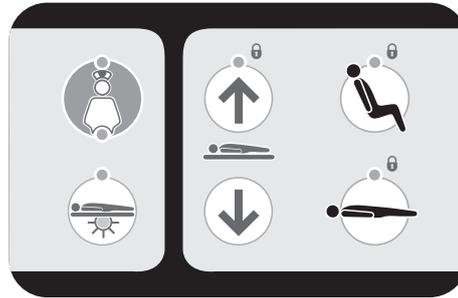

Patient Handset



Patient Control Panel

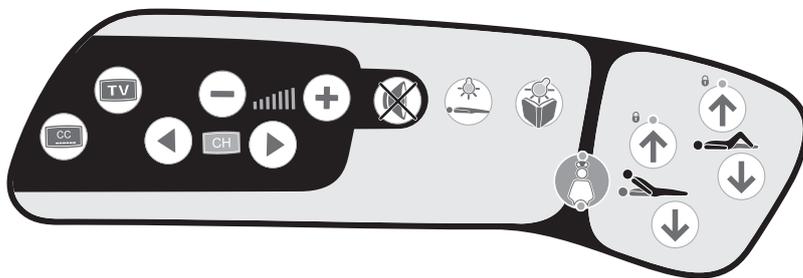

- 

Power On Indicator

- 

Battery Status LED Indicator
 - 

 Battery fully charged
 - 

 Battery charging
 - 

 Battery discharged

- 

Anti-Entrapment System (AES) Indicator - When illuminated, AES system has been activated and obstruction needs to be removed before the frame will lower.

- 
Mattress Platform Height - Raise or lower the mattress platform.

- 
Backrest Angle - Raise or lower the backrest.

- 
Thigh Section - Raise or lower the thigh section.

- 
Calf Section - Raise or lower the calf section.

- 
Lower head end (Trendelenburg position)

- 
Lower foot end (Reverse Trendelenburg position)

- 
CPR Position - Flattens the mattress platform and lowers the bed height

- 
Function Lockout - Press the button, then press selected Attendant Control Panel button(s) to lock or unlock function. Yellow light = locked.



Auto-Chair

(nurse control panel only) - Press and hold for chair position.



Auto-Chair Down

(nurse control panel only) - Press and hold to return to flat level position.



Under Bed Light

(nurse control panel only)



Nurse Call

(available on selected models) - Press to request assistance.



Reading lamp

(available on selected models)- Press to turn on light.



Reading lamp on/off

(available on selected models)



Sound on/off (mute)

(available on selected models)



Volume down

(available on selected models)



Volume up

(available on selected models)



Channel down

(available on selected models)



Channel up

(available on selected models)



Television on/off



(available on selected models)



Closed caption on/off

(available on selected models)

Weigh Scale Control Panel

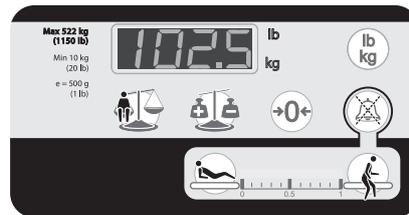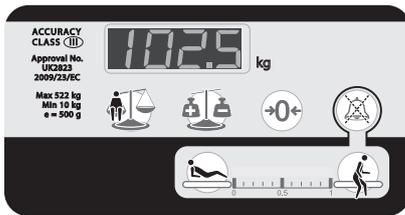

Zero - Press to reset weighing system when bed is first set up and before a new patient is placed on the bed.



Auto Compensation - Press when adding or removing items from the bed without affecting the indicated patient weight.



Weigh - Press to display weight.



Select Units - Press to select weighing unit.



VariZone Mute Button - Press to mute exit alarm for three minutes.

Patient movement detection threshold - An indicator shows the selected sensitivity of patient movement.



In Bed - Press to reactivate or activate patient movement detection and increase sensitivity.



Egress - Press to reactivate or activate patient movement detection and decrease sensitivity.