# Bedside Cabinet

## Instructions for Use

**ARJOHUNTLEIGH**  
GETINGE GROUP

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ZH

## Design Policy and Copyright

® and ™ are trademarks belonging to the ArjoHuntleigh group of companies unless otherwise stated.

© ArjoHuntleigh 2016

As our policy is one of continuous improvement, we reserve the right to modify designs without prior notice. The content of this publication may not be copied either wholly or in part without the consent of ArjoHuntleigh.

## Contents

1.	Introduction .....	2
2.	Components Diagram .....	3
3.	Operation .....	4
4.	Cleaning and Maintenance .....	6
5.	Warranty and Service.....	8
6.	Specifications/Configurations.....	9
7.	Symbols Used.....	11

## Warnings, Cautions and Notes

**WARNINGS** given in this manual identify possible hazards in procedures or conditions which, if not correctly followed, could result in death, injury or other serious adverse reactions.

**Cautions** given in this manual identify possible hazards in procedures or conditions which, if not correctly followed, could result in equipment damage or failure.

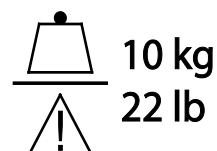
★ This symbol indicates optional items or features.

### WARNING

The carer should ensure the Bedside Cabinet is correctly positioned and adjusted to allow the patient to use it safely and comfortably.

Do not exceed the Bedside Cabinet's Safe Working Load of 100kg evenly distributed. Do not lean or sit on the Bedside Cabinet.

Do not exceed the single drawer's Safe Working Load of 10kg evenly distributed. Take care not to trap hands or fingers when using the Bedside Cabinet.

### Caution

Take care when moving the Bedside Cabinet over rough or uneven surfaces. Lift the product over large ridges and wide gaps.

# 1. Introduction

---

The **ArjoHuntleigh Bedside Cabinet** is a two-drawer bedside cabinet offering storage space for the patient's belongings.

*Features:*

- Wipe-clean top surface and drawers
- Towel/bag hook
- Lockable top drawer
- Large lower drawer with bottle holder
- Castors with kick-stop brakes
- Perimeter rail ★

**BC1 Bedside Cabinet**

*Shown with optional rails*

## 2. Components Diagram

---

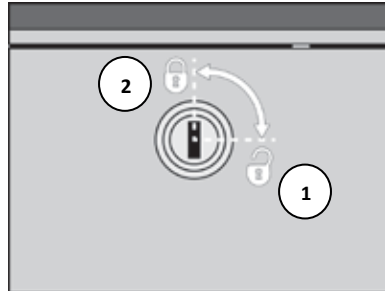
### 3. Operation

---

#### Lock / Unlock the Top Drawer

Using the provided key:

1. Turn the lock clockwise (horizontal position) to unlock the drawer.
2. Turn the lock counter-clockwise (vertical position) to lock the drawer.


#### Remove / Replace the Drawers

*Note: Though the illustrations below show this procedure only for the top drawer, the bottom drawer can also be removed / replaced as described.*


**To remove:**

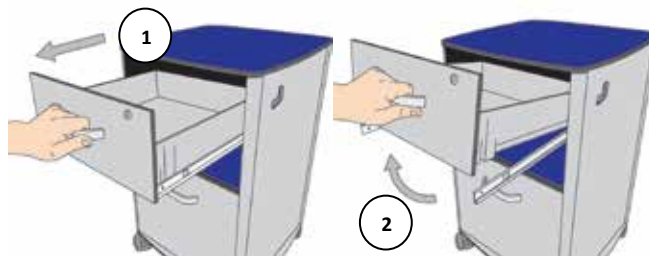1. Pull the drawer outwards to its fully extended position.
2. Pull the drawer directly upwards to disengage it from the drawer runners.


**To replace:**

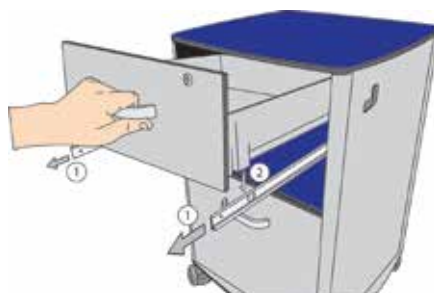1. Fully extend left and right runners.
2. Align the slots on the underside of the drawer with the tabs on the extended runners to secure the drawer.

## Braking Castor

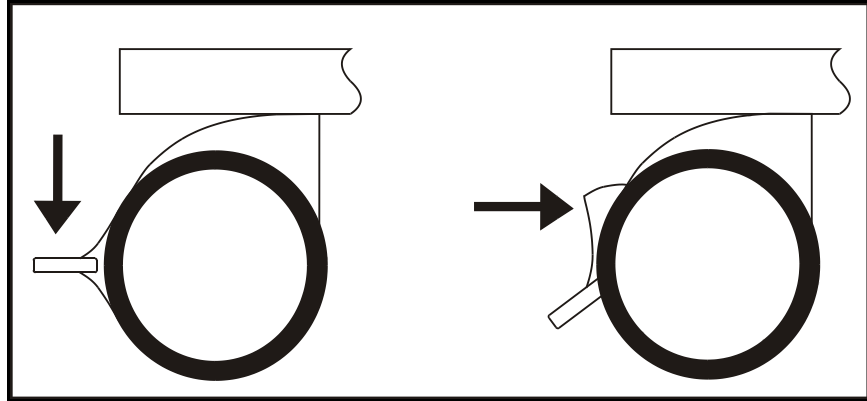**To apply the brake:**  
Press the lever down.

**To release the brake:**  
Push the lever horizontally.


### WARNING

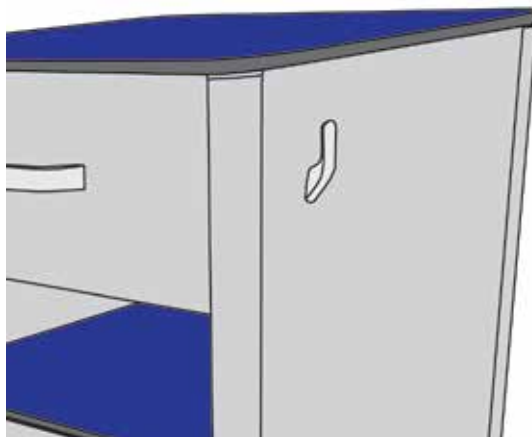**Always apply the brakes when the product is in use; unlock for movement, such as repositioning or cleaning.**

## Bottle Rail

## Towel / Bag Hook

## 4. Cleaning and Maintenance

---

### 4.1 Cleaning

Wearing suitable protective clothing, clean all surfaces with a disposable cloth soaked in a neutral detergent and water.

Start by cleaning the upper sections of the product and work along all horizontal surfaces. Work methodically towards the lower sections and clean the castors last. Take extra care to clean areas that may trap dust or dirt. Drawers can be removed for cleaning.

Rinse with clean water and dry with disposable paper towels.

### 4.2 Disinfecting

After cleaning as described above, wipe all surfaces with sodium dichloroisocyanurate (NaDCC) at a concentration of 1,000 parts per million (0.1%) of available chlorine.

In the case of pooling body fluids, e.g. blood, the concentration of NaDCC should be increased to 10,000 parts per million (1%) of available chlorine.

#### **Caution**

**Do not use abrasive compounds or pads.**

**Do not use phenol-based disinfectants.**

### 4.3 Preventive Maintenance

#### **WARNING**

**Before starting any service or maintenance procedures, ensure that the equipment has been adequately decontaminated**

**Avoid skin contact with lubricants. Rubber gloves and protective clothing should be worn when carrying out maintenance work**

**Use only replacement spare parts that are approved by ArjoHuntleigh**

**Dispose of waste components and materials safely in accordance with applicable national and local regulations**

The following preventive maintenance procedures should be carried out at intervals of 12 months. All service or repair activities must be carried out by properly qualified and trained persons approved by ArjoHuntleigh.

Check that all four castors swivel and rotate freely. Remove any debris that may have been picked up by the wheels.

Check the braking castors for correct operation.

Check that both drawers operate smoothly and easily.

Check that the top drawer locks and unlocks smoothly and easily.

## **4.4 Castors – replacement**

### **4.4.1 Removal**

- a) Turn the Bedside Cabinet onto its back or one side so that the underside is accessible.
- b) Use a 19mm spanner to loosen the castor stem. Unscrew the castor (09-1)(09-2)(09-3) from the base.

#### **Caution**

**When replacing the long stem castors (09-1), remove only one castor at a time as these castors attach the corner extrusions (01-2)(01-3) to the base (01-9)**

### **4.4.2 Replacement**

- a) Turn the Bedside Cabinet onto its back or one side so that the underside is accessible.
- b) Position the castor (09-1)(09-2)(09-3) over the corner extrusion. Use a 19mm spanner to tighten the castor stem.

## **5. Warranty and Service**

---

ArjoHuntleigh's standard terms and conditions apply to all sales. A copy is available on request. These contain full details of warranty terms and do not limit the statutory rights of the consumer.

For service, maintenance and any questions regarding this, or any other ArjoHuntleigh product, please refer to the local contact information provided on the inside back cover of this Instructions for Use document.

## 6. Specifications/Configurations

Overall dimensions	Width	48.5cm
	Depth	47.5cm
	Height	74cm
Product weight	26kg	
Castor size	60mm	
Top surface	48 x 48cm	
Top drawer internal dimensions	Width	37.2cm
	Depth	37.3cm
	Height	8.6cm
Bottom drawer internal dimensions	Width	37.2cm
	Depth	37.3cm
	Height	8.6cm
Safe working load <div><div></div><div>10 kg 22 lb</div></div>	Bedside Cabinet: 100kg evenly distributed	
	Single Drawer: 10kg evenly distributed	
Flame rating	Classification M1 according to Standard NF P 92-507:2004	
Material	Panels	High Pressure Compact Laminate (HPCL)
	Posts	Aluminum Extrusion Anodized Finish
	Drawers	Injection Molded Plastic
Material properties of HPCL	Resistance to scratches and impact from daily use	
	Resistance to moisture, steam, and heat	
Available Configurations		
Colors	AB – AH Blue LL – Lilac LB – Light Blue LG – Light Grey WO – White Oak AT – Ash Tree NW – North Walnut	
Options	BC0 Bedside Cabinet, no rails BC1 Bedside Cabinet, with rails	

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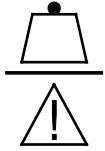
CANADA  
ArjoHuntleigh  
90 Matheson Boulevard West  
Suite 300  
CA-MISSISSAUGA, ON, L5R 3R3  
Tel/Tél: +1 905 238 7880  
Free: +1 800 665 4831 Institutional  
Free: +1 800 868 0441 Home Care  
Fax: +1 905 238 7881  
E-mail: info.canada@arjohuntleigh.com

ČESKÁ REPUBLIKA  
ArjoHuntleigh s.r.o.  
Hlinky 118  
CZ-603 00 BRNO  
Tel: +420 549 254 252  
Fax: +420 541 213 550

DANMARK  
ArjoHuntleigh A/S  
Vassingerødvej 52  
DK-3540 LYNGE  
Tel: +45 49 13 84 86  
Fax: +45 49 13 84 87  
E-mail: dk\_kundeservice@arjohuntleigh.com

DEUTSCHLAND  
ArjoHuntleigh GmbH  
Peter-Sander-Strasse 10  
DE-55252 MAINZ-KASTEL  
Tel: +49 (0) 6134 186 0  
Fax: +49 (0) 6134 186 160  
E-mail: info-de@arjohuntleigh.com

ESPAÑA  
ArjoHuntleigh Ibérica S.L.  
Ctra. de Rubí, 88 1ª planta - A1  
08173 Sant Cugat del Vallés  
ES- BARCELONA 08173  
Tel: +34 93 583 11 20  
Fax: +34 93 583 11 22  
E-mail: info.es@arjohuntleigh.com

FRANCE  
ArjoHuntleigh SAS  
2 Avenue Alcide de Gasperi  
CS 70133  
FR-59436 RONCQ CEDEX  
Tél: +33 (0) 3 20 28 13 13  
Fax: +33 (0) 3 20 28 13 14  
E-mail: info.france@arjohuntleigh.com

HONG KONG  
ArjoHuntleigh (Hong Kong) Ltd  
1510-17, 15/F, Tower 2  
Kowloon Commerce Centre  
51 Kwai Cheong Road  
Kwai Chung  
HONG KONG  
Tel: +852 2207 6363  
Fax: +852 2207 6368

INTERNATIONAL  
ArjoHuntleigh International Ltd  
ArjoHuntleigh House  
Houghton Hall Park  
Houghton Regis  
UK-DUNSTABLE LU5 5XF  
Tel: +44 (0) 1582 745 800  
Fax: +44 (0) 1582 745 866  
E-mail: international@arjohuntleigh.com

ITALIA  
ArjoHuntleigh S.p.A.  
Via Giacomo Peroni 400-402  
IT-00131 ROMA  
Tel: +39 (0) 6 87426211  
Fax: +39 (0) 6 87426222  
E-mail: Italy.promo@arjohuntleigh.com

MIDDLE EAST  
ArjoHuntleigh ME  
Office G005 - Nucleotide Complex,  
Dubitech & Research Park,  
P.O.Box 214742, Dubai,  
United Arab Emirates  
Tel: +971 (0)4 447 0942  
E-mail: international@arjohuntleigh.com

NEDERLAND  
ArjoHuntleigh Nederland BV  
Biezenwei 21  
4004 MB TIEL  
Postbus 6116  
4000 HC TIEL  
Tel: +31 (0) 344 64 08 00  
Fax: +31 (0) 344 64 08 85  
E-mail: info.nl@arjohuntleigh.com

NEW ZEALAND  
ArjoHuntleigh Ltd  
41 Vestey Drive  
Mount Wellington  
NZ-AUCKLAND 1060  
Tel: +64 (0) 9 573 5344  
Free Call: 0800 000 151  
Fax: +64 (0) 9 573 5384  
E-mail: nz.info@ArjoHuntleigh.com

NORGE  
ArjoHuntleigh Norway AS  
Olaf Helsets vei 5  
N-0694 OSLO  
Tel: +47 22 08 00 50  
Faks: +47 22 08 00 51  
E-mail: no.kundeservice@arjohuntleigh.com

ÖSTERREICH  
ArjoHuntleigh GmbH  
Dörrstrasse 85  
AT-6020 INNSBRUCK  
Tel: +43 (0) 512 204 160 0  
Fax: +43 (0) 512 204 160 75

POLSKA  
ArjoHuntleigh Polska Sp. z o.o.  
ul. Ks Piotra Wawrzyniaka 2  
PL-62-052 KOMORNIKI (Poznań)  
Tel: +48 61 662 15 50  
Fax: +48 61 662 15 90  
E-mail: arjo@arjohuntleigh.com

PORTUGAL  
ArjoHuntleigh em Portugal  
MAQUET Portugal, Lda. (Distribuidor Exclusivo)  
Rua Poeta Bocage n.º 2 - 2G  
PT-1600-233 Lisboa  
Tel: +351 214 189 815  
Fax: +351 214 177 413  
E-mail: Portugal@arjohuntleigh.com

SUISSE / SCHWEIZ  
ArjoHuntleigh AG  
Fabrikstrasse 8  
Postfach  
CH-4614 HÄGENDORF  
Tél/Tel: +41 (0) 61 337 97 77  
Fax: +41 (0) 61 311 97 42

SUOMI  
Oy Vestek AB  
Martinkuja 4  
FI-02270 ESPOO  
Puh: +358 9 8870 120  
E-mail: info@vestek.fi

SVERIGE  
ARJO Scandinavia AB  
Hans Michelsensgatan 10  
SE-211 20 MALMÖ  
Tel: +46 (0) 10 494 7760  
Fax: +46 (0) 10 494 7761  
E-mail: kundservice@arjohuntleigh.com

UNITED KINGDOM  
ArjoHuntleigh UK  
ArjoHuntleigh House  
Houghton Hall Park  
Houghton Regis  
UK-DUNSTABLE LU5 5XF  
Tel: +44 (0) 1582 745 700  
Fax: +44 (0) 1582 745 745  
E-mail: sales.admin@arjohuntleigh.com

USA  
ArjoHuntleigh Inc.  
2349 W Lake Street Suite 250  
US-Addison, IL 60101  
Tel: +1 630 307 2756  
Free: +1 800 323 1245 Institutional  
Free: +1 800 868 0441 Home Care  
Fax: +1 630 307 6195  
E-mail: us.info@arjohuntleigh.com

REV 17: 03/2016

---

## GETINGE GROUP

**Getinge Group** is a leading global provider of products and systems that contribute to quality enhancement and cost efficiency within healthcare and life sciences. We operate under the three brands of **ArjoHuntleigh**, **Getinge** and **Maquet**. **Getinge** provides solutions for infection control within healthcare and contamination prevention within life sciences. **Maquet** specializes in solutions, therapies and products for surgical interventions, interventional cardiology and intensive care.

---

## ARJOHUNTLEIGH

GETINGE GROUP

ArjoHuntleigh focuses on patient handling and hygiene, disinfection, DVT prevention, medical beds, therapeutic surfaces and diagnostics.

ArjoHuntleigh AB  
Hans Michelsensgatan 10  
211 20 Malmö, Sweden  
[www.arjohuntleigh.com](http://www.arjohuntleigh.com)

---