ARJOHUNTLEIGH


GETINGE GROUP

MAXISLIDE FLITES

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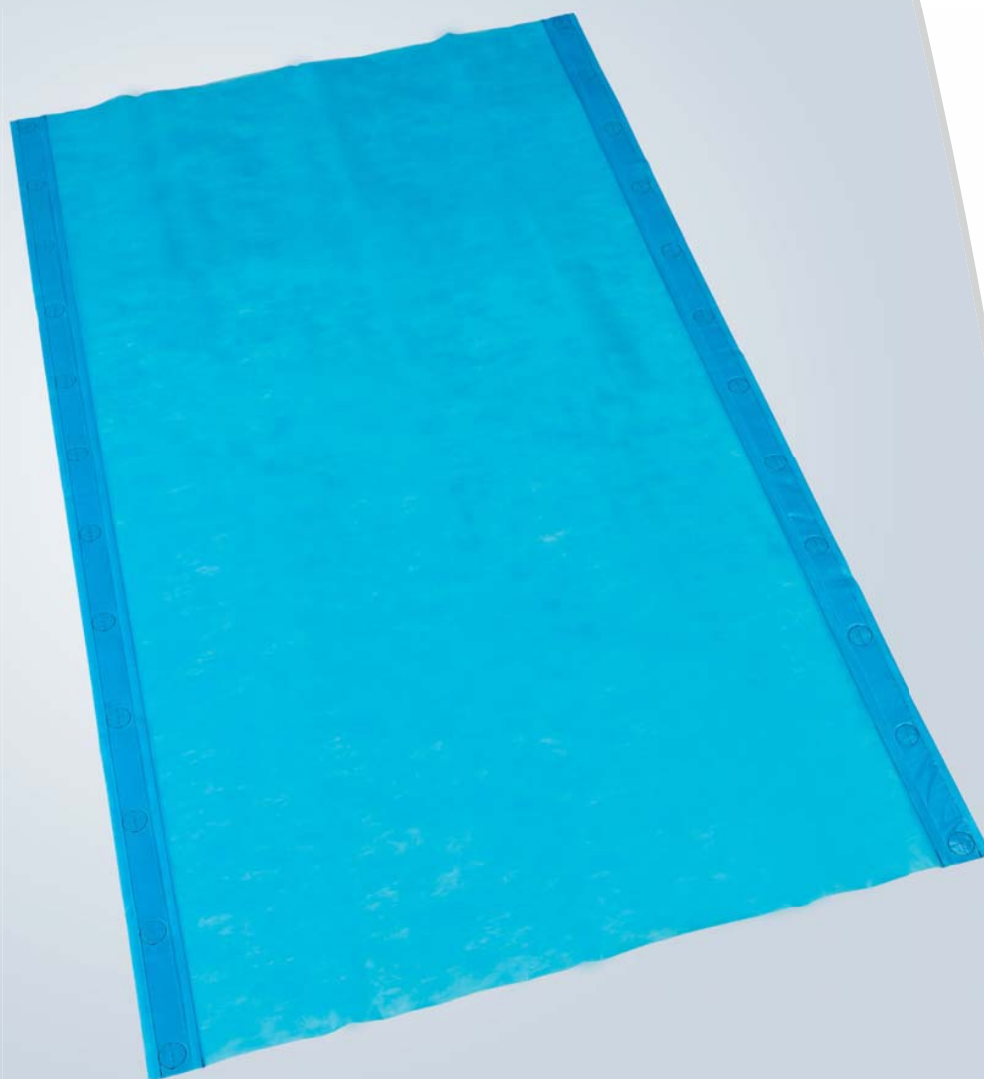CE

© ArjoHuntleigh

ArjoHuntleigh products are patented or patent pending. Patent information is available by contacting ArjoHuntleigh.

The policy of ArjoHuntleigh is one of continuous development and we therefore reserve the right to make technical alterations without notice. The content of this publication may not be copied either whole or in part without the consent of ArjoHuntleigh.

Foreword

Thank you for purchasing ArjoHuntleigh products.

ArjoHuntleigh MaxiSlide Flites is a quality product designed especially for hospital use.

We are dedicated to serving your needs and providing the best products available along with training that will bring your staff maximum benefit from every ArjoHuntleigh product.

Please contact us if you have any questions about the operation or maintenance of your ArjoHuntleigh products.

Please read this manual thoroughly!

Please read this manual in its entirety before using MaxiSlide Flites. Information in the manual is crucial to the proper operation and maintenance of the product and will help protect your product and ensure that it performs to your satisfaction. Some of the information in this manual is important for your safety and must be read and understood to help prevent possible injury.

Unauthorized modifications on any ArjoHuntleigh equipment may affect its safety. ArjoHuntleigh will not be held responsible for any accidents, incidents or lack of performance that occur as a result of any unauthorized modification to its products.

For service and support

Please use the contact information for your country on the last page of this manual.

Table of contents

Safety instructions	5
Product description	6
Using MaxiSlide Flites	7
1. Applying the MaxiSlide Flites	7
1a. The unravelling technique	7
1b. The log-roll technique	8
2. Lateral transfer	8
2a. Lateral transfer to another bed or a trolley	8
3. Repositioning in bed with two caregivers	9
4. Moving up the bed with two caregivers pulling	9
5. Placing a sling under a patient	9
6. Removing the MaxiSlide Flites	10
6a. Removing while the patient remains in a supine position	10
6b. Removing using a reverse log-roll	10
Maintenance, storage and cleaning instructions	11
Labels	12

Read before use

Before using your MaxiSlide Flites, you must read and fully understand these instructions for use. You must be trained on MaxiSlide Flites, and any accessories, as well as on its functions. These instructions for use are mandatory for the safe and effective handling of the MaxiSlide Flites, including the safety of patients and caregivers.

Definitions and symbols used in this manual:

WARNING

Means: failure to follow these instructions may result in injury to yourself or to others.

CAUTION

Means: failure to follow these instructions may cause damage to the product(s).

Intended use

- The ArjoHuntleigh MaxiSlide Flites is a patient specific disposable sliding sheet. The same MaxiSlide Flites should not be used with more than one specific patient. The MaxiSlide Flites is to be used for a limited period only, and by nature of its design, must be treated as a disposable product.
- Sliding sheets are intended to assist caregivers in lateral transfers and repositioning in hospitals. The use of a sliding sheet is recommended on patients with restricted mobility by appropriately trained caregivers with adequate knowledge of the care environment, its common practices and procedures and in accordance with the guidelines in the instructions for use.
- ArjoHuntleigh strongly recommends caregivers to work in pairs when transferring and repositioning patients. Two MaxiSlide Flites are always intended to be used at the same time.

Patient assessment

- An individual assessment of the patient's health condition should be carried out by a qualified nurse or therapist before each actual transfer / repositioning situation.
- Sliding techniques can be used successfully with patients of all dependency levels, but they should be subjected to a suitable assessment. Factors you should consider are environment, characteristics of the patient, the task you are performing and the caregiver's individual capability. These factors will give guidance regarding the number of caregivers needed.

WARNING

- The MaxiSlide Flites sliding sheets should only be used by appropriately trained staff.
- Always use a pair of MaxiSlide Flites, not a single sheet.
- Never leave the patient lying on the MaxiSlide Flites unattended.
- For safety reasons, always ensure that there is a caregiver on both sides of the support surface.
- Transfer/ reposition the patient with caution. Perform the movement at a calm and steady pace.
- Remove MaxiSlide Flites from under the patient once the transfer / repositioning is performed.
- MaxiSlide Flites has not been designed for lifting and must not be used as a lifting aid.
- Always engage the brakes when transferring from bed to bench / trolley / wheel chair / commode chair and vice versa.
- The transfer / support surface should be lowered to its ergonomically correct height for the caregiver during the manoeuvre.
- After the transfer / repositioning, the side rails should be put back into position and the transfer surface should be lowered.
- Do not leave the MaxiSlide Flites on the floor or any other surface since they are extremely slippery and accidents could occur.
- Store away from children.
- Please check each sliding sheet before and after every use.

CAUTION

The MaxiSlide Flites is a patient specific disposable sliding aid. If torn or damaged, soiled, washed or subjected to any liquids, it must be disposed of. It must not be sterilized for re-use.

The MaxiSlide Flites sliding sheets are to be used as an assisting device for lateral transfer of patients in a recumbent position from one horizontal support surface to another. They can also be used to reposition a patient. Sliding sheets are available in three widths; choice depends on type of patient handling, patient size and transfer distance. Straps to gain extra reach are available as accessories.

The MaxiSlide Flites is made of a nonwoven fabric coated with silicone. All materials used in the sliding aid comply with established biocompatibility standards and have been tested accordingly (ISO10993). It is a patient specific device and should be disposed of when showing serious signs of wear or tear, if it has been washed, soiled or subjected to any liquids. Always dispose of the sliding aid when the patient leaves the care facility. The MaxiSlide Flites are individually packed but delivered in a cardboard box. Once removed from the box, mark each sliding sheet with the name of the patient and the date of first use. Please use a permanent marker when writing the name of the patient. Do not store a used MaxiSlide Flites or the box of sheets in a wet environment. After the MaxiSlide Flites has been used, hang it up using the handles.

The MaxiSlide Flites with accessories

NPA0500: MaxiSlide Flites 30", length 77" / 196cm, width 30" / 76cm

NPAV5260:MaxiSlide Flites 45", length 77" / 196cm, width 45" / 114cm

NPAV5000:MaxiSlide Flites 60", length 77" / 196 cm, width 60" / 152cm

NPA0510: MaxiSlide Flites straps, length 32" / 80cm, width $1\frac{9}{16}$ " / 4cm

The techniques shown in this manual are examples of the latest ergonomically correct patient handling manoeuvres, based on sound biomechanical principles and safe utilization. Pushing actions are carried out at a high level to avoid jolting and to allow the caregiver to profit from his or her strong leg muscles. Pulling is carried out at a lower height, with the elbows straight, to avoid overuse of shoulders and to allow use of the leg muscles to achieve movement. Always work with a good body balance and using calm and controlled movements.

If a pressure releasing mattress is used, gently push down to assist fitting. You may also find it helpful to set the mattress pressure to a firm setting for the procedure.

In addition to the patient assessment, always make an assessment of the transfer distance to ensure that the correct size of MaxiSlide Flites is chosen.

1. Applying the MaxiSlide Flites

ArjoHuntleigh recommends two different techniques for positioning the sliding sheets under the patient, the unravelling and the log-rolling techniques. Some patients, e.g. those who are in pain or in a poor medical condition, may not be suitable for log-rolling. In these cases, the unravelling technique is a more appropriate way to apply the MaxiSlide Flites.

1a. The unravelling technique

For the purpose of easy understanding, figures 1-2 are shown without the patient. Place a pair of Maxi-Slide Flites sliding sheets together lengthwise. Fold the sheets in 8-inch / 20-cm folds, see figure 1. Then turn the folded MaxiSlide Flites over, so the folded side is closest to the transfer surface, see figure 2.

Figure 1

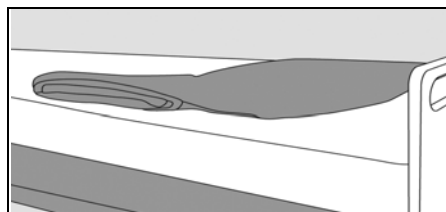
Figure 2

Place the folded sheets under the patient's head and a pillow, see figure 3, to unravel from head to feet. Before starting to unravel, make sure that the sheets are placed as far down as possible so that there is no excessive amount of material above the patient's head.

Work in pairs. Start by holding the top end of the sliding sheets with your outside hand to secure the sheets in position. By using clear commands, simultaneously unravel the sheets in stages - one fold at a time - with your inside hand while keeping the back of the hand in contact with the support surface, see figure 4. Do not pull upwards. Unfold the folds by pulling slightly outwards away from the patient, keeping the sliding sheets as flat and stretched as possible to avoid wrinkles causing skin shear.

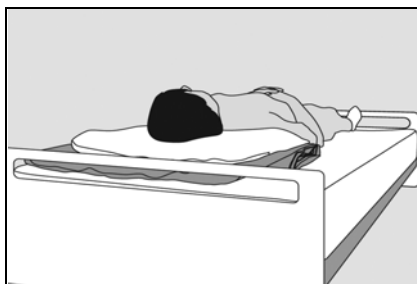
Figure 3

Figure 4

1b. The log-roll technique

Place a pair of MaxiSlide Flites sheets together lengthwise. Bend the patient's leg on the far side and place your hands on thigh and shoulder. Tilt the patient towards you, in the opposite direction in which the lateral transfer will be performed, see figure 5. Apply the MaxiSlide Flites under the patient's body, so that there is as much material as possible for the patient to slide on. The caregiver on the other side of the bed places the sliding aids under the patient. Gently tilt the patient back into a supine position. If needed to ascertain that the whole body of the patient is positioned on the MaxiSlide Flites, please tilt the patient towards the other side, see figure 6. Pull and straighten out the MaxiSlide Flites if needed.

Figure 5

Figure 6

2. Lateral transfer

Before performing a lateral transfer, ensure that the wheels are solidly locked. To facilitate the transfer, position the receiving surface slightly lower than the surface where the transfer starts from. Choose a pair of MaxiSlide Flites which will cover the transfer distance. If there is a risk that the MaxiSlide Flites will not cover the full length of the transfer, place a third sliding sheet on the receiving surface.

2a. Lateral transfer to another bed or a trolley

When making a lateral transfer over a longer distance, for instance from a bed to a shower trolley, we recommend that the caregiver on the receiving side who will perform the pulling action uses the accessory MaxiSlide Flites straps. The straps prevent the caregivers from having to overstretch and will increase their working area. Two straps are fitted on the pulling receiving side of the upper MaxiSlide Flites. These straps are attached to the handles that are located at the heaviest parts of the patient body, probably the shoulder and hip, see figure 7. Please check that the straps fit nicely and comfortably into each caregiver's hand.

The caregiver on the pulling side grasps the straps. He or she uses a walk stance and steps backwards from the trolley, using his or her body weight to move the patient, see figure 8. The caregiver on the pushing side will assist with initiating the transfer, taking care not to overstretch.

The patient assessment may suggest that two caregivers need to be positioned on the pulling receiving side. Four straps are then fitted to the upper MaxiSlide Flites spread out along the length of the patient. One carer is still positioned on the pushing side.

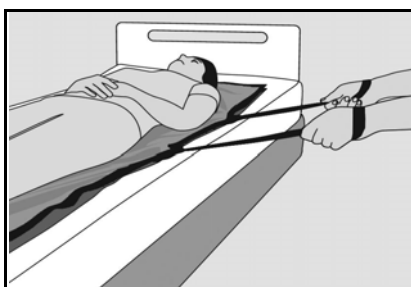
Figure 7

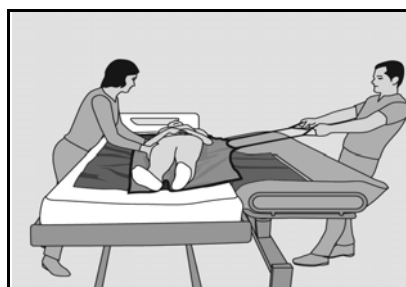
Figure 8

If necessary, a final positioning of the patient in the centre of the other bed or the trolley is achieved by using the handles on the MaxiSlide Flites.

3. Repositioning in bed with two caregivers

Ensure the bed is at the correct height for positioning the MaxiSlide Flites using the unravelling technique, see section 1a.

Using the principles of normal movement wherever possible, prepare the patient to turn to the side desired. The patient is moved towards one side of the bed to allow space to turn onto his or her side. To reposition, lower the bed into an ergonomically appropriate position for the caregivers.

The caregiver grasps the MaxiSlide Flites handles and adopts a walk stance moving backwards. Using his or her body weight, the caregiver grasps the upper MaxiSlide Flites with both hands as close to the patient as possible to ease the upward pull and then pulls the patient gently towards the second carer on the other side of the bed, see figure 9.

Gradually change the pulling angle more upwards to finalize the repositioning of the patient. The other carer secures the patient by standing on the opposite side of the bed. The patient may need to have his or her hips, shoulders and legs repositioned for comfort.

Figure 9

Remove both MaxiSlide Flites sheets from behind the patient rather than the front, as this is where the majority of the slide material ends up when using the described technique.

4. Moving up the bed with two caregivers pulling

Use of a profiling bed can prevent a patient from slipping down towards the foot end. However, if such a bed is not available the following technique can be used to slide the patient back towards the head end. This technique may be used where the bed is at a low height; therefore a pulling action is desirable. The patient is unable to assist with the movement. The MaxiSlide Flites sheets have been fitted under the patient to remove any shearing effects of the movement.

Each caregiver stands on opposite sides of the bed and holds the upper MaxiSlide Flites with the hand closest to the bed. Ensuring good communication, the caregivers move simultaneously and slowly backwards to the head of the bed, see figures 10 and 11.

Figure 10

Figure 11

5. Placing a sling under a patient

Position the sling at the top of the bed so that the middle of the sling is aligned with the patient's spine. Then place the sling in between the two sliding sheets which have already been applied under the patient.

Each caregiver grasps the top of the upper MaxiSlide Flites with his or her outer hand and one leg piece of the sling each with his or her inside hand, see figure 12. By using clear commands, simultaneously pull the sling downwards under the patient, see figure 13. Do not pull upwards and do not pull too far down.

Figure 12

Figure 13

The curved bottom of the sling should be placed just above the patient's coccyx. Remove the upper MaxiSlide Flites sheet only, see paragraph 6 below, and attach the sling to the lift. Leave the lower sliding sheet in bed and remove it after the patient has been lifted off the bed.

6. Removing the MaxiSlide Flites

6a. Removing while the patient remains in a supine position

If the MaxiSlide Flites has been applied by the unravelling technique, we suggest the following technique for removing the sliding aid. Remove the MaxiSlide Flites by inserting your hand under the sliding aid where the lowest amount of pressure is, for instance under the patient's neck, under the curve of the back and under the hollow of the knee, see figure 14. Grab both sheets and pull them towards you slowly removing them simultaneously. Use the handles for storage.

Figure 14

6b. Removing using a reverse log-roll

When removing the MaxiSlide Flites by performing a reversed log-roll, we recommend the tilting of the patient to be done in the same direction as the lateral transfer was performed, see figure 15 where the reverse log-roll is performed using a draw sheet. By doing so, you minimize the amount of MaxiSlide Flites material to be removed by the caregiver behind the patient's back. Gently tilt back the patient into a supine position. If necessary, tilt the patient onto the other side to remove both sheets. Use the handles for storage.

Figure 15

The MaxiSlide Flites sliding sheet is a patient specific disposable sliding sheet. The same MaxiSlide Flites should not be used with more than one patient. The MaxiSlide Flites is to be used for a limited period only, and by nature of its design, must be treated as a disposable product.

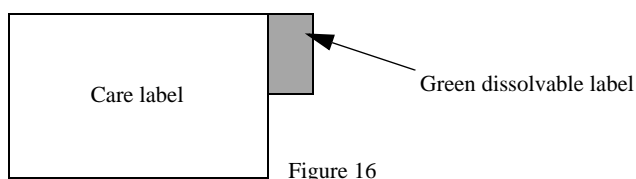
Do not store the sliding sheets or the boxes with unused sliding sheets in a wet environment.

Check the sliding sheets regularly for any signs of wear and tear to make sure that they are undamaged. Dispose and replace the slides when damaged. Wrinkle marks from folding and handling do not affect the strength or performance of the slide.

Always ensure that the sliding sheets are packed away or hung by the handles in order to prevent people accidentally slipping on them.

Do not wet wipe, wash, sterilize or soil the MaxiSlide Flites. If a sliding sheet has been subjected to any such treatment, it must be discarded.

If you are uncertain whether your MaxiSlide Flites has been washed or not, check that the green, water dissolvable label stitched to the sheet on the right side of the Care label is intact, see figure 16. If the label is completely or partially dissolved, the sheet must be discarded. A visible and intact water dissolvable label indicates that the sliding aid has not been subjected to washing.

Before placing a pair of MaxiSlide Flites under a patient, make sure that each sheet is marked with the patient's name and the date it was used for the first time. Please use a permanent marker when writing the name of the patient.

When the patient leaves the hospital, the MaxiSlide Flites must always be disposed of.

Care and Washing Symbols	
	No bleaching allowed
	No tumble drying allowed
	No ironing allowed
	No dry cleaning allowed
	Do NOT wash

Certificates/Markings	
	CE marking in accordance with the Medical Devices Directive 93/42/EEC. The equipment has been tested and fulfils all essential requirements.

Flites Symbol	
	Only on Flites. (Disposable and resident specific)

Fibre Content	
PP	Polypropylene
PE	Polyethylene

Misc. Symbols	
	Safe Working Load (SWL)
	Resident's name symbol
	Record symbol
	Read the <i>IFU</i> before use
REF	Article number
	Manufacturing date and year
	Manufacturer name and address

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NORGE

ArjoHuntleigh Norway AS
Olaf Helsets vei 5
N-0694 OSLO
Tel: +47 22 08 00 50
Faks: +47 22 08 00 51
E-mail:
no.kundeservice@arjohuntleigh.com

ÖSTERREICH

ArjoHuntleigh GmbH
Dörrstrasse 85
AT-6020 INNSBRUCK
Tel: +43 (0) 512 204 160 0
Fax: +43 (0) 512 204 160 75

POLSKA

ArjoHuntleigh Polska Sp. z o.o.
ul. Ks Piotra Wawrzyniaka 2
PL-62-052 KOMORNIKI (Poznan)
Tel: +48 61 662 15 50
Fax: +48 61 662 15 90
E-mail: arjo@arjohuntleigh.com

PORTUGAL

ArjoHuntleigh em Portugal
MAQUET Portugal, Lda. (Distribuidor Exclusivo)
Rua Poeta Bocage n.º 2 - 2G
PT-1600-233 Lisboa
Tel: +351 214 189 815
Fax: +351 214 177 413
E-mail: Portugal@arjohuntleigh.com

SUISSE / SCHWEIZ

ArjoHuntleigh AG
Fabrikstrasse 8
Postfach
CH-4614 HÄGENDORF
Tél/Tel: +41 (0) 61 337 97 77
Fax: +41 (0) 61 311 97 42

SUOMI

Oy Vestek AB
Martinkuja 4
FI-02270 ESPOO
Puh: +358 9 8870 120
E-mail: info@vestek.fi

SVERIGE

ARJO Scandinavia AB
Hans Michelsensgatan 10
SE-211 20 MALMÖ
Tel: +46 (0) 10 494 7760
Fax: +46 (0) 10 494 7761
E-mail: kundservice@arjohuntleigh.com

UNITED KINGDOM

ArjoHuntleigh UK
ArjoHuntleigh House
Houghton Hall Park
Houghton Regis
UK-DUNSTABLE LU5 5XF
Tel: +44 (0) 1582 745 700
Fax: +44 (0) 1582 745 745
E-mail: sales.admin@arjohuntleigh.com

USA

ArjoHuntleigh Inc.
2349 W Lake Street Suite 250
US-Addison, IL 60101
Tel: +1 630 307 2756
Free: +1 800 323 1245 Institutional
Free: +1 800 868 0441 Home Care
Fax: +1 630 307 6195
E-mail: us.info@arjohuntleigh.com

ARJOHUNTLEIGH

GETINGE GROUP

GETINGE GROUP is a leading global provider of products and systems that contribute to quality enhancement and cost efficiency within healthcare and life sciences. We operate under the three brands of ArjoHuntleigh, GETINGE and MAQUET. ArjoHuntleigh focuses on patient mobility and wound management solutions. GETINGE provides solutions for infection control within healthcare and contamination prevention within life sciences. MAQUET specializes in solutions, therapies and products for surgical interventions and intensive care.

www.ArjoHuntleigh.com

 ArjoHuntleigh AB
Hans Michelsensgatan 10
211 20 Malmö
Sweden