ARJOHUNTLEIGH

GETINGE GROUP

Flowtron Hydroven 12 Arm and Leg Garments

Instructions for Use, Notice d'utilisation
Bedienungsanleitung, Gebruiksaanwijzing
Brugervejledning, Instruktioner för användning
Instruções de Utilização, Οδηγίες χρήσης
Istruzioni per l'uso, Instrucciones de uso
Kullanım Talimatları

CE
0086

...with people in mind

To be used only under the direction of a physician.
100% Latex Free.


For use with **ArjoHuntleigh Flowtron® Hydroven 12** pump only.

Refer to the **Flowtron Hydroven 12 User Manual 316903EN** for complete information on using the pump and garments.


Garments and Garment Inserts


Arm Garment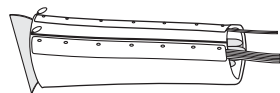

Arm Garment with Insert

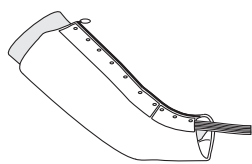Leg Garment

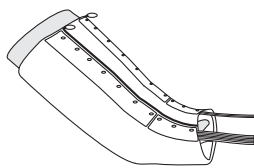Leg Garment with Insert

Applying the Garments

1. Make sure the garment is the correct size for the limb, and that a garment insert is used if necessary.
2. Undo the zip on the garment.
3. If a garment insert is fitted to the garment, fully fasten one of the zips between the garment and insert, leaving the other unfastened.
4. Before applying the garment (and insert, if fitted) to the limb, zip up the first 150 mm (6") of the unfastened garment zip. Put the garment (and insert) onto the limb and fully fasten the zip.
5. Make sure the patient is in a comfortable position with the limb supported or elevated as necessary.
6. Connect the garment(s) to the two push-fit, polarised 12-way connectors on the pump. If only one garment is used, fit the rubber cover to the lower pump connector.
7. Connect the pump to a suitable mains power supply. Switch on the pump.
8. Refer to the **Flowtron Hydroven 12 User Manual** for further information on changing the therapy settings on the pump control panel.

Warnings

- Do not apply the garment to the limb unless it is partially zipped, as you may damage the garment zip.
- Do not undo the zips or attempt to remove the garments during the therapy session or you may damage the zips. Make sure the therapy session has stopped and the garments have deflated before you remove the garments.
- Do not stand or walk with the garments fitted.
- In the event of a power failure or fault whereby the garment remains inflated, disconnect the tubeset(s) in order to deflate the garment(s) and then remove the garment(s) from the limb(s).

Cleaning the Garment

Wipe down using a neutral detergent or soap powder at 40°C (104°F).

Do not iron.

Do not dry clean.

Do not tumble dry.

Do not machine wash. Air dry thoroughly.

Disinfecting the Garment

After cleaning, wipe complete garment over using 70% isopropyl alcohol wipe or a chlorine-releasing agent at 1000 p.p.m. available chlorine.

Do not use phenol or phenol-derivative disinfectant.

Rinse with clean water to remove any residue. Air dry thoroughly.

Sterilising the Garment

After cleaning, gas sterilisation is possible.

Do not exceed 51°C (124°F). Do not autoclave.

Cleaning the Tubeset

Use a soft brush. Air dry only.

Do not immerse in water. Do not machine wash.

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CANADA

ArjoHuntleigh Canada Inc.
1575 South Gateway Road
Unit "C"
MISSISSAUGA, ON, L4W 5J1
Tel/Tél: +1 905 238 7880
Free: +1 800 665 4831 Institutional
Free: +1 800 868 0441 Home Care
Fax: +1 905 238 7881
E-mail: info.canada@arjohuntleigh.com

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GETINGE GROUP is a leading global provider of products and systems that contribute to quality enhancement and cost efficiency within healthcare and life sciences. We operate under the three brands of **ArjoHuntleigh**, **GETINGE** and **MAQUET**. **ArjoHuntleigh** focuses on patient mobility and wound management solutions. **GETINGE** provides solutions for infection control within healthcare and contamination prevention within life sciences. **MAQUET** specializes in solutions, therapies and products for surgical interventions and intensive care.

316901_05: 02/2012

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ArjoHuntleigh Inc.
ArjoHuntleigh GmbH
ArjoHuntleigh AB