ARJOHUNTLEIGH

GETINGE GROUP

Flowtron Hydroven 3 Garments

Instructions for Use, Notice d'utilisation
Bedienungsanleitung, Gebruiksaanwijzing
Brugervejledning, Instruktioner för användning
Instruções de Utilização, Οδηγίες χρήσης
Istruzioni per l'uso, Instrucciones de uso
Kullanım Talimatları

CE
0086

...with people in mind


To be used only under the direction of a physician
 • 100% Latex Free.

For use with ArjoHuntleigh **Flowtron®**
Hydroven 3 pump only. Refer to the **Flowtron**
Hydroven 3 User Manual for complete
 information on using the pump and garments.


*Before fitting the garment ensure all quick air release
 bungs are closed, as this will effect the efficiency of the
 garment.*

Applying the Garment

1. Check the type of garment - **Hydroven 1**
 (single chamber) or **Hydroven 3** (three
 chamber) - and the length of the garment
 required.
2. Undo the zip on the garment.
3. If a garment insert is fitted to the garment, fully
 fasten one of the zips between the garment and
 insert, leaving the other unfastened.
4. Before applying the garment (and insert, if
 fitted) to the limb, zip up the first 150 mm (6")
 of the unfastened garment zip. Put the garment
 (and insert) onto the limb and fully fasten the
 zip. Make sure that the quick air release bung is
 secured.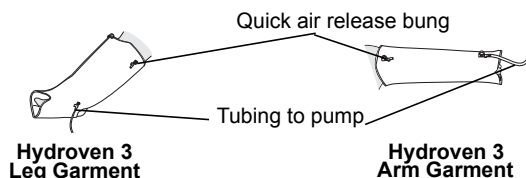

5. Make sure the patient is in a comfortable
 position with the limb supported or elevated as
 necessary.
6. Check that the insert piece connecting tube is
 not kinked and is attached to the garment using
 the lower outlet bung as shown below.

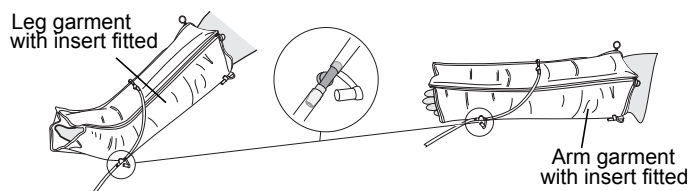7. Attach the garment tubing to the pump ensuring
 a "click" is heard from each snap-lock
 connector.
8. If only one garment is to be used, attach the
 garment to either port on the pump. The system
 will automatically identify that only one
 garment is to be used.

9. Switch on the pump and adjust the pressure
 control accordingly (refer to the pump user
 manual).

Caution

**Do not apply the garment to the limb unless
 it is partially zipped, as you may damage the
 garment zip.**

**Do not apply or remove the garment while it
 is attached to the pump and the pump is in
 operation, as you may damage the garment
 zip.**

Warnings

- In the event of a power failure, or fault
 whereby the garment remains inflated,
 remove the garment.
- Patients must not walk or stand when wearing
 leg garments.

Cleaning - Disinfection

	Wipe down using a neutral detergent or soap powder at 40°C (104°F).		Do not iron.
	Do not dry clean.		Do not tumble dry.
Do not machine wash. Air dry thoroughly. Do not immerse tubaset in water			
 1000ppm NaOCl NaDCC	After cleaning, wipe complete garment over using 70% isopropyl alcohol wipe or a chlorine-releasing agent at 1000 p.p.m. available chlorine.		
 PHENOL	Do not use phenol or phenol-derivative disinfectant.		
Rinse with clean water to remove any residue. Air dry thoroughly.			

To Sterilise

After cleaning, gas sterilisation is possible.

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CANADA

ArjoHuntleigh Canada Inc.
1575 South Gateway Road
Unit "C"
MISSISSAUGA, ON, L4W 5J1
Tel/Tél: +1 905 238 7880
Free: +1 800 665 4831 Institutional
Free: +1 800 868 0441 Home Care
Fax: +1 905 238 7881
E-mail: info.canada@arjohuntleigh.com

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GETINGE GROUP is a leading global provider of products and systems that contribute to quality enhancement and cost efficiency within healthcare and life sciences. We operate under the three brands of **ArjoHuntleigh**, **GETINGE** and **MAQUET**. **ArjoHuntleigh** focuses on patient mobility and wound management solutions. **GETINGE** provides solutions for infection control within healthcare and contamination prevention within life sciences. **MAQUET** specializes in solutions, therapies and products for surgical interventions and intensive care.

510901_04: 02/2012

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ArjoHuntleigh Inc.
ArjoHuntleigh GmbH
ArjoHuntleigh AB