

THE FULLY AUTOMATIC FLUSHER DISINFECTOR

Typhoon™



Efficiency meets high performance in Typhoon, a fully automatic flusher disinfectant with user-friendly features that improve efficiency and infection control.

The Typhoon is known for its distinctive VIP (view-in-process) glass door, which allows you to check on the process at any time during the cycle.

Typhoon also features a sliding door which opens and closes automatically thanks to a built-in IR sensor, and an automatically rotating holder as a standard feature.

Key features



INFECTION CONTROL

Typhoon is an efficient, ergonomic solution at the heart of a smooth-running sluice room. It empties, cleans and disinfects soiled receptacles automatically, ensuring a secure supply of clean goods in circulation.

VIP (VIEW-IN-PROCESS)

Allows you to check at a glance on the process at any point during the cycle.

ROTATING HOLDER

After the door is locked, the holder rotates 150 degrees and empties the receptacles. The rotating holder also has integrated nozzles to help achieve effective cleaning and disinfection results.

TOUCH-FREE AUTOMATIC DOOR

Typhoon is fitted with a built-in IR sensor, supporting touch-free operation. Just place your hand in front of the sensor and the door will open or close automatically.

SLIDING DOOR

A sliding door offers a sleek and space-saving alternative to a swing door.

COOLS RECEPTACLES QUICKLY

When the cycle is completed, the door opens approximately 10 cm to facilitate rapid cooling of the goods.

PRODUCT INFORMATION

Measurements	600 x 600 x 1840 mm
Weight	135 kg (297 lbs)
Noise level	60 dB(A) accordingly to EN ISO 3747:2000
Max. external temperature	40° C (100° F)
Electrical connection	230V, 1N, 50 Hz, 1x16A
External steam connection	Contact your local Arjo representative
Energy consumption	0,25 kWh/cycle

Water consumption	Economy program 25 liters/cycle ±10% Normal program 31 liters/cycle ±10% Intensive program 39 liters/cycle ±10%
External steam consumption	0,3 kg/cycle* * Only applicable if external steam is selected
Water connection	Hot and cold water De-limed (softened) water (Option) Pressure 1-8 bar (14-116 PSI) 15 mm (1/2"), Rate of flow > 0,3 l/s
Steam connection	15 mm (1/2"), pressure 0,3-3 bar (4-43 PSI)* *Only applicable if external steam is selected
Drain connection	Wall ø90 or ø110 mm (3 ½ or 4 ½) Floor ø90 or ø110 mm (3 ½ or 4 ½)