INSTRUCTIONS FOR USE

Arjo Clean

EN • DE • FR • IT • NL • NO • DA • SV • PL • CS • PT • FI

Bedienungsanleitung · Notice d'utilisation · Istruzioni per l'uso · Gebruiksaanwijzing · Bruksanvisning · Brugsvejledning ·
Instruktioner för användning · Instrukcja obsługi · Návod k použití · Instruções de Utilização · Käyttöohjeet

WARNING

To avoid injury, always read this Instructions for Use and accompanied documents before using the product.

Mandatory to read the Instructions for Use

Design Policy and Copyright

® and ™ are trademarks belonging to the Arjo group of companies.

© Arjo 2019.

As our policy is one of continuous improvement, we reserve the right to modify designs without prior notice. The content of this publication may not be copied either whole or in part without the consent of Arjo.

Contents

- Foreword 4
- Intended Use 4
- Safety Instructions 5
- Preparations and Use 6
 - Actions Before First Use 6
 - Actions Before Every Use 6
 - Dilution (e.g. for spray bottle) 6
 - End Of Life Disposal 6
- Labels 7
 - Label/Print explanation 7
 - Symbol explanation 7
- Technical Specifications 8
- ArjoClean Order Numbers 8

EN

Foreword

Thank you for purchasing Arjo equipment

Your *ArjoClean* is a part of a series of quality products designed especially for hospitals, nursing homes and other health care use.

We are dedicated to serving your needs and providing the best products available along with training that will bring your staff maximum benefit from every Arjo product.

Please contact us if you have any questions about the operation or maintenance of your Arjo equipment.

Please read this *Instructions for Use* thoroughly!

Please read this *Instructions for Use (IFU)* in its entirety before using your *ArjoClean*. Information in this *IFU* is crucial to the proper use. Some of the information in this *IFU* is important for your safety and must be read and understood to help prevent possible injury.

If a serious incident occurs in relation to this medical device, affecting the user, or the patient then the user or patient should report the serious incident to the medical device manufacturer or the distributor. In the European Union, the user should also report the serious incident to

the Competent Authority in the member state where they are located.

Arjo strongly advise and warn that only Arjo designed parts, which are designed for the purpose, should be used on equipment and other appliances supplied by Arjo, to avoid injuries or malfunctions caused by the use of inadequate parts.

Unauthorized modifications on any Arjo equipment may affect its safety. Arjo will not be held responsible for any accidents, incidents or lack of performance that occur as a result of any unauthorized modification to its products.

Definitions in this *IFU*

WARNING

Means:

Safety warning. Failure to understand and obey this warning may result in injury to you or to others.

Means:

The name and address of the manufacturer.

Intended Use

Intended Use

***ArjoClean* must be used in line with its intended use and in accordance with these safety instructions. Anyone using *ArjoClean* must also have read and understood the instructions in this *Instructions for Use (IFU)*.**

If there is anything you are not sure about, contact your Arjo representative.

This product is a disinfectant cleaner intended to clean and disinfect medical device equipment within the Arjo healthcare solution product range. The product is intended for use in the medical area on hard surfaces. *ArjoClean* is to be used by appropriately trained caregivers with adequate knowledge of the care environment, its common practices and procedures. *ArjoClean* is to be used in accordance with the *Instructions for Use* for the medical device equipment to be cleaned and disinfected, as well as in accordance with the disinfectant cleaner *Instruction for Use*.

ArjoClean should only be used for the purpose specified in the disinfectant cleaner *Instructions for Use*. Any other use is prohibited. The effectiveness of *ArjoClean* is only valid during shelf life. The shelf life is valid under circumstances stated on the label on the product.

Patient/Resident assessment

Not applicable; product not in contact with patient; only used in between patients to clean and disinfect medical device equipment.

Installation requirements

The product must be installed, if appropriate, by appropriately trained personnel according to the Medical Device Equipment *Instructions for Use*.

Expected service life

Not applicable; consumable that does not have a service life.

Safety Instructions

WARNING

To avoid unintended use, keep out of reach of children and store behind a lock and key.

WARNING

To maintain product quality and ensure proper disinfection, make sure the receptacle is kept tightly sealed. Store at room temperature in original container and do not store in direct sunlight.

WARNING

Make sure to refer to the compatible Arjo medical device equipment's *IFU* or *Operating and Product Care Instructions* when using it together with *ArjoClean* to avoid bodily injury.

WARNING

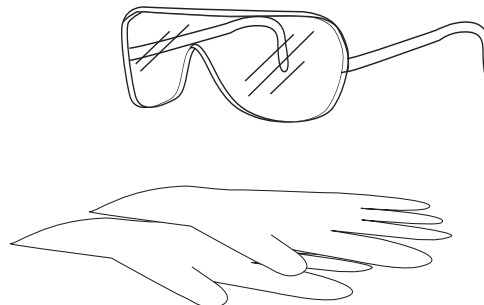
Make sure the disinfectant agent acts the required contact time, to ensure proper disinfection.

Failure to clean equipment according to instructions can cause cross-contamination.

WARNING

To avoid eye or skin irritation, never disinfect in the presence of a patient.

WARNING

Always use protective gloves and protective eye wear when dealing with disinfectant. Make sure to rinse equipment properly before further use. Avoid getting it on your skin, in your eyes or on your clothes.

If contact occurs, rinse with extensive amounts of water.

Remove contact lenses, if easy to do, and continue rinsing.

If skin or eyes become irritated, consult a physician. Carefully follow the instructions on the Arjo disinfectant bottle.

Thorough disinfection and cleaning routines are necessary to prevent the build-up of bacteria on and in the equipment and to prevent cross-contamination. This is particularly vital in cases where there is a shared facility.

EN

Preparations and Use

Actions Before First Use

(8 steps)

- 1 Visually check the package for damage. If the product looks damaged due to freight, make a complaint to the transport agency.
- 2 The packaging should be recycled according to local regulations.
- 3 Read the *IFU*.
- 4 Prepare the *ArjoClean* by qualified personnel. See section *Cleaning and Disinfection Instructions* in the medical device *IFU*.
- 5 Clean the equipment according to section *Cleaning and Disinfection Instructions* in the medical device equipment *IFU*.
- 6 Choose a designated area, where this *IFU* should be kept and is easily accessible at all times.
- 7 In case of exposure refer to the *Safety Data Sheet (SDS)* to find the emergency contact information and specific instructions.
- 8 If you have any questions, please contact your local Arjo representative for support and service. The contact number appears on the last page of this *IFU*.

Actions Before Every Use

(3 steps)

- 1 Make sure that all patient exposed surfaces are thoroughly cleaned mechanically prior to applying *ArjoClean*, to optimise efficacy of the active ingredients.
- 2 Clean the equipment according to section *Cleaning and Disinfection Instructions* in the medical device equipment *IFU*.
- 3 Make sure the disinfectant agent acts for 10 minutes.

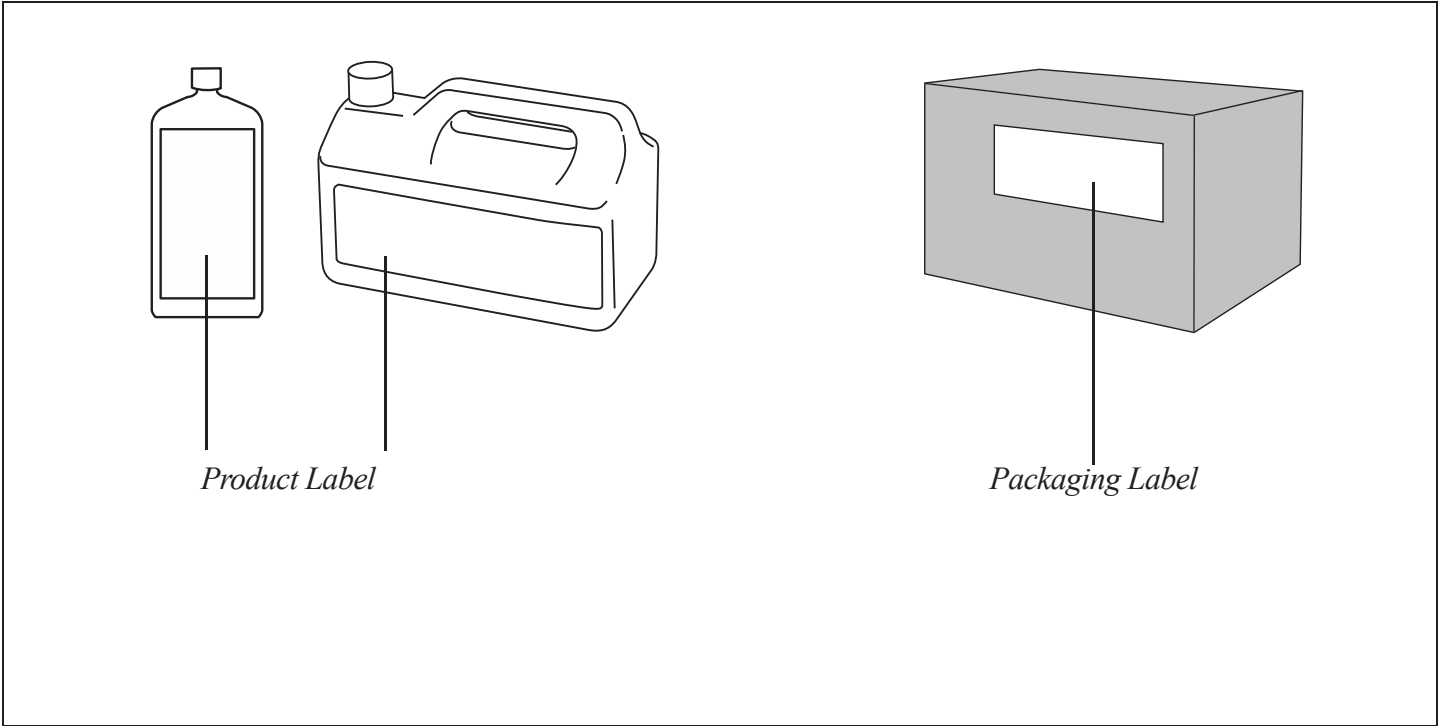
Dilution (e.g. for spray bottle)

See section *Technical Specifications* on page 8.

End Of Life Disposal

- Equipment that has electrical and electronic components should be disassembled and recycled per Waste of Electrical and Electronic Equipment (WEEE) or in accordance with local or national regulation.
- Disinfection liquids – if you have extra liquid, it's safe to pour it down the drain with running water. These products are water soluble and usually mixed with water when used for cleaning, your municipality will treat these cleaners the same way they would if you used them for cleaning.
- The packaging containers should be recycled according to the national or local regulations.

Labels

EN

Label/Print explanation

Product and Packaging Label	States general product information e.g. product name, type of product, hazard statements and transport/storage conditions.
Identification Print (on the bottle)	YYDXXXX YY = Manufacturing year D = Disinfectant XXXX = Batch number EXP MM/YYYY EXP = Expiration MM = Expiration month YYYY = Expiration year

Symbol explanation


	Mandatory to read the <i>IFU</i> before use.
	CE marking indicating conformity with European Community harmonised legislation. Figures indicate Notified Body supervision.

	The name and address of the manufacturer.
	Date of the manufacture
	Date of expiration
	Causes severe skin burns and eye damage.
	Very toxic to aquatic life with long lasting effects.
	Reference number
	Lot number
	Indicates the product is a Medical Device according to EU Medical Device Regulation 2017/745


Technical Specifications

Dilution (not for integrated systems with <i>ArjoClean</i>)	
Pour 16 ml of <i>ArjoClean</i> into a one litre bottle. Add water until the total mixture reaches one litre.	
Antimicrobial efficacy (At 16 ml per 1 litre <i>ArjoClean</i>)	
<ul style="list-style-type: none">• Virucidal activity demonstrated against Avian influenza virus (H5N1) - EN 14476:2005• Yeasticidal activity demonstrated using C. Albicans (DGHM Standardmethoden 8,9)• Bactericidal activity demonstrated against Pseudomonas aeruginosa, Staphylococcus aureus, Salmonella choleraesuis, Escherichia coli, E. Hirae (DGHM Standardmethoden 8,9)	
Transport and storage	
Temperature	0 °C to + 30 °C (+32 °F to + 86 °F)
Allowed Combinations	
See the medical device equipment <i>Instructions for Use</i> to find out if the equipment can be cleaned and disinfected with <i>ArjoClean</i> .	
No other combinations are allowed.	

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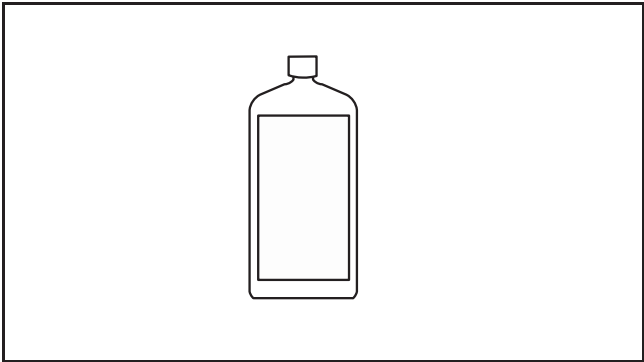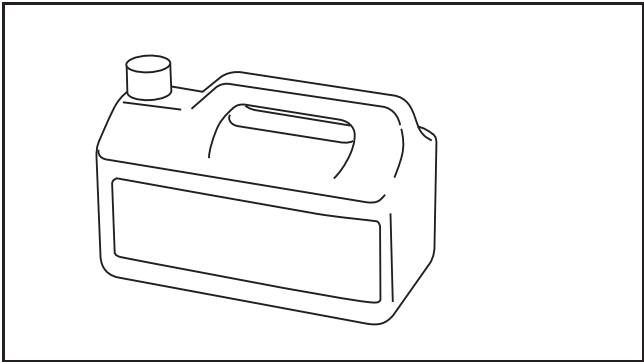HONG KONG

Arjo Hong Kong Limited
Room 411-414, 4/F, Manhattan Centre,
8 Kwai Cheong Road, Kwai Chung, N.T.,
HONG KONG
Tel: +852 2960 7600
Fax: +852 2960 1711

ITALIA

Arjo Italia S.p.A.
Via Giacomo Peroni 400-402
IT-00131 ROMA
Tel: +39 (0) 6 87426211
Fax: +39 (0) 6 87426222
E-mail: Italy.promo@arjo.com

MIDDLE EAST

Arjo Middle East FZ-LLC
Office 908, 9th Floor,
HQ Building, North Tower,
Dubai Science Park,
Al Barsha South
P.O Box 11488, Dubai,
United Arab Emirates
Direct +971 487 48053
Fax +971 487 48072
Email: Info.ME@arjo.com

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Address page - REV 24: 04/2019

At Arjo, we are committed to improving the everyday lives of people affected by reduced mobility and age-related health challenges. With products and solutions that ensure ergonomic patient handling, personal hygiene, disinfection, diagnostics, and the effective prevention of pressure ulcers and venous thromboembolism, we help professionals across care environments to continually raise the standard of safe and dignified care. Everything we do, we do with people in mind.

ArjoHuntleigh AB
Hans Michelsensgatan 10
211 20 Malmö, Sweden
www.arjo.com

arjo

CE
2797

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Arjo Inc.