




Flowtron ACS900

Reading the patient run timer — hours and minutes

- 1** Hours are displayed by default whenever the pump is powered on.
The “h” shows the value in hours (e.g., 12 h).

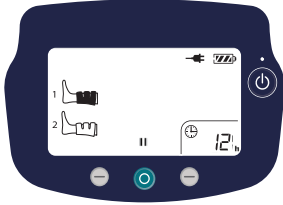

- 2** Press the right button for 1 second to display the minutes.

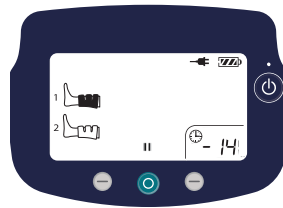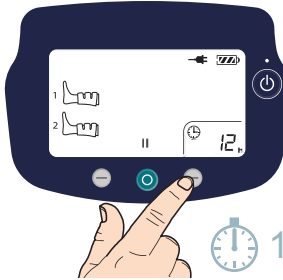


Minutes are displayed for 5 seconds when the button is pressed.

The minutes value is preceded by a hyphen (e.g., - 14 = 14 minutes).




- 3** Hours resume being displayed automatically.
Compression time = 12 hours, 14 minutes.

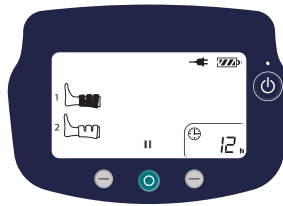


Resetting the patient run timer — hours and minutes

- 1** Place the pump into pause. Press and hold the right button for 2 seconds until the hours value flashes.

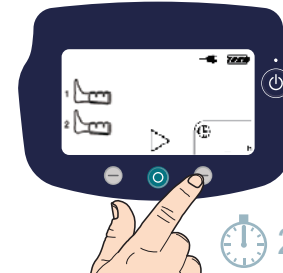

- 2** Press and hold the center button for 3 seconds to reset to zero.

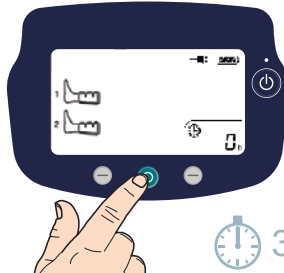

- 3** Press the right button to confirm the reset.

