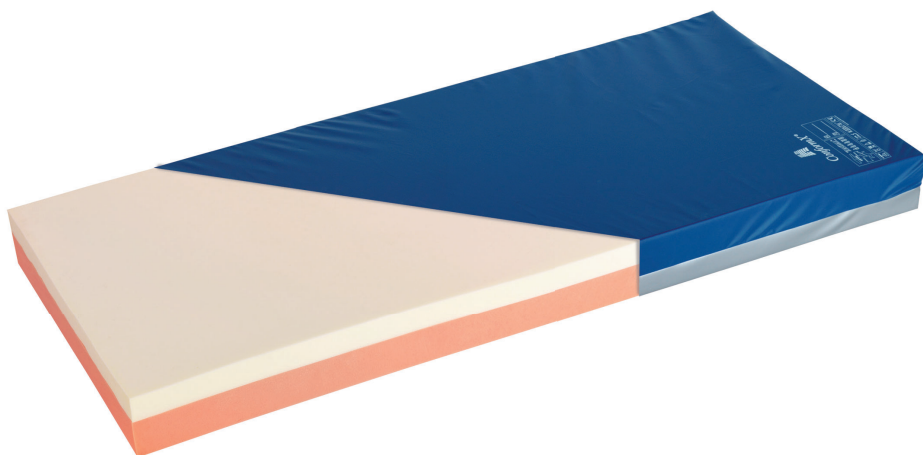
INSTRUCTIONS FOR USE

# ConformX

2-way turn mattress overlay

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Instructions for use · Brugsvejledning · Bedienungsanleitung · Instrucciones de uso  
Mode d'emploi · Uputama za upotrebu · Istruzioni per l'uso · Gebruiksaanwijzing  
Navodila za uporabo · Uputstvo za upotrebu · Bruksanvisning

## **WARNING**

**To avoid injury, always read this Instructions For Use and accompanied documents before using the product.**

**Mandatory to read the Instructions For Use.**

Design Policy and Copyright

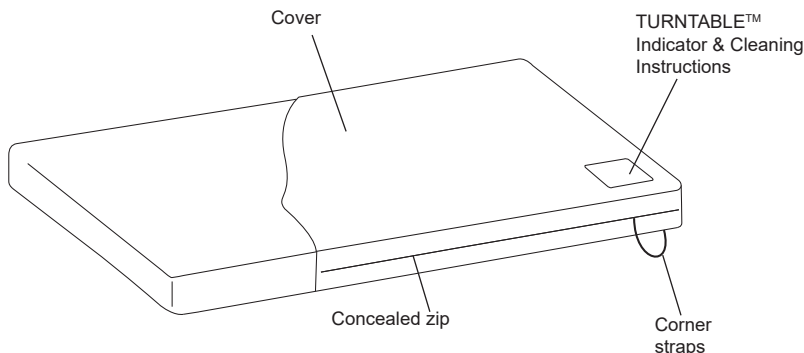
® and ™ are trademarks belonging to the Arjo group of companies.

© Arjo 2019.

As our policy is one of continuous improvement, we reserve the right to modify designs without prior notice. The content of this publication may not be copied either whole or in part without the consent of Arjo.

## Product Descriptions

The ConformX® overlay design comprises of a 2-way turn visco elastic foam with a 2-way stretch, vapour permeable, water resistant cover.

## Clinical applications

### Indications for Use

ConformX overlay is suitable for comfort only.

#### WARNING

The cover of this product is vapour permeable but not air permeable and may present a suffocation risk. It is the responsibility of the care giver to ensure that the user can use this product safely.

### Instructions for Use

1. Remove all packaging.
2. Lay the ConformX mattress overlay on top of a normal mattress, ensuring that there are no sharp edges which might rip the cover. Ensure that the contoured side of the mattress is uppermost.
3. Use the four corner straps to safely secure the overlay to the underlying mattress.
4. Ensure the correct month is showing on the Turntable™ indicator at the foot of the cover. If the incorrect month is showing, rotate the mattress to show correct month at the foot end. Months are indicated as follows:  
1/12 = January  
2/12 = February, etc.

5. Use the TURNTABLE™ indicator as a guide for the monthly rotation and turning of the mattress. This helps to even out wear of the mattress and to extend the life of the product.

#### CAUTION

Do not use the mattress without a cover. It provides a protective barrier against biological contamination of the foam.

## Decontamination

### To Clean the Cover

Wipe all exposed surfaces with a cloth which has been moistened in simple detergent and water. Ensure to remove any organic debris that may be adherent to the surfaces. Dry thoroughly.

### To Disinfect the Cover

For disinfection we recommend a chlorine-releasing agent such as NaOCl or NaDCC diluted to 1,000 ppm (range may vary between 250 & 10,000 ppm depending on contamination status and local policy).

Wipe with a cloth moistened with clean water and dry thoroughly.

Alternatively, use a cloth moistened with alcohol (70%) all allow to air dry.

## Routine checks

The foam core and cover should be inspected regularly for any signs of damage, soiling or staining.

## Technical Data

Typical bed size			Acute Single Size			Coummunity Standard Size	Community Double Size
Suitable for use on Arjo bed frame	20050C Child Anniversary™	Contoura™ 420 and 430	Contoura 350, 360, 460, 480, 560 and Minuet® 2	Minuet 1	Contoura 880 and 980		
Size (cm) L x W x D	170 x 86 x 9.5	182 x 78 x 9.5	198 x 86 x 9.5	200 x 90 x 9.5	202 x 88 x 9.5	190 x 92 x 9.5	190 x 138 x 9.5
ConformX Overlay Product Code	CO170/086	CO182/078	CO198/086	CO200/090	CO202/088	CO190/092	CO190/138
Material	Polyurethane foam						
Cover material	Polyurethane coated polyester fabric with enhanced durability						

ConformX overlays are available in other custom made sizes, e.g. COYYY/ZZZ, where YYY specifies the length (between 140 and 216cm) and ZZZ specifies the width (between 66 and 140cm). All sizes must be specified in increments of 2cm.

When combining beds and mattresses outside the table given above, it is the customer's responsibility to ensure compatibility of mattress and bed frame. Please refer to your bed manufacturers recommended mattress sizes. For further details contact Arjo.

### Fire retardancy:

System meets the requirements of BS EN 597-1: 1995, BS EN 597-2: 1995, and BS 6807: 2006, 5.

#### FIRE WARNING

**Cigarettes, lighters, naked flames and exposed heating elements e.g. in electric fires (and other electrical appliances) are sources of fire hazard and could ignite bed linen, blankets and clothing. It is essential that these fire sources are kept away from the mattress.**

**NB: It is highly dangerous to smoke in bed.**

## Disposal

- For ultimate disposal of ConformX products, we recommend the following advice be followed. However, regardless of this, please ensure that local regulations and, if applicable, healthcare facility standard operating procedures are followed.
- Foam cores suspected of contamination should be disposed of as clinical waste.
- Foam cores not suspected of contamination may be disposed of through landfill waste.

**Note: Follow all local regulations and, if applicable, healthcare facility standard operating procedures.**

## Warranty and Service

Standard terms and conditions apply to all sales. A copy is available upon request. These contain full details of warranty terms and do not limit the statutory rights of the consumer.

For service, maintenance, and any questions regarding this, or any other product, please contact your local distributor.

## Symbols

Recommended wash temperature:  
15 min at 60°C  
Maximum wash temperature: 15 min  
at 95°C

Do not iron

Do not use phenol-based cleaning  
solutions

Hospital name

Tumble dry at 60°C  
Maximum drying temperature 80°C

Wipe all surfaces with cleaning solution,  
then wipe with a cloth moistened with  
water and dry thoroughly.

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**HONG KONG**

Arjo Hong Kong Limited  
Room 411-414, 4/F, Manhattan Centre,  
8 Kwai Cheong Road, Kwai Chung, N.T.,  
HONG KONG  
Tel: +852 2960 7600  
Fax: +852 2960 1711

**ITALIA**

Arjo Italia S.p.A.  
Via Giacomo Peroni 400-402  
IT-00131 ROMA  
Tel: +39 (0) 6 87426211  
Fax: +39 (0) 6 87426222  
E-mail: Italy.promo@arjo.com

**MIDDLE EAST**

Arjo Middle East FZ-LLC  
Office 908, 9th Floor,  
HQ Building, North Tower,  
Dubai Science Park,  
Al Barsha South  
P.O Box 11488, Dubai,  
United Arab Emirates  
Direct +971 487 48053  
Fax +971 487 48072  
Email: Info.ME@arjo.com

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Address page - REV 24: 04/2019

At Arjo, we are committed to improving the everyday lives of people affected by reduced mobility and age-related health challenges. With products and solutions that ensure ergonomic patient handling, personal hygiene, disinfection, diagnostics, and the effective prevention of pressure ulcers and venous thromboembolism, we help professionals across care environments to continually raise the standard of safe and dignified care. Everything we do, we do with people in mind.

ArjoHuntleigh AB  
Hans Michelsensgatan 10  
211 20 Malmö, Sweden  
**[www.arjo.com](http://www.arjo.com)**

**arjo**

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Arjo Inc.