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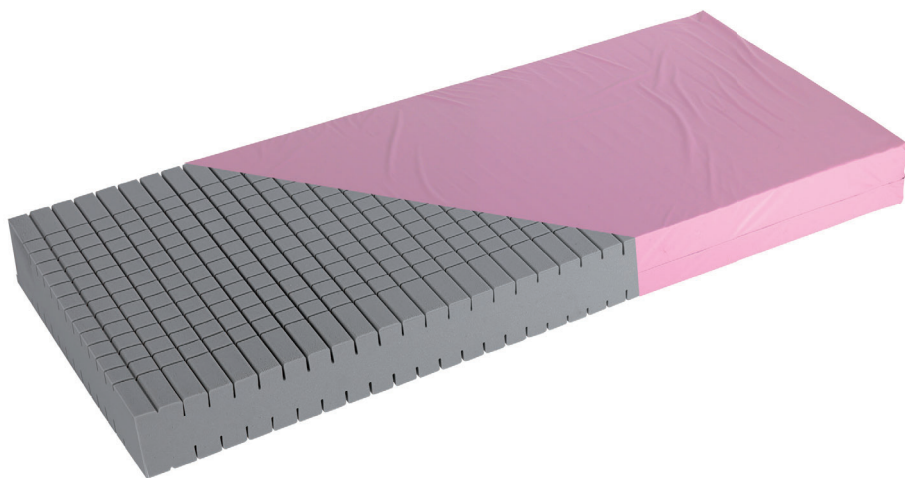
Pentaflex

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EN · ZH

Instructions for use · 使用说明书

WARNING

To avoid injury, always read this Instructions For Use and accompanied documents before using the product.

Mandatory to read the Instructions For Use.

Design Policy and Copyright

® and ™ are trademarks belonging to the Arjo group of companies.

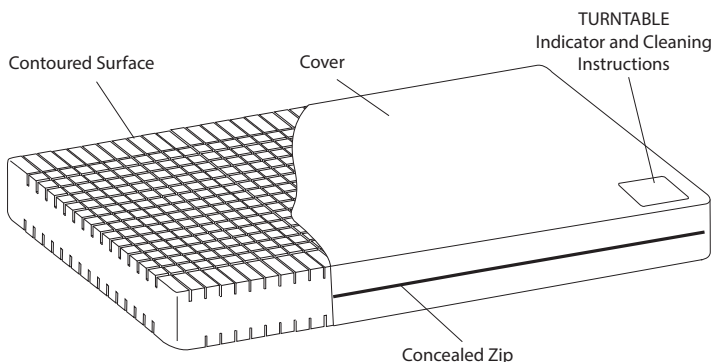
© Arjo 2019.

As our policy is one of continuous improvement, we reserve the right to modify designs without prior notice. The content of this publication may not be copied either whole or in part without the consent of Arjo.

Product Descriptions

Mattress replacement system

The Pentaflex® mattress replacement design comprises of a reversible 4-way turn, high resilience polyurethane core with 5 anatomical pressure redistributing zones per side, protected by a 2-way stretch, vapour permeable, water resistant cover.

A *Pentaflex* foam squab is available for use with in-built bed extensions, for more details refer to the user leaflet 609903.

Indications for Use

Pentaflex 120 is indicated for users up to 120 kg (264 lb / 19 stones). *Pentaflex* 250 is indicated for users up to 250 kg (550 lb / 39 stones).

WARNING

The cover of this product is vapour permeable but not air permeable and may present a suffocation risk. It is the responsibility of the care giver to ensure that the user can use this product safely.

Instructions for Use

Mattress replacement system:

1. Remove all packaging.
2. Lay the *Pentaflex* mattress replacement onto the bed frame, ensuring that there are no sharp edges which might rip the cover.

CAUTION

Do not use the mattress without a cover. It provides a protective barrier against biological contamination of the foam.

3. Ensure the correct month is showing on the Turntable™ indicator at the foot of the cover. If the incorrect month is showing, rotate the mattress to show correct month at the foot end. Months are indicated as follows:
1/12 = January
2/12 = February, etc.
4. Use the *Turntable* indicator as a guide for the monthly rotation and turning of the mattress. This helps to even out wear of the mattress and to extend the life of the product.

Decontamination

To Clean the Cover

Wipe all exposed surfaces with a cloth which has been moistened in simple detergent and water. Ensure to remove any organic debris that may be adherent to the surfaces. Dry thoroughly.

To Disinfect the Cover

For disinfection we recommend a chlorine-releasing agent such as NaOCl or NaDCC diluted to 1,000 ppm (range may vary between 250 & 10,000 ppm depending on contamination status and local policy).

Wipe with a cloth moistened with clean water and dry thoroughly.

Alternatively, use a cloth moistened with alcohol (70%) all allow to air dry.

Routine checks

The foam core and cover should be inspected regularly for any signs of damage, soiling or staining.

Technical Data

Typical Bed size	-	-	Acute Single Size	-	-	Community Standard Size	Community Double Size
Suitable for use on Arjo Bedframe	20050C Child Anniversary™	Contura™ 420 and 430	Contura 350, 360, 460, 480, 560 and Minuet® 2	Minuet 1	Contura 880 and 980	-	-
Size (cm) L x W x D	170 x 86 x 17	182 x 78 x 17	198 x 86 x 17	200 x 90 x 17	202 x 88 x 17	190 x 92 x 17	190 x 138 x 17
PENTAFLEX 120 Product Code	120PM 170/086	120PM 182/078	120 PM 198/086	120PM 200/090	120PM 202/088	120PM 190/092	120PM 190/138
PENTAFLEX 250 Product Code	250PM 170/086	250PM 182/078	250PM 198/086	250PM 200/090	250PM 202/088	250PM 190/092	250PM 190/138
Material	High resilience polyurethane foam						
Cover Material	Polyurethane coated polyester fabric with enhanced durability						

Pentaflex mattresses are available in other custom made sizes, e.g. 120PMYYY/ZZZ, where YYY specifies the length (between 140 and 216 cm) and ZZZ specifies the width (between 66 and 140 cm). All sizes must be specified in increments of 2 cm.

When combining beds and mattresses outside the table given above, it is the customer's responsibility to ensure compatibility of mattress and bed frame. Please refer to your bed manufacturers recommended mattress sizes. For further details contact Arjo.

Fire retardancy:

System meets the requirements of BS EN 597-1: 1995, BS EN 597-2: 1995, and BS 6807: 2006, 5.

FIRE WARNING

Cigarettes, lighters, naked flames and exposed heating elements e.g. in electric fires (and other electrical appliances) are sources of fire hazard and could ignite bed linen, blankets and clothing. It is essential that these fire sources are kept away from the mattress.

NB: It is highly dangerous to smoke in bed.

End of Life Disposal

For ultimate disposal of *Pentaflex* products, we recommend the following advice be followed.

- Foam cores suspected of contamination should be disposed of as clinical waste.
- Foam cores not suspected of contamination may be disposed of through landfill waste.

Note: Follow all local regulations and, if applicable, healthcare facility standard operating procedures.

Warranty and Service

Standard terms and conditions apply to all sales. A copy is available upon request. These contain full details of warranty terms and do not limit the statutory rights of the consumer.

For service, maintenance, and any questions regarding this, or any other product, please contact your local distributor.

Symbols

Recommended wash temperature:
15 min at 60°C
Maximum wash temperature: 15 min
at 95°C

Do not iron

Do not use phenol-based cleaning
solutions

Hospital name

Tumble dry at 60°C
Maximum drying temperature 80°C

Wipe all surfaces with cleaning solution,
then wipe with a cloth moistened with
water and dry thoroughly.

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At Arjo, we are committed to improving the everyday lives of people affected by reduced mobility and age-related health challenges. With products and solutions that ensure ergonomic patient handling, personal hygiene, disinfection, diagnostics, and the effective prevention of pressure ulcers and venous thromboembolism, we help professionals across care environments to continually raise the standard of safe and dignified care. Everything we do, we do with people in mind.

ArjoHuntleigh AB
Hans Michelsensgatan 10
211 20 Malmö, Sweden
www.arjo.com

arjo