
INSTRUCTIONS FOR USE

Pentaflex

2-way turn mattress replacement system

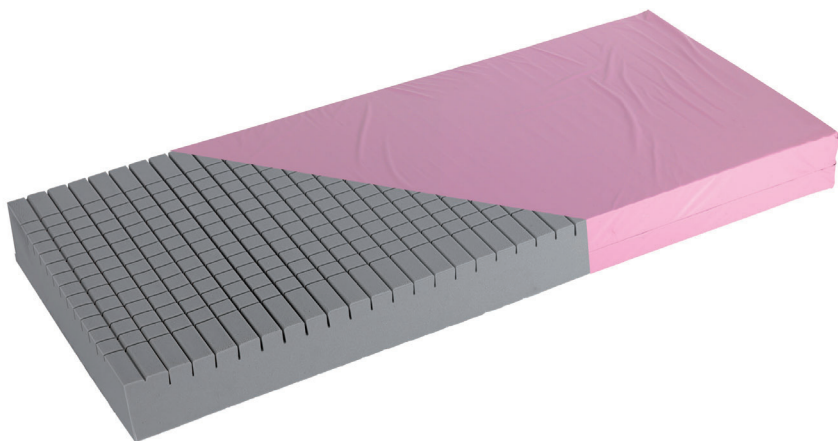EN · ES · FR

Instrucciones de uso · Mode d'emploi

WARNING

To avoid injury, always read this Instructions For Use and accompanied documents before using the product.

Mandatory to read the Instructions For Use.

Design Policy and Copyright

® and ™ are trademarks belonging to the Arjo group of companies.


© Arjo 2019.

As our policy is one of continuous improvement, we reserve the right to modify designs without prior notice. The content of this publication may not be copied either whole or in part without the consent of Arjo.

Product Descriptions

Mattress replacement system

The Pentaflex® mattress replacement design comprises of a 2-way turn, high resilience polyurethane core with 5 anatomical pressure redistributing zones, protected by a 2-way stretch, vapour permeable, water resistant cover.

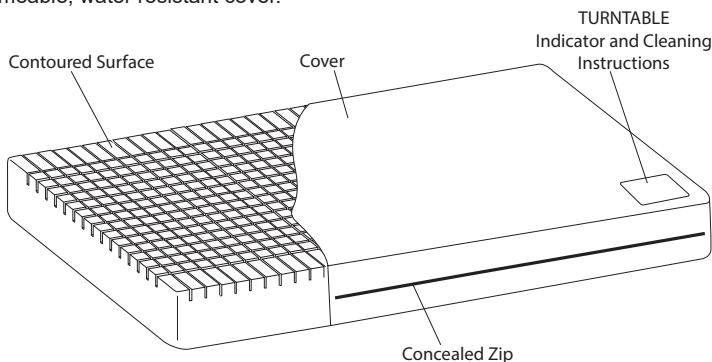A *Pentaflex* foam squab is available for use with in-built bed extensions, for more details refer to the user leaflet 609903.

Clinical applications

Indications for Use

Pentaflex 150 is indicated for users up to 150 kg (330 lb). *Pentaflex* 250 is indicated for users up to 250 kg (550 lb). Suitable for a wide range of patients including those with superficial ulcers (in association with an individualised plan of care).

Note: These are guidelines only and should not replace clinical experience or judgment. If there is no improvement in the patient's condition, specialist advice should be sought.

WARNING

The cover of this product is vapour permeable but not air permeable and may present a suffocation risk. It is the responsibility of the care giver to ensure that the user can use this product safely.

Instructions for Use

1. Remove all packaging.
2. Lay the *Pentaflex* mattress replacement system onto the bed frame, ensuring that there are no sharp edges which might rip the cover.

CAUTION

Do not use the mattress without a cover. It provides a protective barrier against biological contamination of the foam and helps extend the life of the product.

3. Ensure the correct month is showing on the Turntable™ indicator at the foot of the cover. If the incorrect month is showing, rotate the mattress to show correct month at the foot end. Months are indicated as follows:
1/12 = January
2/12 = February, etc.
4. Use the *Turntable* indicator as a guide for the monthly rotation and turning of the mattress. This helps to even out wear of the mattress and to extend the life of the product.

WARNING

Alignment of the bed frame, safety sides and the mattress should leave no gap wide enough to entrap a patient's head or body, or to allow egress to occur in a hazardous manner where entanglement with the mains power cable and tubeset or air hoses may result. Care should be exercised to prevent occurrence of gaps by compression or movement of the mattress. Death or serious injury may occur.

Decontamination

To Clean the Cover

Wipe all exposed surfaces with a cloth which has been moistened in simple (neutral) detergent and water. Ensure to remove any organic debris that may be adherent to the surfaces. Dry thoroughly.

To Disinfect the Cover

For disinfection we recommend a chlorine-releasing agent such as NaOCl or NaDCC diluted to 1000 ppm (range may vary between 250 & 10,000 ppm depending on contamination status and local policy).

Wipe with a cloth moistened with clean water and dry thoroughly.

Alternatively, use a cloth moistened with alcohol (70%) all allow to air dry.

Routine checks

The foam core and cover should be inspected regularly for any signs of damage, soiling or staining.

Technical Data

	Product Code	Size (cm)	Size (in)	Weight (kg)	Weight (lb)
PENTAFLEX 150	150SL202/088NA	202 x 88 x 15	80 x 35 x 6	150	330
	150SL212/088NA	212 x 88 x 15	84 x 35 x 6	150	330
	150SL198/086NA	198 x 86 x 15	79 x 34 x 6	150	330
PENTAFLEX 250	250SL202/088NA	202 x 88 x 15	80 x 35 x 6	250	550
	250SL212/088NA	212 x 88 x 15	84 x 35 x 6	250	550
	250SL198/086NA	198 x 86 x 15	79 x 34 x 6	250	550

Fire retardancy:

System meets the requirements of 16 CFR Part 1633.

FIRE WARNING

Cigarettes, lighters, naked flames and exposed heating elements e.g. in electric fires (and other electrical appliances) are sources of fire hazard and could ignite bed linen, blankets and clothing. It is essential that these fire sources are kept away from the mattress.

NB: It is highly dangerous to smoke in bed.

End of Life Disposal

For ultimate disposal of *Pentaflex* products, we recommend the following advice be followed.

- Foam cores suspected of contamination should be disposed of as clinical waste.
- Foam cores not suspected of contamination may be disposed of through landfill waste.

Note: Follow all local regulations and, if applicable, healthcare facility standard operating procedures.

Warranty and Service

Standard terms and conditions apply to all sales. A copy is available upon request. These contain full details of warranty terms and do not limit the statutory rights of the consumer.

For service, maintenance, and any questions regarding this, or any other product, please contact your local distributor.

Symbols

Recommended wash temperature:
15 min at 60°C (140°F)
Maximum wash temperature:
15 min at 95°C (203°F)

Do not iron

Do not use phenol-based cleaning solutions

Hospital name

Tumble dry at 60°C (140°F)
Maximum drying temperature
80°C (176°F)

Wipe all surfaces with cleaning solution, then wipe with a cloth moistened with water and dry thoroughly.

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HONG KONG

Arjo Hong Kong Limited
Room 411-414, 4/F, Manhattan Centre,
8 Kwai Cheong Road, Kwai Chung, N.T.,
HONG KONG
Tel: +852 2960 7600
Fax: +852 2960 1711

ITALIA

Arjo Italia S.p.A.
Via Giacomo Peroni 400-402
IT-00131 ROMA
Tel: +39 (0) 6 87426211
Fax: +39 (0) 6 87426222
E-mail: Italy.promo@arjo.com

MIDDLE EAST

Arjo Middle East FZ-LLC
Office 908, 9th Floor,
HQ Building, North Tower,
Dubai Science Park,
Al Barsha South
P.O Box 11488, Dubai,
United Arab Emirates
Direct +971 487 48053
Fax +971 487 48072
Email: Info.ME@arjo.com

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Address page - REV 24: 04/2019

At Arjo, we are committed to improving the everyday lives of people affected by reduced mobility and age-related health challenges. With products and solutions that ensure ergonomic patient handling, personal hygiene, disinfection, diagnostics, and the effective prevention of pressure ulcers and venous thromboembolism, we help professionals across care environments to continually raise the standard of safe and dignified care. Everything we do, we do with people in mind.

ArjoHuntleigh AB
Hans Michelsensgatan 10
211 20 Malmö, Sweden
www.arjo.com

arjo

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Arjo Inc.