


Sara Combilizer



Pinch protection bar


1

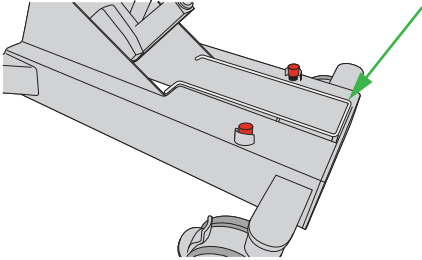

Press down the Pinch protection bar to automatically stop any downward movement.

Emergency stop buttons

1

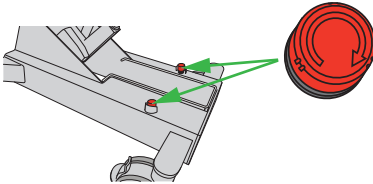


Press Emergency stop button(s) to cut all power to all electrical components.

2



Emergency function box

1

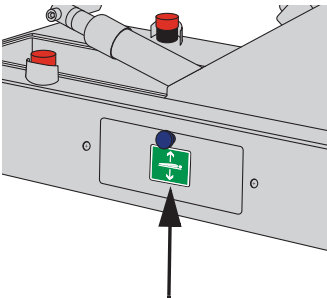

Located on the lower frame (right hand side).

2




3



Tilt table release function

1



Quick lowering of patient from standing to horizontal position. Two caregivers may be required.

2




How to use the Tilt table release function


1



Apply brakes on Sara Combilizer

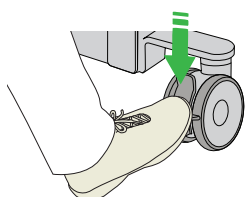

2

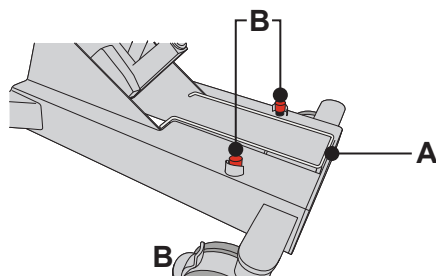


Do not activate Pinch protection bar (A) or Emergency buttons (B).

3




Keep patient's head on pillow or mattress during the whole process.

4

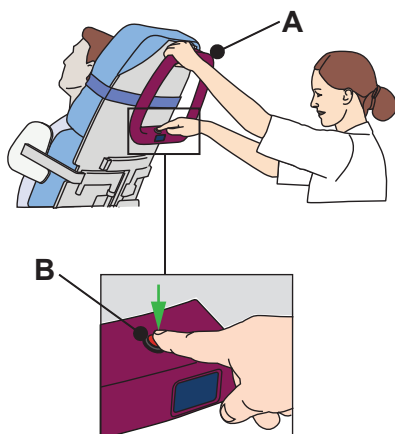


Push transport handle (A). Press Tilt table release button (B) until a "click" is heard. Beeping sound starts.

5

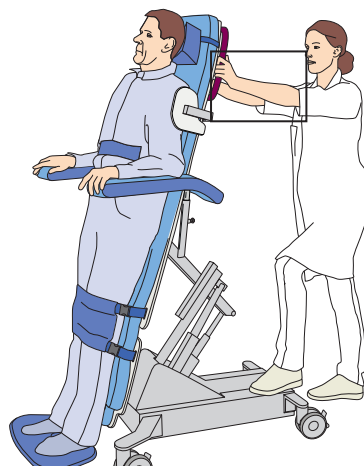

Position both hands firmly on transport handle. For better support keep one foot on the chassis beam.

6

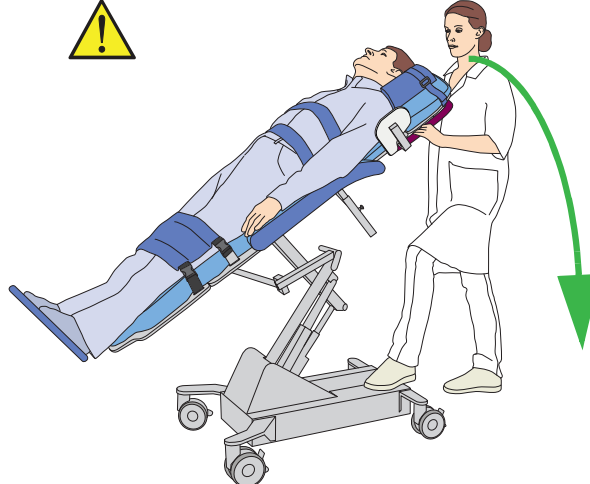


Pull tilt table down to full horizontal position. For support keep one foot on the chassis beam.

7




Press stretcher down until beeping sound stops. Sara Combilizer is locked in horizontal position in Normal mode.

8

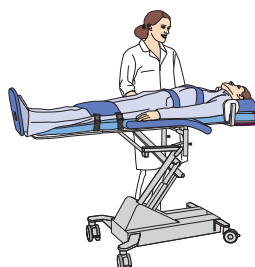

If needed, adjust stretcher height.

9

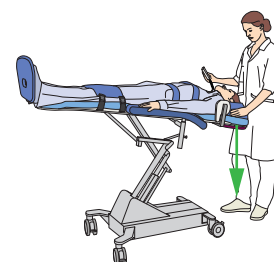

If needed, place patient in the Trendelenburg position.