


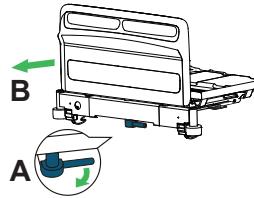
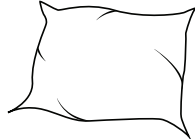

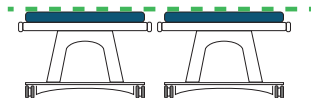
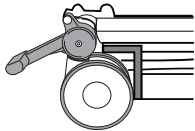
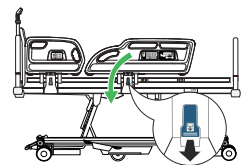
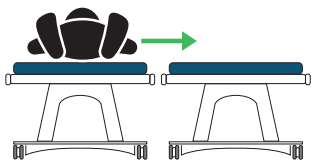
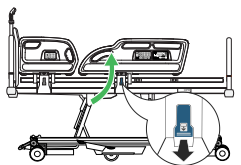


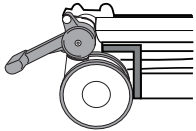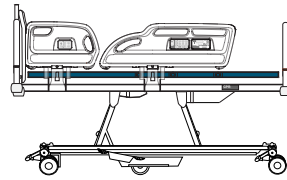

Citadel Plus

Patient placement – preparation

<p>1</p>  <p>Apply caster brakes.</p>	<p>2</p>  <p>Level patient surface.</p>	<p>3</p>  <p>Adjust bed to a comfortable working height.</p>	<p>4</p>  <p>Extend the frame and mattress platform, if required.</p>	<p>5</p>  <p>Add pillows, blankets and other equipment.</p>	<p>6</p>  <p>Press Zero Button to reset weighing scale to zero.</p>
---	---	---	---	---	---

<p>7</p>  <p>Adjust height of the patient surface to the same as the patient transfer surface.</p>	<p>8</p>  <p>Apply caster brakes on all units.</p>	<p>9</p>  <p>Lower side rails.</p>	<p>10</p>  <p>Transfer patient, following all applicable safety rules and institution protocols.</p>	<p>11</p>  <p>Raise and lock all side rails.</p>	<p>Weigh the patient</p> <p> Not for medicine dosage</p>  <p>Weigh the patient, if required.</p>
--	---	--	---	---	--

Patient placement – completion

1




Press In Bed or Egress buttons to activate and set sensitivity of VariZone patient movement detection system.

2



Bring the bed to the lowest practical height for patient comfort.

3

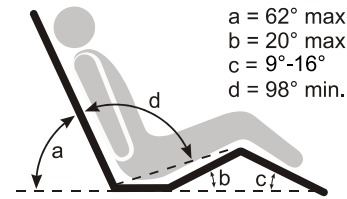

Bring the bed to the desired angle for patient comfort.


4



Lock-out bed functions, if required.


Patient transfer from bed

1

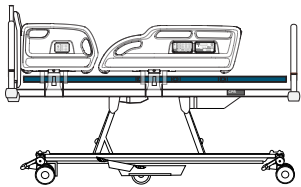


Level patient surface.

2

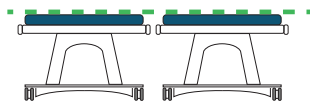


Adjust height of the patient surface to the same as the patient transfer surface.

3

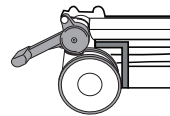


Apply caster brakes on all units.

4

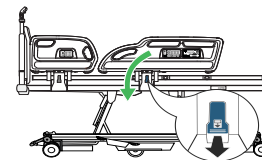

Lower side rails.

5

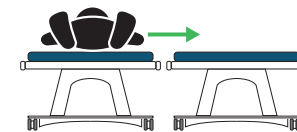

Transfer patient following all applicable safety rules and institution protocols.


6



If the patient will not return:
- Unplug power cord and press battery power button.
- Disable battery power and power down all functions.


Patient transport

1

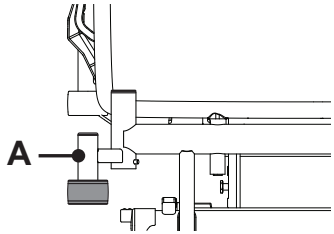

Place patient's IV therapy onto IV poles, if required.

2

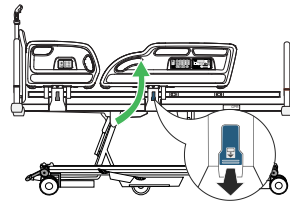


Raise and lock side rails.

3




Push in all width extensions before transport.

4

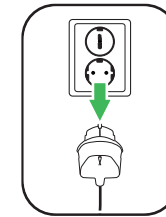


Unplug power cord from wall outlet.

5

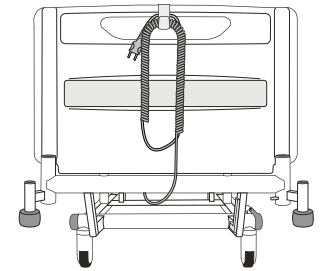

Wrap cord on to brackets at the head end of the bed.

6

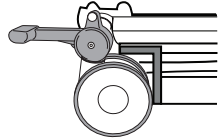


Release the brakes.

7




Use power drive to assist in transport.

8

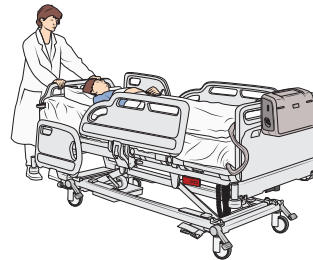

Transport patient, following all applicable safety rules and institution protocols.

9

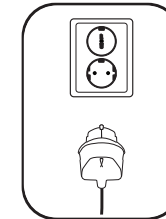

Plug power cord into wall outlet after patient transport.