

MaxxAir ETS



Install mattress replacement system

1

Level bed and apply brakes.
Remove existing mattress.

2

Undo storage straps.
Place storage straps in pouch or store in turning bladder area.

3

Position the mattress on the bed frame with hose set at foot end of bed.

4

Secure mattress to bed frame using straps.


5

Place mattress cover, make sure:

- mattress cover is not wrinkled
- buckles on patient's left head side and right foot side of cover are properly connected under mattress


If hose connections are attached, they do not need to be disconnected.

6

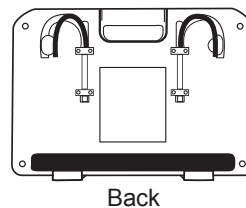


Secure therapy unit on footboard of bed by extending the arms.

7

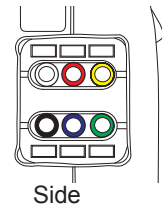

Attach hose connectors to therapy unit. Colour match hoses and sockets.

8

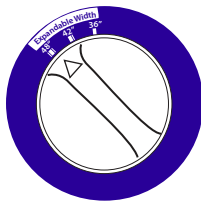


Adjust width of mattress using the expansion knob.

36 inch setting: No expansion.

42 inch setting: Patient left side surface expands.

48 inch setting: Both sides of patient surface expand.

9

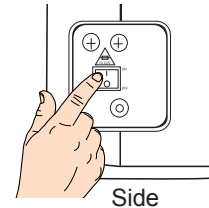

Plug power cord into grounded power source.

Route the cord under the bed frame.

Turn power ON, the AIR OFF LED on front panel turns on.

Prepare transfer of *non-ambulatory* patient


1



On front panel:

Push AIR ON button. The AIR ON LED turns green and the therapy unit starts to run.

2

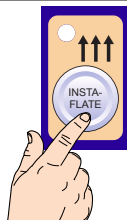

Push INSTAFLATE button for fast inflation. Allow four to five minutes for full inflation.


3

Wait until surface is fully inflated.

Verify all brakes are applied.

Adjust bed height for patient transfer or placement.

4

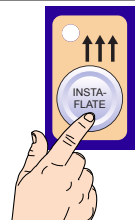

Push INSTAFLATE button to deactivate the fast inflation.

Raise and lock side rails and make sure any positioning aids are properly in place.



Adjust bed height to lowest level.

Prepare transfer of *ambulatory* patient

Ambulatory patients enter and exit a deflated unit.

- Verify all brakes are applied.
- Adjust bed height to lowest level.
- With Caregiver assistance, ambulatory patient should be properly positioned and enter at side of bed.

CPR
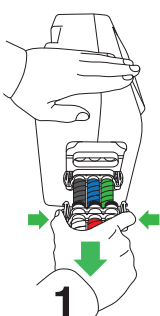
<div data-bbox="65 454 103 510">1</div>  <div data-bbox="359 454 397 510">2</div>  <p>Firmly place hand on therapy unit and disconnect the 2 hose sets.</p>	<div data-bbox="809 454 847 510">2</div> <p>Begin CPR. Patient's body weight and CPR process deflates mattress.</p>	<div data-bbox="1179 454 1217 510">3</div> <p>After CPR is performed and patient is clinically stable, reinsert hose set and adjust Comfort Control to its original setting. Raise Side Rails.</p>
---	--	---