


MaxxAir ETS




Adjust air pressure

1

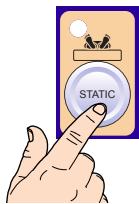


Push STATIC button.

2

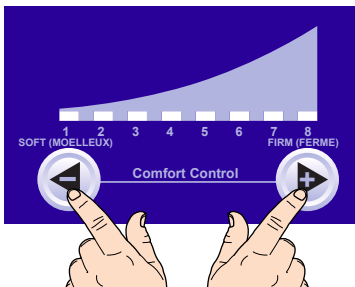

Adjust the Comfort Control by pressing the SOFT/ FIRM key to achieve the maximum patient comfort and proper air pressure adjustment.

3

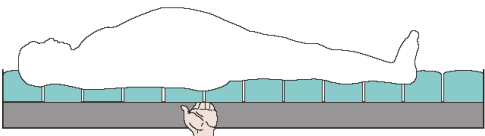


Verify placement:

- Place hand under the patient's buttocks between cells and foam. The patient should have at least 1.5 in (35 mm) of clearance between buttocks and bottom of mattress.
- Verify air pressure with hand beneath shoulders, hips and heels during turn cycles.
- Perform hand check on both sides of the mattress.

4


Verify patient comfort and air pressure adjustments with patient in sitting and side lying positions.

Turning function




Side rails in full upright and locked position.

1

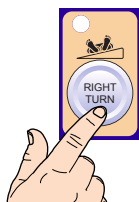


Press RIGHT TURN button for right hold.
Cycle time displayed: zero-zero (00).

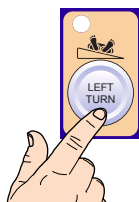


Press LEFT TURN button for left hold.
Cycle time displayed: zero-zero (00).

2

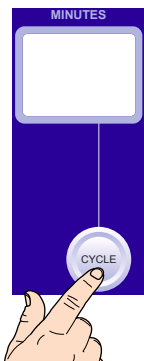


Press the CYCLE button to enable mattress to turn to right **or** left and back to center and adjust turn time from 3-30 minutes. Cycle time can be adjusted from 3-20 minutes in one-minute increments and from 20-30 minutes in five-minute increments.

3

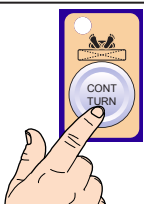


Press CONT TURN Button to continuously turn mattress to the right, back to center, to the left, and back to center.

4

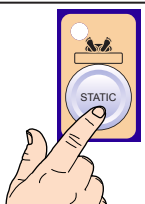

Press STATIC to deactivate all functions.

Reset alarm



The alarm sounds when power has been interrupted or turned off.

If this occurs, push ALARM RESET button on front of unit, and/or re-apply power.