


# Bolero

## Transfer from bed




To avoid entrapment, make sure the path of the movement is free from obstacles.

1

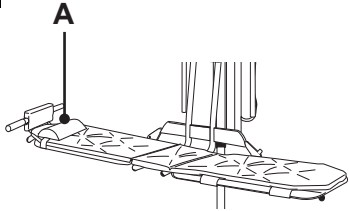


Fasten the pillow (A) at the desired head end.

2

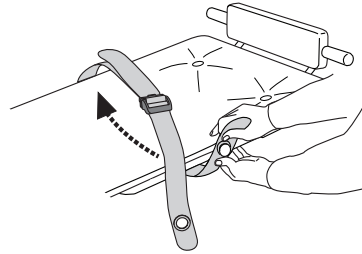


Attach the safety belt for the hip through to the knob. Place safety belt on top of the stretcher.

3

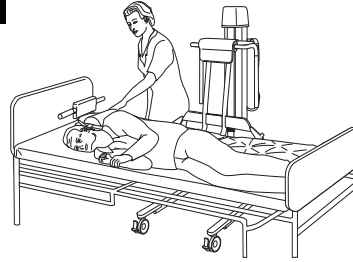


Roll the patient to their side. Move the Bolero lift bath trolley close to the patient and lower it so that it is resting firmly on the bed.

4

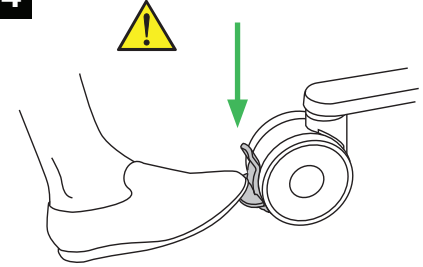


Apply brakes on all castors.

5

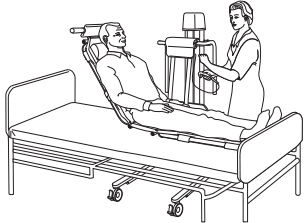


Roll the patient onto the stretcher. Raise the backrest. Check that the backrest is locked.

6

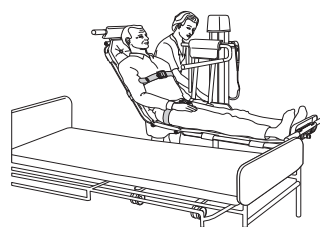


Fasten the safety belt over the chest. Raise the stretcher, fasten the safety belt over the hip. Lower the handle bar.

7

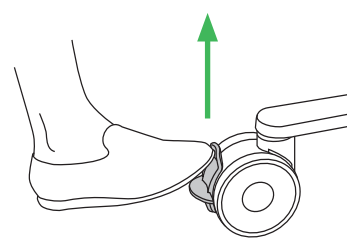

Release the castor brakes.

8

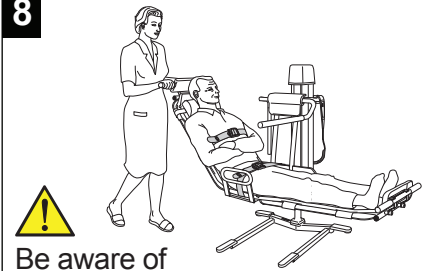

Be aware of stationary objects when lowering.

Pull the Bolero lift bath trolley out from the bed and lower Bolero to a comfortable height.


## Transfer to bed

1




Raise the Bolero lift bath trolley to a height where it is possible to move it in over the bed.

2

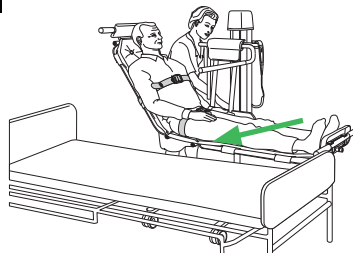


Move the Bolero lift bath trolley in over the bed.

3

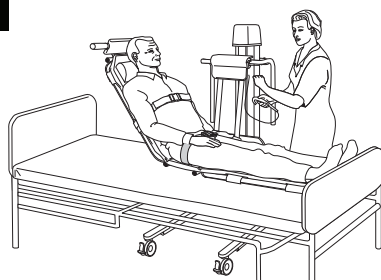


Lower so that the Bolero lift bath trolley is resting firmly on the bed.

4

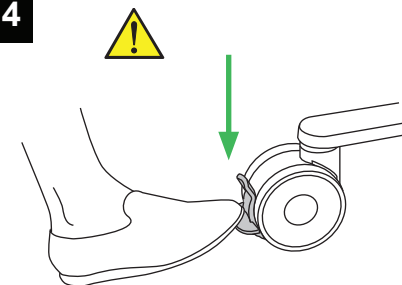

Apply brakes on all castors.


5

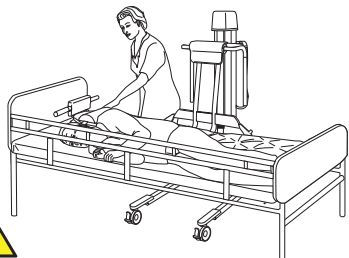

Make sure the side rail is up on the bed.

Remove the safety belts.  
Lower back rest.  
Log roll the patient off the Bolero lift bath trolley on to the bed.

6

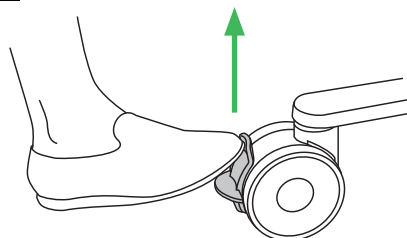

Release the castor brakes and pull the Bolero lift bath trolley out from the bed.

## Bath the patient


1

Read the separate instructions for your bath system.

### NOTE

Read safety instructions in the IFU before bathing.


2

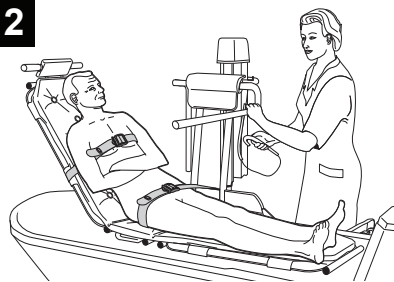

Positioned the patient in accordance with the IFU.

Raise the Bolero lift bath trolley to clear the edge. Move the patient over the bathtub.

3

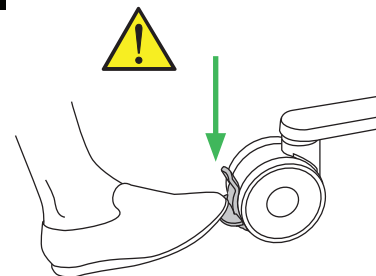

Apply brakes on all castors.

4




Raise the bathtub to an ergonomic height. Lower the Bolero lift bath trolley so that it rests against the bottom of the bathtub.

5


Remove the safety belt over the chest and loosen the safety belt over the hip then bath the patient.

6

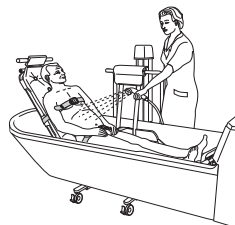


After bathing, fasten safety belts. Raise the Bolero lift bath trolley above the water, then give the patient a shower. Dry the patient after the shower.

7

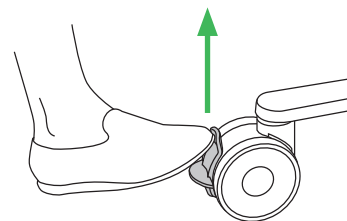

Release the castor brakes.

8

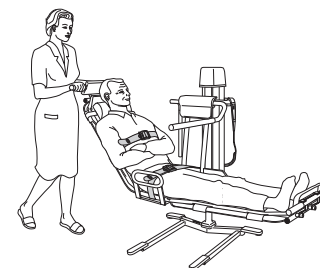

Return the patient to their bed.

