

Maxi 500



It is recommended that your *Maxi 500* and its accessories are cleaned and disinfected between each patient use, or daily as a minimum.

Pay special attention to parts that are most likely to be touched such as:

- Hand control
- Control box
- Battery pack
- Lift handle
- Spreader bar

Clean

1. Use a damp cloth with warm water to which a cleaner has been added.

CAUTION

Never use a wet cloth on the control box, the battery support or the battery pack as it may cause corrosion on electrical components.

If a hot air dryer is used to dry the lift, the temperature must not exceed 80°C (176°F.) Do not use petroleum based solvents or similar, as this may damage plastic parts.

Disinfect

2. Use a damp cloth with warm water to which a disinfectant has been added.

