



Flowtron ACS900


Pokretanje terapije

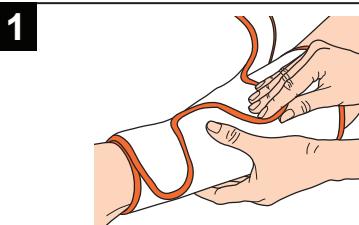

Postavite kompresivnu navlaku na pacijenta.

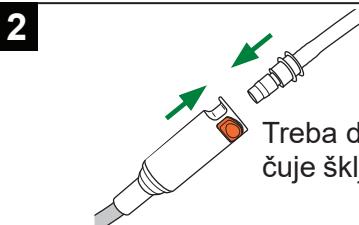


Treba da se čuje šklijocanje.
Priključite kompresivnu navlaku na pumpu.

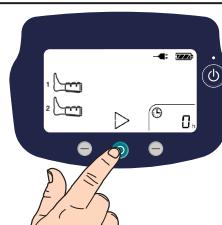


Pritisnite srednje kontrolno dugme da biste pokrenuli.


Zaustavljanje terapije

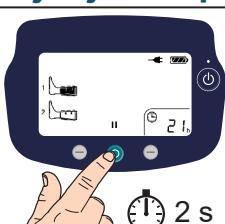

Pritisnite i zadržite srednje kontrolno dugme.

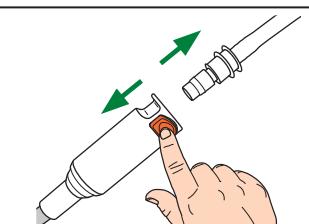

Isključite kompresivnu navlaku iz pumpe.

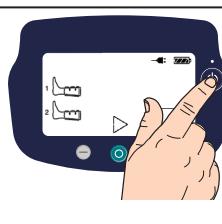

Pritisnite i zadržite dugme za napajanje da biste isključili pumpu.


Očitavanje vremena rada po pacijentu – sati i minuti (od verzije softvera 2.210)



Sati se prikazuju kada je pumpa uključena. (12 č)



Pritisnite desno kontrolno dugme da bi se prikazali minuti.

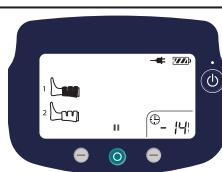


Minuti se prikazuju na 5 sekundi. (-14 = 14 min).




Sati će automatski biti ponovo prikazani.
Vreme kompresije = 12 sati, 14 minuta.


Resetovanje vremena rada po pacijentu – sati i minuti

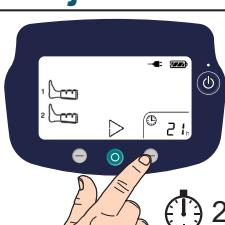

Pritisnite i zadržite desno kontrolno dugme.

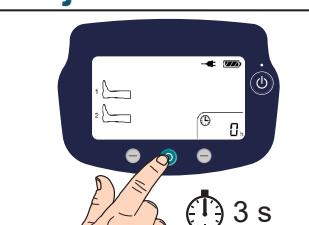


Pritisnite i zadržite srednje kontrolno dugme da biste resetovali na 0.

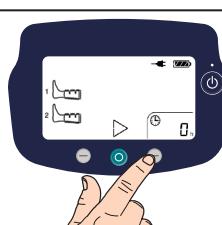


Pritisnite desno kontrolno dugme da biste sačuvali novo podešavanje.


Promena podešavanja jačine zvučnog alarma

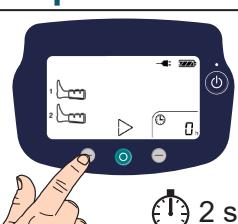

Pritisnite levo kontrolno dugme.

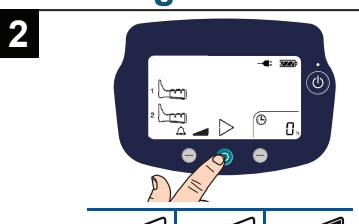

Pritisnite srednje kontrolno dugme da biste promenili jačinu zvuka.

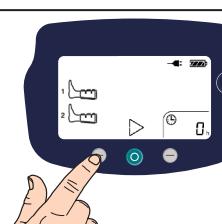

Pritisnite levo kontrolno dugme da biste sačuvali.



ArjoHuntleigh AB
Hans Michelsensgatan 10
211 20 Malmö, Sweden

www.arjo.com