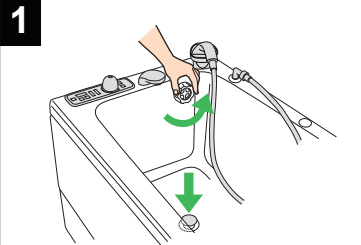


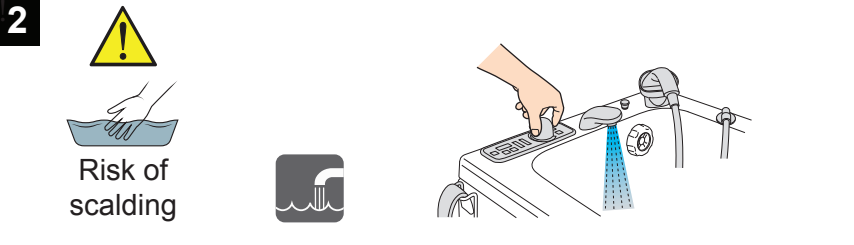
Malibu/Sovereign



Assisted bathing with transfer chair

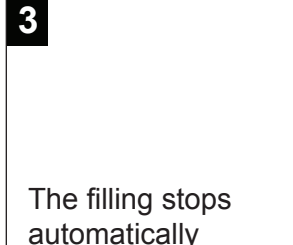
1

Close the drain.

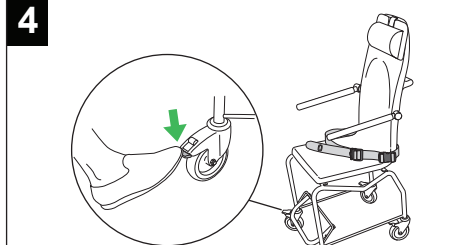
2

Risk of scalding

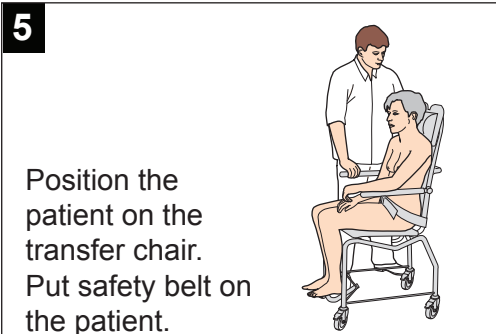
Push auto fill button. Feel the water with a bare hand. Adjust temperature if needed.

3

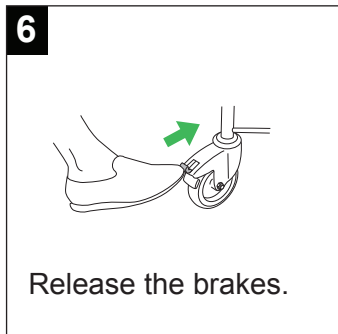
The filling stops automatically when the bathtub is full.

4

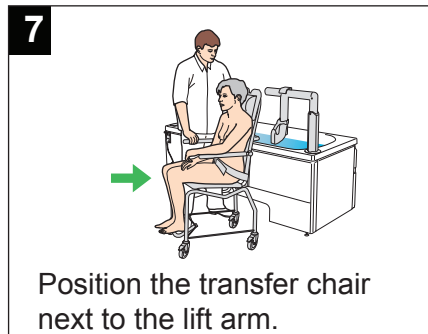
Apply the brakes.

5

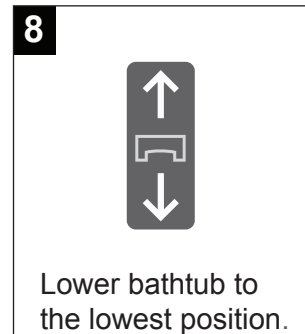
Position the patient on the transfer chair. Put safety belt on the patient.

6

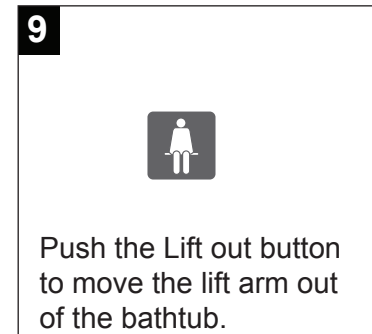
Release the brakes.

7

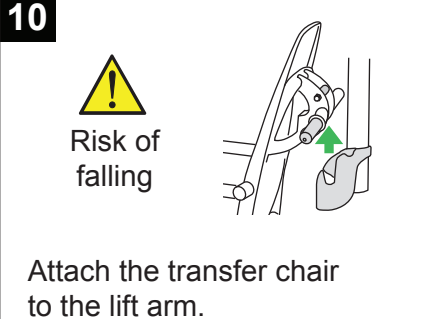
Position the transfer chair next to the lift arm.

8

Lower bathtub to the lowest position.


9

Push the Lift out button to move the lift arm out of the bathtub.

10

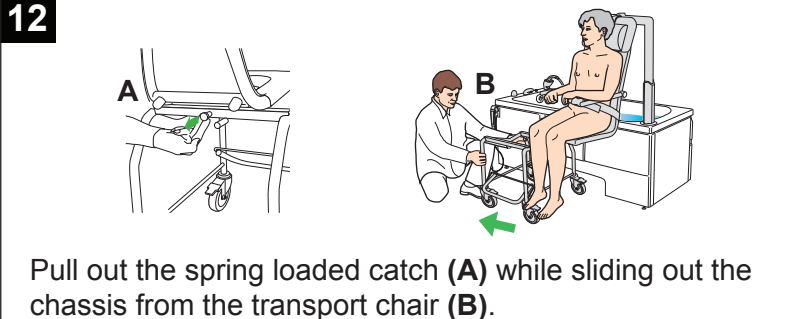
Risk of falling

Attach the transfer chair to the lift arm.

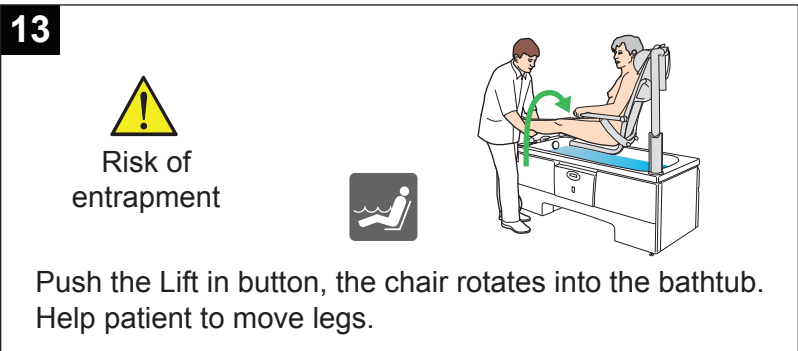
11

Risk of entrapment

Raise the lift arm by pushing down the Lift in button so that the transfer chair is secured on the lift arm and the chassis is clear of the floor.


12

Pull out the spring loaded catch (A) while sliding out the chassis from the transport chair (B).

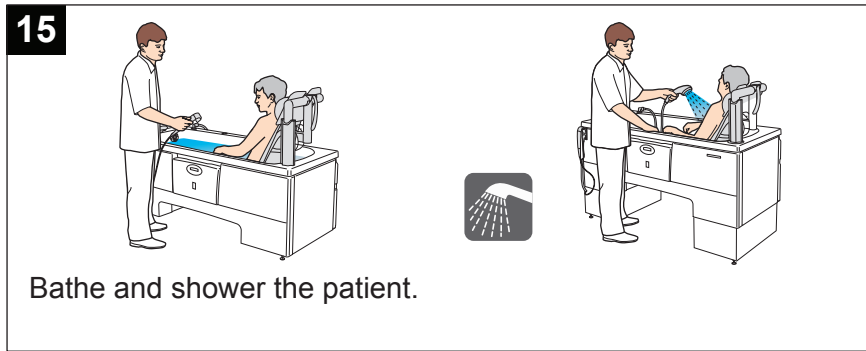
13

Risk of entrapment

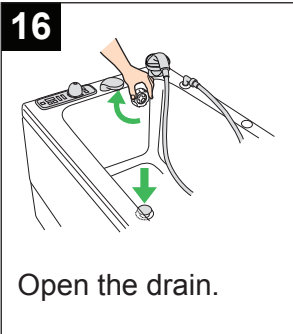
Push the Lift in button, the chair rotates into the bathtub. Help patient to move legs.

14

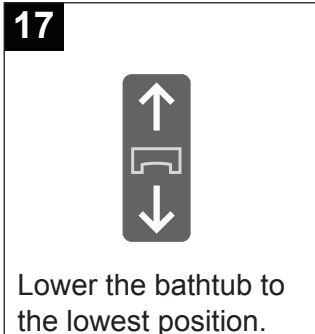
Adjust bathtub height to suit caregiver.

15

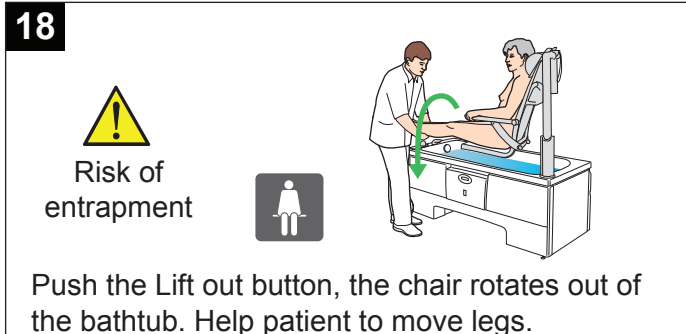
Bathe and shower the patient.

16

Open the drain.

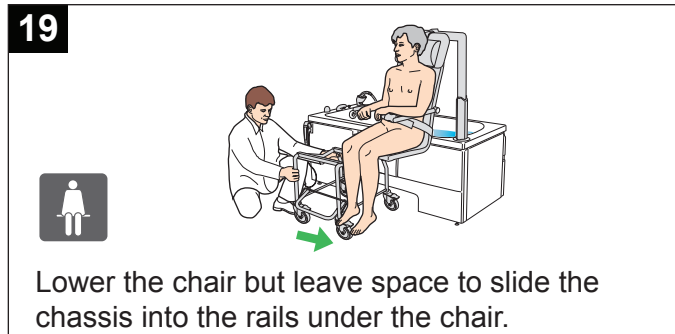
17

Lower the bathtub to the lowest position.

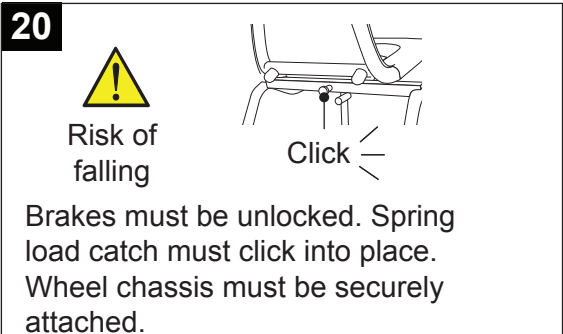
18

Risk of entrapment

Push the Lift out button, the chair rotates out of the bathtub. Help patient to move legs.

19

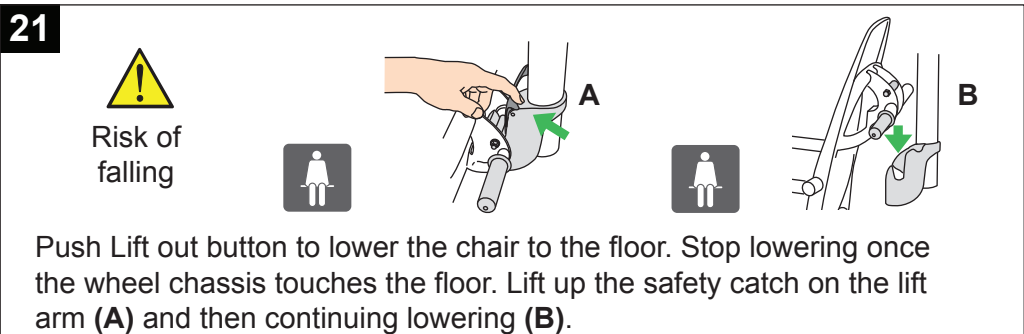
Lower the chair but leave space to slide the chassis into the rails under the chair.

20

Risk of falling

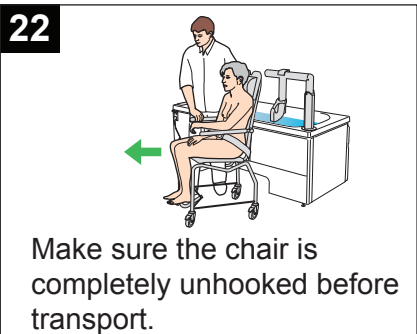
Click

Brakes must be unlocked. Spring load catch must click into place. Wheel chassis must be securely attached.

21

Risk of falling

Push Lift out button to lower the chair to the floor. Stop lowering once the wheel chassis touches the floor. Lift up the safety catch on the lift arm (A) and then continuing lowering (B).

22

Make sure the chair is completely unhooked before transport.

End bath

ArjoHuntleigh AB
Hans Michelsengatan 10
211 20 Malmö, Sweden
www.arjo.com

08.AZ6.01_1EN • 07/2020

arjo
with people in mind