

Malibu/Sovereign



Emergency stop

Press button to stop all functions.
Turn clockwise to release.

Patient lock-out

Push and hold the patient lock out button **(A)** for 3 seconds to disable all panel and remote buttons. Patient lock-out is indicated by a yellow LED **(B)**.

Push and hold for 3 seconds to re-enable all functions.

Hand control – Lift in

Push and hold the Lift in button **(A)**. The lift arm will complete a full sequence of movement; from outside the bathtub, over the bathtub edge and into the bathtub.

Hand control – Lift out

Push and hold the Lift out button **(B)**. The sequence will be performed in the reverse order.

The bathtub must be in the lowest position.

Temperature display

Shows current bathtub water temperature.

Shows the temperature of the active filling or shower.

Scalding protection

The electronic scalding protection shuts off the water if a dangerous temperature level is detected.

The LED between the displays lights up if a temperature above or equal to 45 °C (113 °F) after 10 seconds is detected.

Auto fill

The auto fill function has a preset water level.

- Activate it by pushing the auto fill button.
- Once the filling level is reached it stops by itself.
- Push the auto fill button again and filling continues for another 60 seconds.

Auto flush

The auto flush function prevents stagnating water in the shower system.

The auto flush function will start 30 hours after the last use of the bathtub and then every 24 hours if the bathtub is not used.

To avoid water spillage the shower handle should be placed in the holder after every use.

Air spa system (optional feature)

The massage effect is obtained by injecting air through air jets into the bath water. The recommended maximum treatment with the Air spa system is 15 minutes.

1

Push the Air spa system ON/OFF button, the air blower will start at minimum speed.

2

The intensity of the air stream will increase by pushing the button again.


3

Push the Air spa system ON/OFF button again to stop.


Malibu/Sovereign




Safety belt

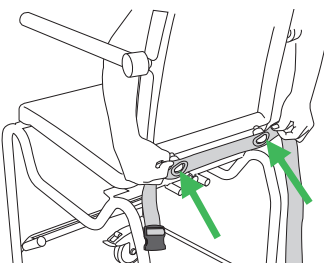

Attach safety belt.




Buckle safety belt.

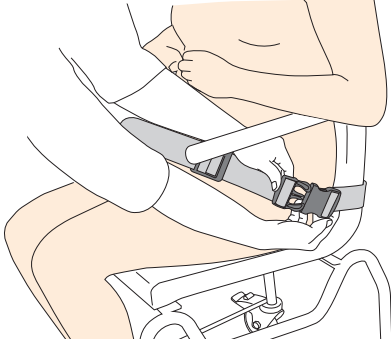


Unbuckle safety belt.


Adjust safety belt

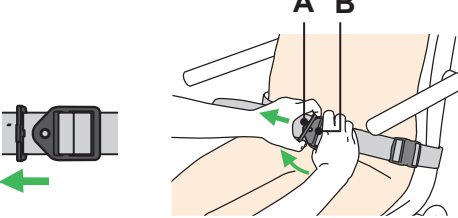


Unlock fastening clip. Pull the fastening clips (A) and (B) apart by twisting the smaller fastening clip (A) slightly upwards.

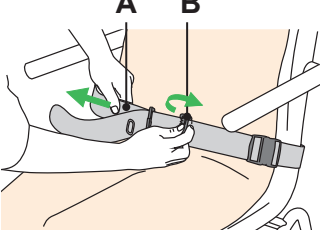

Loosen the safety belt. Pull the inner strap (A) at the side while lifting the larger fastening clip (B).

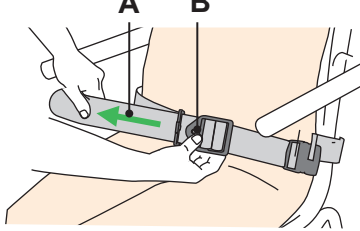


Tighten the safety belt. Pull the outer strap (A) at the side while holding the larger fastening clip (B).

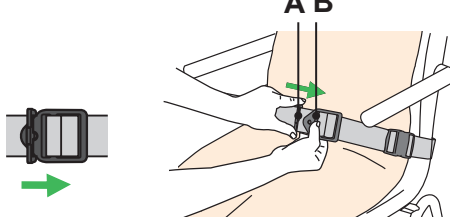


Lock fastening clip. Lock the fastening clips by inserting the smaller fastening clip (A) into the larger fastening clip (B).


Attach seat




1 With both hands start pushing the front end of the seat against the front frame.



2 Push down the bent part of the seat.




3 Push one side around the armrest and make sure it snaps into place.

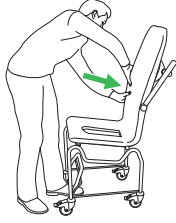


4 Repeat on the other side.


Remove seat



1 With both hands detach the seat on each side around the armrest.

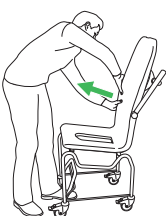


2 Repeat on the other side.




3 Pull the bottom of the seat straight up.

Attach armrest

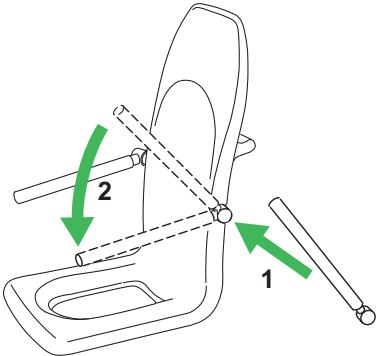

Hold the armrest with the handle pointing up wards. Guide the armrest slots onto the transfer chair (1). Fold the arm rest forward (2).

Remove armrest

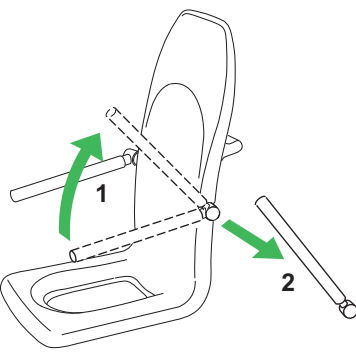

1 2