

Maxi Sky 1000



WARNING



To avoid eye and skin damage, always use protective glasses and gloves. If contact occurs rinse with plenty of water. If eyes or skin becomes irritated, contact a physician. Always read the instructions for use and material safety data sheet of the disinfectant.

Clean and disinfect before new patients.

Accessories needed:

- Protective glasses and gloves
- Disposable cloth

1. Use a cloth soaked with water to remove excessive residues from the lift and its accessories from top to bottom.
2. Use a damp cloth (warm water mixed with a mild detergent) to scrub the equipment from top to bottom to remove all visible residues.
3. Rinse off any remaining residual detergent with clean water.
4. Wipe clean with a dry cloth.

Areas listed are most likely to contain dirt and microbes, pay attention to:

- Edge of hand control
- Back and handle of hand control
- Joint between spreader bar and strap
- Frame and padding of spreader bar
- Latch

For a better rolling surface for the trolley wheels, clean the inside of the rail every four months. To do so, insert a damp cloth in the opening and slide it from one end of the rail to the other.