


Maxi Sky 440

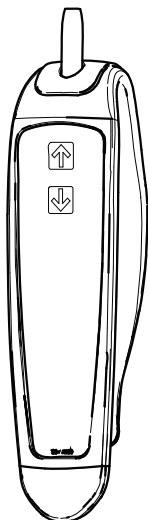

Hand control




Up button



Down button

Control panel

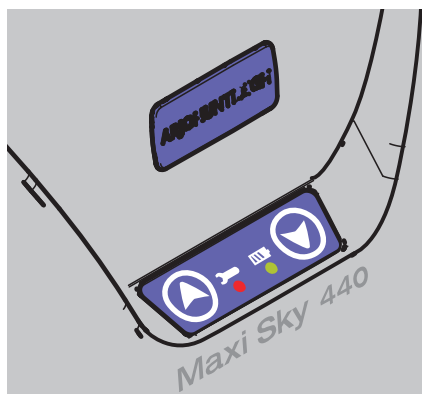

Up and down buttons

The up and down buttons are used to complete a transfer if a problem occurs with the hand control during the operation.



Maintenance required - Red light,

Flashing (no beep) - Maintenance required by your local agent.
Solid (with beep) - Maintenance is overdue. Stop using the lift and contact your local agent for service.




Battery charge level - Yellow light

Flashing - In process of charging batteries.
Solid - Charging done; batteries charged.

Emergency shut-off

1

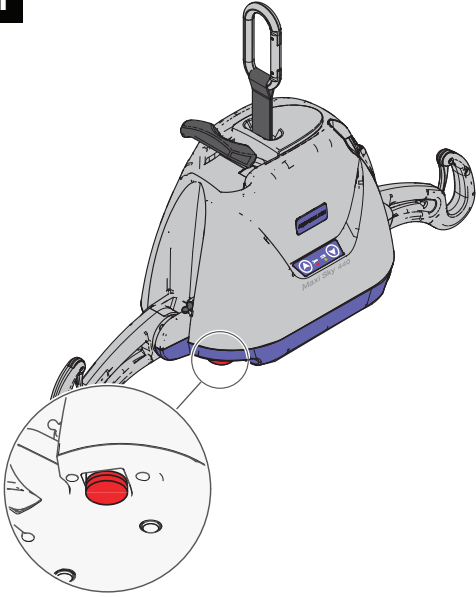

The operator can shut off the electrical power at any time by pressing the red mushroom-shaped emergency button under the lift.

The button is reset by twisting it clockwise in the direction of the arrows.

First time users should practice the shutoff manoeuvre before operating the lift with a patient.


Emergency brake

The emergency brake is an additional safety feature that automatically prevents the patient from falling in the unlikely event of a transmission or motor failure.

The emergency brake is intended for single activation and therefore can only be used once. Call your Arjo agent to arrange for the unit to be replaced. The emergency lowering device cannot be used when the emergency brake has deployed.


Emergency lowering

1

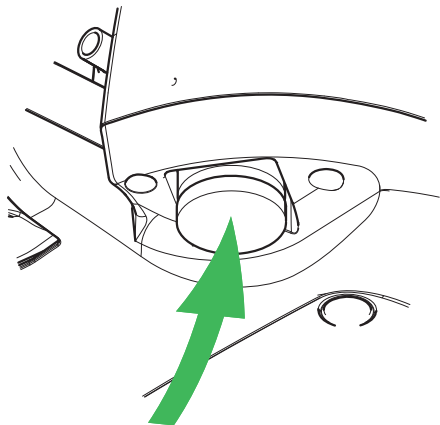


Press on the red, emergency shut-off button under the lift. Move the lift with the patient over the bed or the wheelchair.

2

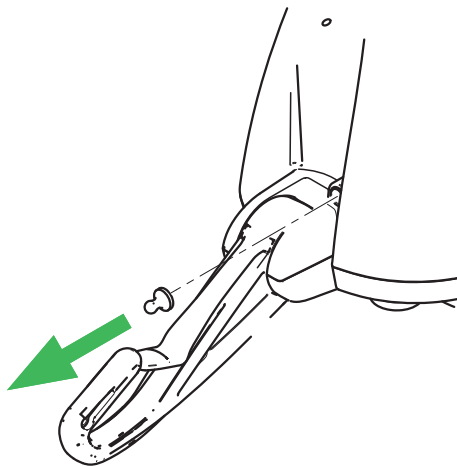

Remove the black plastic cap, located on the side of the lift.

3

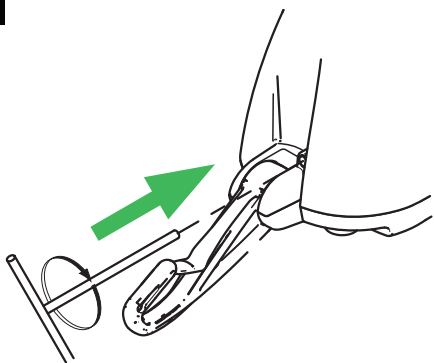

Insert the special tool into the opening. Turn it clockwise to lower the patient. The tool rotation turns the motor directly, so that for each tool turn, the patient will lower slightly.

Once the patient is lowered safely on a bed or into a wheelchair, remove the tool from the lift. Call a qualified technician to service the lift.

