


# System 2000 Rhapsody, Primo, Harmonie




## Preparations

1




Lower the bathtub to its lowest height.

2

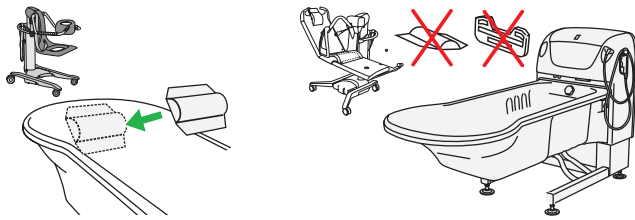

Bathing with hygiene chair: attach the head pillow.  
Bathing with bath trolley: do not attach the head pillow or foot support.


3

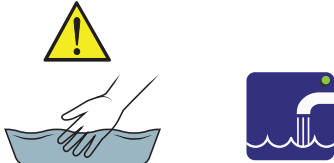

Risk of scalding

Press the Autofill button. Check the water temperature with a bare hand. Adjust temperature if needed.


4

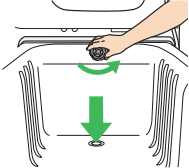

Close the drain.


## Bathe patient with lift hygiene chair

5

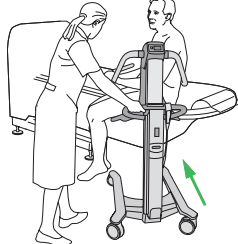

Position lift hygiene chair diagonally to the side of the bath.


6



Risk of pinching

Raise the lift hygiene chair enough to give space between the patient's genitals and the bathtub rim.


7

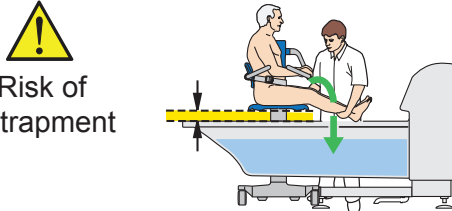

Risk of entrapment


Move the lift hygiene chair towards the bath. Lift the patient legs over the bathtub rim while rotating the lift into place.

8




Let the patient feel the water with their foot. Adjust temperature if needed.

9

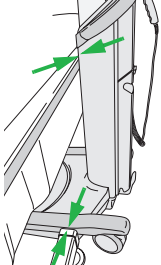

Centre chair over the bathtub until it stops against the bathtub legs and rim.


10



Risk of pinching

Adjust the lift hygiene chair to an ergonomic height for the caregiver. Raise the bathtub. Make sure that there is enough space between the patient's genitals and the bottom of the bathtub.


11

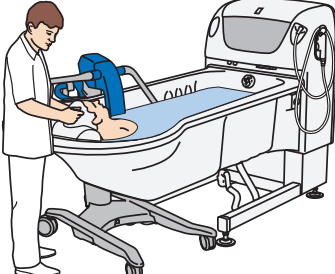

Attach foot support if needed. Bathe the patient.


## Bathe patient with lift bath trolley

12

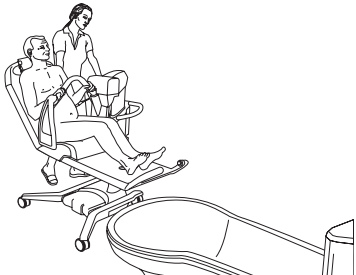


Position the lift bath trolley behind and to the side of the bath.

13

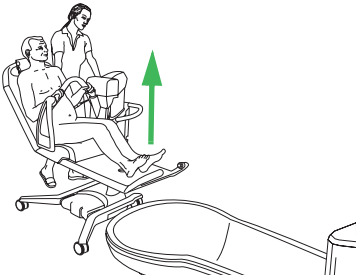


Raise the lift bath trolley stretcher enough to clear the edge of the bathtub.

14

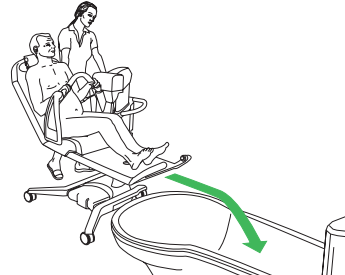


Move the lift bath trolley with patient diagonally in over the bathtub rim.

15

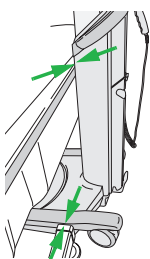


Centre the trolley until it stops against the bathtub legs and rim.

16

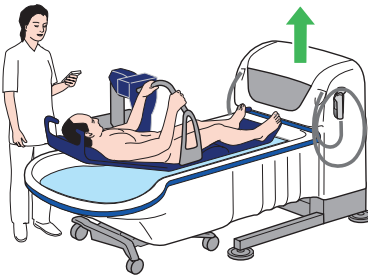

Adjust the lift bath trolley to an ergonomic height for the caregiver. Raise the bathtub until the stretcher is touching the bottom of the bathtub.

17



Attach foot support if needed. Bathe the patient.

Hydromassage (optional)

Risk of entrapment

Install the suction cover before use.

Fill up with water to cover the Hydromassage system outlet (A) and top sensor (B).

Press the Hydromassage system button. The default runtime is 15 minutes. Start with low intensity. Adjust if needed.

Adjust the direction of the water-jet.

After treatment, turn off the Hydromassage system.

Hydrosound (optional)

Never use when pregnant.

Fill up with water to cover the Hydrosound system Transducer Plate. Add Hydrosoundwater conditioner.

Risk of burns and hearing damage.

Do not touch the Transducer Plate and keep patients head above water.

Press the Hydrosound system button. The default runtime is 15 minutes.

Adjust the Hydrosound intensity.

After treatment, turn off the Hydrosound system.

Shower the patient

18

Open the drain.

19

Raise the back rest. Let the patient hold on to the hand rest.

20

Lower the bathtub to its lowest height.

21

Take the shower handle. Press the shower button. Direct the shower handle away from the patient.

22

Risk of scalding

Press the trigger on the shower handle. Check the water temperature with a bare hand. Adjust temperature if needed.

23

Let the patient feel the water by hand before directing the water stream on the patient. Adjust temperature if needed.

24

Shower the patient.

25

Unlock the shower handle trigger if lock button is activated. Press the shower button. Return the shower handle.

End the bath

26

Dry the patient.

27

Keep the patient warm using towels or a bathrobe.

28

Lower the bathtub to its lowest height.

29

Raise the lift hygiene chair enough to give space between the patient's genitals and the bathtub rim.

30

Risk of pinching

Help the patient to move the legs over the rim while rotating the lift away from the bathtub.

31

Make sure there is enough space between the bathtub side and lift hygiene chair.

32

Lower the lift hygiene chair. Dry the remaining wet areas.

33

Risk of contamination

**After each patient:** clean and disinfect the bathtub according to the instructions in the IFU or Cleaning and disinfection QRG.