

# Sara 3000

## Transfer sling

(For more information on slings, please read the sling documentation.)


1

**NOTE**  
Do not use the leg fixation strap when using the Transfer sling.



Lean the patient forward and place the sling around their lower back, 5 cm (2 inches) above their waste line.

2

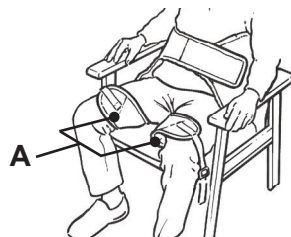

Make sure that the support strap is separated.

Take each leg section (A) of the sling and slide under each leg.

3



Press the buckles (if available) or hook and loop strap (if available) together.

Tighten the strap so that it is comfortable for the resident.


4



The sling chest support strap must always be applied and fastened when using the sling.


Move Sara 3000 into position.

5

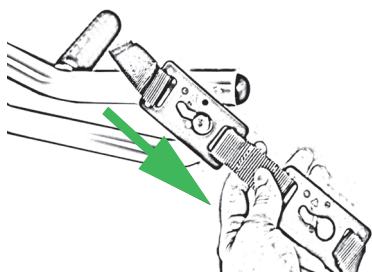


Apply the clips to the attachment points. Pull the clips as shown to secure them.

6

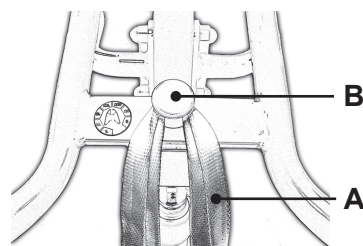

Attach each attachment loop (A) on each leg-side (left and right) on the central lug (B).

7

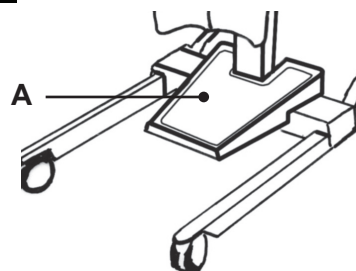


Place the residents feet on the foot support (A).

8

**NOTE**  
The chassis legs may be opened to go around a chair, by operating the appropriate control button (A).



9

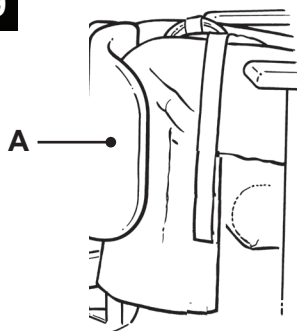

Push Sara 3000 in towards the resident to make full lower leg contact with the knee support (A).

10



Check that the sling clips are securely connected and fully in position before lifting, during the lifting cycle and in tension, as the weight is gradually taken up.

11




Place the resident's hand(s) on the support grips.

12

**NOTE**  
Do not raise the resident higher than seating position.

**NOTE**  
The resident shall not aid in being raised.

13

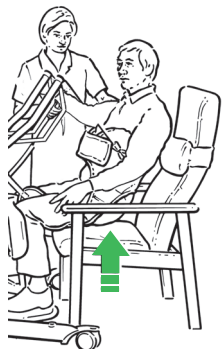

Raise the resident just clear of the chair.


14



Do not attempt to release the attachment clips, loops or the chest fixation strap while the resident is supported by the sling.

**NOTE**  
Make sure that the chassis legs are in closed position before transfer.

15

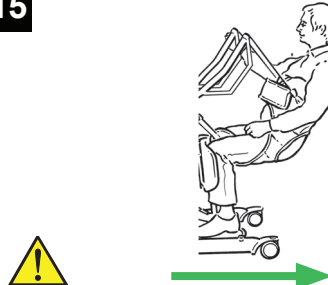

**Always move in the direction shown.**

Transfer the resident to the desired destination.

16



When lowering, make sure that the resident is fully supported by the seat, chair, toilet, etc. the resident is lowered on.

Lower resident.

When the resident is seated, open chest support strap and pull the clips of the sling upward to unlock them from the Sara 3000 resident support arms.

## Standing sling

1



Lean the patient forward and place the sling around their lower back, 5 cm (2 inches) above their waste line.

2



Make sure that the inner support strap is separated.

3



Press the buckles (if available) or hook and loop strap (if available) together.

Tighten the strap so that it is comfortable for the resident.


4



**The sling chest support strap must always be applied and fastened when using the sling.**


Move Sara 3000 into position.

5

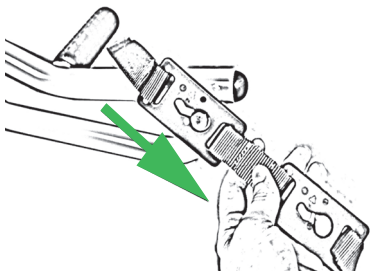

Apply the clips to the attachment points. Pull the clips as shown to secure them.

6

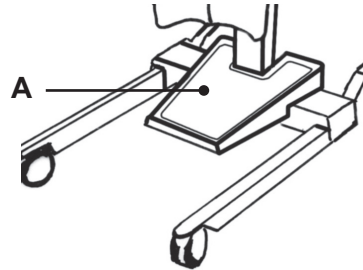


Place the residents feet on the foot support (A).

7

### NOTE

The chassis legs may be opened to go around a chair, by operating the appropriate control button.

8

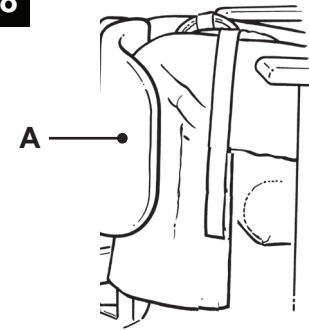

Push Sara 3000 in towards the resident to make full lower leg contact with the knee support (A).

9



Check that the sling clips are securely connected and fully in position before lifting, during the lifting cycle and in tension.

10



Place the resident's hand(s) on the support grips.

11

The resident's body posture shall go from seated to standing position.

Encourage the resident, if able, to give as much assistance as possible to raise from the chair and/or steady themselves.

12



Raise the resident.

13




Do not attempt to release the attachment clips, loops or the chest fixation strap while the resident is supported by the sling.

#### NOTE

Make sure that the chassis legs are in closed position before transfer.

14

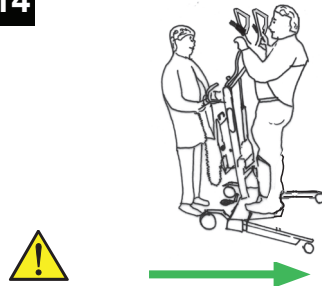

Always move in the direction shown.

Transfer the resident to the desired destination.

15



When lowering, make sure that the resident is fully supported by the seat, chair, toilet, etc. the resident is lowered on.

Lower resident.

When the resident is seated, open chest support strap and pull the clips of the sling upward to unlock them from the Sara 3000 resident support arms.

