





Bathe patient

1



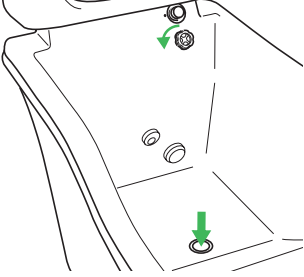

Close the drainage plug.

2




Risk of scalding






Press the Tub Fill button. Feel the water with a bare hand before patient enters the bath. Adjust temperature if needed.

3




Risk of entrapment




Position the lift hygiene chair next to the end of the bath. Raise the lift hygiene chair and move the patient over the edge of the bath.

4




Risk of entrapment




Position the lift hygiene chair in the middle of the bath and lower the lift as far as possible.

5




Risk of injury

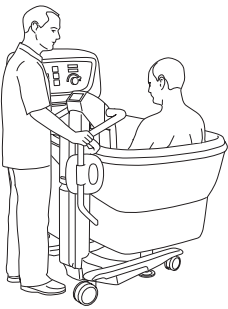


Bathe the patient. Do not leave the patient alone.

6

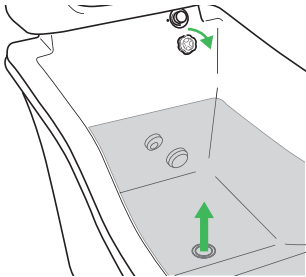

After the bath, open the drainage plug and help the patient to sit up.

7



Position the back rest against the patient's back. Raise the lift hygiene chair to a convenient working height.

Shower patient

8






Press the Patient Shower button.

9





Lift off the shower handle, direct it down into the bathtub. Press the trigger on the shower handle.

10




Risk of scalding

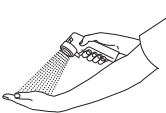



Feel the water with a bare hand. Adjust temperature if needed. Shower the patient.

11

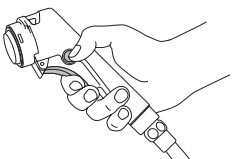


Press the shower handle trigger and push in the Lock button on the side to get continuous water flow.


12

Dry the patient.

13




Risk of falling



Raise the lift hygiene chair and pull it away from the end of the bath. Help the patient to move their legs over the edge of the bath.

14




Risk of contamination



After each patient: clean and disinfect according to the Instructions for Use or Cleaning and disinfection QRG for Century.

Hydromassage (optional)

1




Risk of entrapment

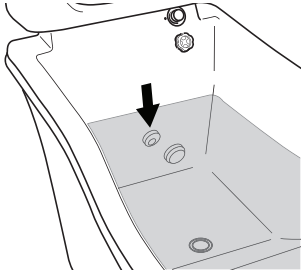

Fill up with water to cover the Hydromassage system outlet.



2

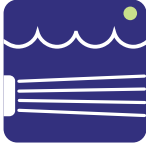

15 min



Press the Hydromassage system button. The default runtime is 15 minutes.

3

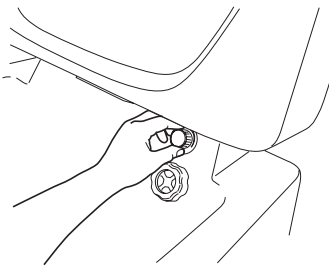

Adjust the water flow with the intensity level knob. Start with low intensity, turn clockwise to increase.

4

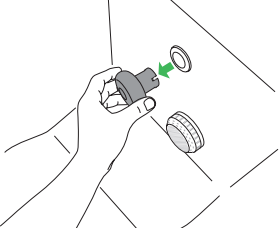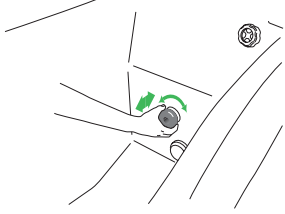


Adjust the water jet direction by turning the outlet. Pull out the adjustable nozzle to get the jet to cover a wider area.

5




After treatment is finished, turn off the Hydromassage.



Hydrosound (optional)



Risk of injury and burns. Do not use when pregnant.




1




Fill the bathtub over the Hydrosound transducer plate.

2




Press the ArjoSound water conditioner button during the filling process. Make sure the conditioner mixes into the water.

3

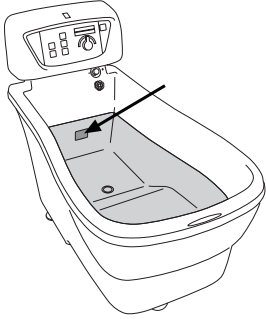



Make sure the water level is above the top of the Hydrosound system transducer plate.

4




Press the Hydrosound system button. After completion, wait 15 minutes before the Hydrosound system can be restarted.

5

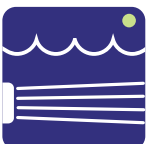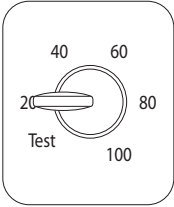


Start with a low intensity level. Turn clockwise to increase. The LED shows a steady green light and a buzzing sound is heard when any active level is selected.

6




Risk of hearing damage. Keep patient's head above water.



Bathe the patient.

7



Raise the patient out of the water and shower the patient thoroughly.