

Century



Auto fill

Autofill ON/OFF button



Press auto fill button. The filling stops automatically at preset water level.




Stop filling at any time by pressing the auto fill button.

Shower handle

Shower ON/OFF button



Press to activate shower function.

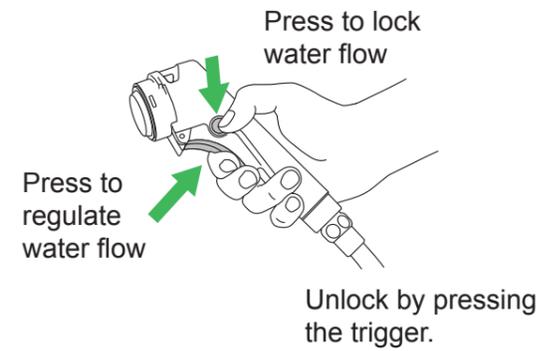

Temperature display



Shows current bathtub water temperature.

Shows the current water temperature in the outlet for bathtub filling or the shower.

Scalding protection



45 °C
(113 °F)

If scalding protection is activated, move the patient safely from the bath.

The scalding protection shuts off the water in less than 10 seconds.



The Disinfection/Autoclean button and the Patient Shower button flash.

Hydromassage (optional)

Bath Oil button



Press button to dispense bath oil.

Hydromassage ON/OFF button



Default run time
15 min

Intensity level knob



Start at low intensity.

Hydrosound (optional)

ArjoSound water conditioner button




Press button to dispense ArjoSound water conditioner.

Hydrosound ON/OFF button




Default run time
15 min

Intensity level knob

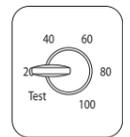

Disinfection unit


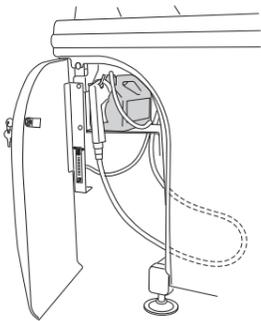


The disinfection bottles, disinfection spray handle and flow meter are stored behind a door on the left side of the panel.

Disinfection/Autoclean button



Press the Disinfection/Autoclean button to activate disinfection unit.

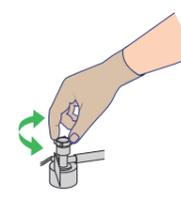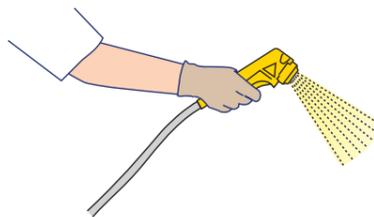

Determine disinfection mix ratio: spray disinfectant into the bathtub while checking the flowmeter, adjust the bottle cap by hand if needed.