
22000 Limb Clip Flites – 22001 Limb Loop Flites

Patient Specific Disposable
 Jetable, patient spécifique
 Paciente desechable específico
 Patientenspezifische Einweg
 Specifico paziente monouso
 Patiënt Specifieke Disposable
 患者一次性专用产品

EN Instruction for use
 FR Instructions d'utilisation
 ES Instrucciones de uso
 GB Gebrauchsanweisung
 IT Istruzioni per l'uso
 NL Instructies voor gebruik
 CN 使用说明书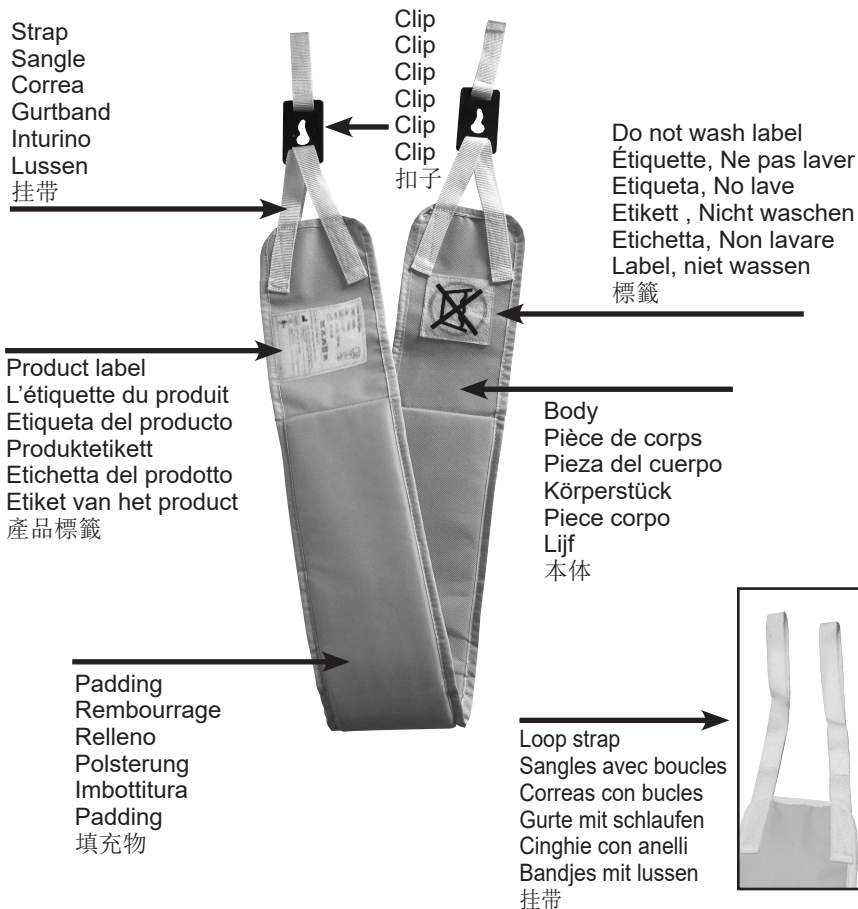

Warning! - Avertissement! - Advertencia! - Attenzione! - Warnung! - Waarschuwing - 警告! 請

Always read this instruction and the instruction for the lifter to be used with the sling
 Toujours lire cette instruction et l'instruction pour l'élévateur à être utilisés avec la fronde
 Siempre lea esta Instrucción y la Instrucción para el elevador que se utilizarán con la Honda
 Lesen Sie immer diese Anleitung und die Anleitung für die Heber mit der Schlinge verwendet werden
 Leggere sempre questa Istruzione e l'Istruzione per il sollevatore da utilizzare con la fionda
 Lees altijd deze instructie en de instructie voor de lifter te gebruiken met de draagdoek
 务必阅读本使用说明书以及使用吊兜的升降机的说明书。

Technical Specifications

Part no.	Description	Size	SWL	Material	Dimension
22000	Limb Clip Flites	Onesize	Max: 70 kg/154 lbs.	Polypropylene (PP) Polyethylene (PE)	120 x 14 CM 47 x 5.5 "
22001	Limb Loop Flites	Onesize	Max: 70 kg/154 lbs.	Polypropylene (PP) Polyethylene (PE)	120 x 14 CM 47 x 5.5 "

Definition in this IFU

Warning: Means Safety warning. Failure to understand or obey this warning may result in injury to you or to others.

Note means: Important information for the correct use of this system or equipment.

Intended Use

The Limb Flites is designed to assist the caregiver for lifting and supporting limbs of patients with reduced mobility, when positioning the limb to perform various tasks, such as; foot care, wound care, venous wrapping etc.

Safety instructions

- Always read the label and check that the sling is in good condition without signs of tear and wear such as loose seams, frays and material inconsistency.
- Always make sure not to exceed the Service Work Load (Safe Working Load) defined for the product.
- Always read the IFU for the lifting equipment to perform the lifting

Warning! Do not use the product if there is any signs of deterioration. Do Not Wash - this product is designed for single patient use only and should be disposed if in contact with liquid of any kind.

Before and after every use

1. Always perform a risk assessment before starting the specific lifting session in order to make sure the product model is correct and that the lifting procedure is fully understood.
2. Before using the Limb Flites, read this IFU and the IFU for the lifting equipment to be used
3. When soiled or stained the sling must be replaced immediately.
4. Check the Limb Flites for any fraying, loose stitching, attachments or any deterioration.
5. **Warning:** Remember to write the patient's name on the care label in order to avoid cross infection between patients.
6. After each use, check sling according to step 3 and 4 above.
7. Store in a dry area.

Attachments

The Limb Loop Flites will fit most lifts standard hangerbars with loop attachment hooks. Check the Lift manufacturers documentation for details.

The Limb Clip Flites will fit most hangerbars with clip attachment lugs. Check the Lift manufacturers documentation for details.

Applying the Limb Flites

1. Fold the Limb Flites (to avoid straps touching the skin). Place it below the patient limb in the desired position, and carefully pull the straps to each side.
2. Check patient position and limb sling position carefully before lifting.
3. Place the loop or clip straps in the hangerbar according to the IFU for the lifter
4. Carefully raise the hangerbar and perform the lifting, while keeping eye on the patient's position.

Expected service life

The service life is expected to be 56 performed lifting sessions, however this will be subject to the actual conditions during use.

The Limb Flites are disposable and designed for single patient use and should be disposed after patient hospital stay or when soiled or subjected to liquid.

Storing and Shelf life

The Limb Flites should be stored in a dry and clean environment and the shelf life is 5 years in original packaging. Store in temperatures between 0°-40° C (+32°-104°F)

Safe Working Load (SWL)

Max 70 kg / 154 lbs.

Product care & Maintenance

- Always follow the instructions on the product label and check the sling for deteriorations.
- Mark the sling with the specific patient name and keep in dry and clean environment.
- This is a patient specific disposable sling, that must be disposed after the patient has been discharged.

Service & Maintenance

The Limb Flites shall be checked by authorized personnel every 6 months according to ISO10535.

Customer support/Trouble shooting

Please refer to the IFU for the lifting equipment used.
If in doubt, please contact your dealer who supplied the product

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