



Total-Lift Bed™ VG-TLB425T

Operation Manual

(For detailed instructions, safety guidelines and warnings please refer to the owner's manual)

Please Note:

When reaching the end of the movement of each function (unless stated differently) two beeps will be heard as confirmation. All Functions should be pressed continuously.

Safety Note:

When the bed is in tilting position all other functions will not work except for (11) & (12) (12 after unlock) for raising the bed to standing position and lowering the bed from standing to flat position.

Unlocking (Nurse Only!)

Press Buttons (11) and (12) together for 3 seconds until two beeps will be heard.

Emergency Safe Mode (In case the Bed "Freezes")

Step 1 – Hold buttons 5 + 6 together for 5 seconds. The buzzer will give continues short signals indicating the Bed is in safe Mode.

To tilt down - Press 11 until the bed is flat.

To lower Backrest - Press 2 until the backrest is flat.

Any time you have had to use the Safe Mode to lower the Total Lift bed, the bed should be inspected by a qualified technician.



Blinking blue LED's on the side-rails indicates a temporary bed stoppage to help prevent a bed malfunction.

To Restart, simultaneously press buttons 3&4 continuously (10 beeps will be heard) until beeping stops. LED's should return to solid indicating bed is fully functioning.

In case Blinking continues or repeating, call a technician.

