


# Flowtron ACS900

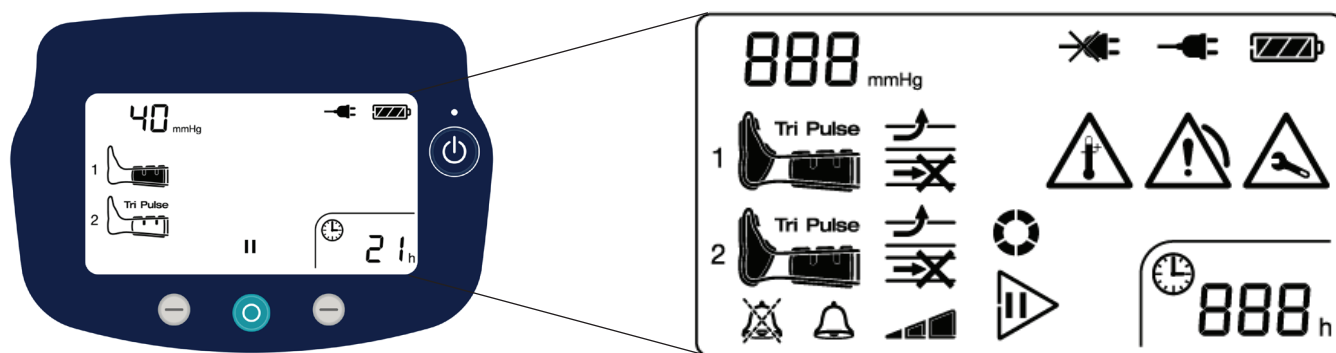

## Power button and Mains indicator

Press and hold for Standby mode, Mains indicator turns on.  
Press and hold to switch off, Mains indicator turns off.



## Middle Control button

Press to start therapy.  
Press and hold to stop therapy.



## Left Control button

Press to mute audible alarm.  
Press and hold for Alarm Volume Settings.



## Right Control button

Press and hold to enter Reset Patient Run Time mode.

888 mmHg

## Garment pressure indicator



## Garment type and inflation indicators

Connected  
not inflated

Connected  
inflated



Foot



Uniform calf  
(or calf & thigh)



Sequential (Tri Pulse)  
calf (or calf & thigh)

1  
2

## Tubeset connector numbers



## Audible alarm muted indicator



## Audible alarm status indicator

During alarm: indicates that alarm can be muted.



## Volume indicator



## Warning and alarm indicators

Alarm detected which may cause an interruption in therapy.



Low pressure or garment leak.



Kinked or blocked tubeset.



High temperature.



Hardware fault detected or periodic service required.



## Wait – initializing indicator



## Start therapy indicator

Displayed in Standby when at least one garment is connected.



## Stop therapy indicator

Displayed in Run mode.



## Mains power indicators

Pump connected to mains power supply.



Pump NOT connected to mains power supply.



## Battery indicators



Battery empty



Battery 1/4 full



Battery 1/2 full



Battery 3/4 full



Battery full



## Patient run time indicator