In-bed

In bed mobilisation

Where out of bed mobilisation is contraindicated or options are limited, there are methods for supporting rehabilitation and recovery for patients confined to bed:

Regular repositioning -

For those patients where out of bed mobilisation is contraindicated or options are limited, there are still methods for supporting early mobility and recovery.

- Daily passive movements and stretches are important to maintain joint range of motion and muscle length.
- The use of positioning therapy has long been advocated for the management of respiratory conditions in critically ill patients³⁶. Regular repositioning into alternate side lying positions or the use of Continuous Lateral Rotation Therapy (CLRT) or Kinetic Therapy has a number of benefits.



- The side lying position is useful for aiding in drainage of pulmonary secretions, with evidence to support reduced incidence of pneumonia with regular positional change³³ and appropriate turn angles. Kinetic therapy is defined as rotation of at least 40° for at least 18 hours per day and has been associated with the prevention and treatment of pulmonary complications in critically ill and mechanically ventilated patients³⁷.
- Regular repositioning is also essential to help prevent the development of pressure injuries/ulcers whenever patients are spending extended periods in bed.
- A selection of repositioning and transfer solutions to assist caregivers may be required.



Patient repositioning with Maxi Sky® 2 and Maxi Transfer Sheet

Progressive verticalisation -

Alongside side lying, progressive verticalisation in the bed into either seated or standing positions is recommended in the early stages of a patients recovery.

- The orthostatic challenge provided by this early verticalisation can help to reduce the deterioration in cardiac function or act as an early challenge for those with postural hypotension.
- This may start with a gradual move into upright sitting positions in bed, utilising the reverse Trendelenburg or cardiac chair position. Whilst not being as effective as sitting out in a chair due to the supportive surface, this seated position has a number of benefits. The change in perspective allows reorientation of the patient with their surroundings, providing a better position for communication, eating and drinking or functional activities where appropriate.



Citadel[®] Patient Care System Reverse Trendelenburg



Citadel Patient Care System Chair Position

Upright Positioning -

As appropriate the patient can be progressed to full in bed tilting to achieve standing positons while in bed.

- This provides additional benefits over the chair position by facilitating weight bearing through the lower limbs preventing or reducing the impact of immobilisation of bone demineralisation.
- Upright positioning is often used as an adjunct to therapy

 i.e. challenged sitting regularly during the day²¹ and can
 be particularly useful for those patients where regular
 sitting out is more challenging or requires high numbers
 of staff.

Benefits of upright positioning

- Provides an orthostatic challenge to prevent deterioration in CVS $^{\mbox{\tiny 35}}$
- Increase functional residual capacity ³⁸
- Better position for active exercise
- Slight increase in physiological demand ³⁸



Upright positioning with Total Lift Bed™

Cycle ergometry

Cycle ergometry is another method of supporting in bed rehabilitation, either during acute phase of illness when out of bed mobility is contraindicated (e.g. open abdomen or poorly tolerated ET tube) or as an adjunct to progressive mobility to improve strength and cardiorespiratory fitness. The safety and feasibility of using cycle ergometry for critically ill patients has been demonstrated³⁹, with evidence to suggest when utilised, patients receiving additional rehabilitation sessions of cycle ergometry were able to walk further at the point of hospital discharge in comparison to controls²⁸.

Electrical muscle stimulation

Electrical muscle stimulation may also be utilised during periods of immobility, although at present the effectiveness of this remains inconclusive⁴⁰.

References

- 1. Kress, JP. Clinical trials of early mobilization of critically ill patients. Critical Care Medicine. 2009; 37[Suppl.]:S442-S447.
- Puthacheary Z, Rawal J, Mcphail M, et al. Acute skeletal muscle wasting in critical illness. J Am Med Assoc. 2013;310:1591-600
- De Jonghe B, Bastuji-Garin S, Durand MC, et al. Respiratory Weakness Is Associated With Limb Weakness and Delayed Weaning in Critical Illness. Critical Care Medicine 2007;35:2007-2015
- Garnacho-Montero J, Amaya-Villar R, Garcia-Garmendia JL, et al. Effect of Critical Illness Polyneuropathy on the Withdrawal From Mechanical Ventilation and the Length of Stay in Septic Patients. Critical Care Med 2005;33:349-354
- 5. Griffiths RD, Hall JB. Intensive care unit-acquired weakness. Crit Care Med 2010; 38: 779–787
- Herridge MS, Tansey CM, Matte A, et al. Functional disability 5 years after acute respiratory distress syndrome. N Eng J Med. 2011;364(14):1293–304
- Griffiths J, Hatch RA, Bishop J, et al. An exploration of social and economic outcome and associated health related quality of life after critical illness in general intensive care unit survivors: a 12-month follow-up study. Crit Care. 2013;17(3):R100.
- Yende S, Austin S, Rhodes A, Finfer S, Opal S, Thompson T, et al. Long-term quality of life among survivors of severe sepsis. Crit Care Med. 2016;44(8):1461-7.
- Cooper AB, Thornley KS, Young GB, et al. Sleep in critically ill patients requiring mechanical ventilation Chest. 2000; 117(3):809-18
- Bannon L, McGaughley J, Clarke, M. Impact of nonpharmacological interventions on prevention and treatment of delirium in critically ill patients: protocol for a systematic review of quantitative and qualitative research, Systematic Reviews.2016; 5 (75)
- Pandharipande PP, Girard TD, Jackson JC, et al. Long-Term Cognitive Impairment after Critical Illness. N Engl J Med 2013; 369:1306-1316
- 12. Truong AD, Fan E, Brower RG, Needham DM. Bench to bedside review: mobilizing patients in the intensive care unit-from pathophysiology to clinical trials. Crit Care. 2009;13:1–8
- 13. Convertino V, et al. Cardiovascular responses to exercise in middle-aged men after 10 days of bed rest. Circulation.

1982;65(1):134-40.

- 14. Convertino VA, Bloomfield SA, Greenleaf JE. An overview of the issues: physiological effects of bed rest and restricted physical activity. Med Sci Sports Exerc. 1997;29:187-90
- Winkleman C. Bed rest in health and critical illness—a body systems approach. AACN Adv Crit Care. 2009;20(3):254-66.
- Topp R, et al. The effect of bed rest and potential of prehabilitation on patients in the intensive care unit. AACN Clin Issues. 2002;13(2):14.
- Bloomfield S. Changes in musculoskeletal structure and function with prolonged bed rest. Med Sci Sports Exerc. 1997;29(2):197-206
- Zerwekh, JE. The effects of twelve weeks of bed rest on bone histology, biochemical markers of bone turnover, and calcium homeostasis in eleven normal subjects. Journal of Bone and Mineral Research. 1998; 13(10), p.1594-601.
- Rawal J, et al. A pilot study of change in fracture risk in patients with acute respiratory distress syndrome. Crit Care. 2015;19(1):165.
- 20. Parker A, Sricharoenchai T, Needham DM. Early rehabilitation in the intensive care unit: preventing physical and mental health impairments. Curr Phys Med Rehabil Rep. 2013;1(4):307-314.
- 21. Morris PE, Berry MJ, Files DC, Thompson JC, Hauser J, Flores L, et al. Standardized rehabilitation and hospital length of stay among patients with acute respiratory failure. J Am Med Assoc. 2016;315(24):2694–9.
- 22. Mcwilliams D, Weblin J, Atkins G, et al. Enhancing rehabilitation of mechanically ventilated patients in the intensive care unit: a quality improvement project. J Crit Care. 2015;30(1):13-8.
- National Institute for Health and Care Excellence [NICE].
 (2009) Rehabilitation after critical illness. London: NICE (Nice guideline no 83)
- 24. Baron R, Binder A, Biniek R, et al. Evidence and consensus based guideline for the management of delirium, analgesia, and sedation in intensive care medicine. (DAS- Revised Guideline 2015) - short version. Ger Med Sci. 2015 Nov 12;13:Doc19.
- 25. Devlin JW, YOanna S, Gelinas C, et al. Clinical Practice Guidelines for the Prevention and Management of Pain, Agitation/Sedation, Delirium, Immobility, and Sleep

Disruption in Adult Patients in the ICU. Critical care Medicine. 2018; 46(9)

- 26. Needham D, Korupolu R, Zanni JM, et al. Early physical medicine and rehabilitation for patients with acute respiratory failure: A quality improvement project. Arch Phys Med Rehabil 2010; 91:536-542
- 27. Chiang LL, Wang LY, Wu CP, et al: Effects of physical training on functional status in patients with prolonged mechanical ventilation. Phys Ther 2006; 86:1271-1281
- 28. Burtin C, Clerckx B, Robbeets C, et al: Early exercise in critically ill patients enhances short-term functional recovery. Crit Care Med.2 2009; 37:2499-2505
 41. Collings N, Cusack R. A repeated measures, randomized cross-over trial, comparing the acute exercise response
- Schweickert W, Pohlman MC, Pohlman AS, Nigos C, Pawlik AJ, Esbrook CL, et al.Early physical and occupational therapy in mechanically ventilated, critically illpatients: a randomised controlled trial. Lancet 2009;373:1874–82.13
- 30. Schaller SJ, Anstey M, Blobner M, et al. Early, goal-directed mobilisation in the surgical intensive care unit: a randomised controlled trial. Lancet 2016;388:1377-88
- 31. Berney S, Harold M, Webb S, et al. Intensive care unit mobility practices in Australia and New Zealand: a point prevalence study. Crit Care Resusc. 2013;15(4):260–5.
- 32. Nydahl P, Ruhl AP, Bartoszek G et al. Early mobilization of mechanically ventilated patients: a 1-day point-prevalence study in Germany. Crit Care Med. 2014;42(5):1178–86.
- Parry, S.M., Knight, L.D., Connolly, B. et al. Factors influencing physical activity and rehabilitation in survivors of critical illness: a systematic review of quantitative and qualitative studies. Intensive Care Med. 2017; 43: 53
 Parry S and Puthucheary Z. The impact of extended bed rest on the musculoskeletal system in the critical care environment. Extreme Physiology and Medicine. 2015; 4(16)
- 34. Hodgson CL, Stiller K, Needham DM, et al. Expert consensus and recommendations on safety criteria for active mobilisation of mechanically ventilated critically ill adults. Critical Care 2014; 18:658: 1-9
- McWilliams, D.J., Westlake, E.V., Griffiths, R.D. Weakness on the Intensive Care Unit - Current Therapies. British Journal of Intensive care. 2011 Summer edition. pp. 23-27
- Vollman KM (2004). The right position at the right time: mobility makes a difference. Intensive Crit Care Nurs, 20:179-182
- Ahrens T, Kollef M, Stewart J, Shannon W (2004). Effect of Kinetic Therapy on Pulmonary Complications. American Journal of Critical Care, 13(5): 376-382

 Chang AT, Boots RJ, Brown MG, et al. Ventilatory changes following head-up tilt and standing in healthy subjects. Eur J Appl Physiol. 2005;95(5-6):409-17

39. Kimawi I, Lamberjack B, Nelliot A, et al. Safety and Feasibility of a Protocolized Approach to In-Bed Cycling Exercise in the Intensive Care Unit: Quality Improvement Project. Physical Therapy. 2017; 97(6):593-602

40. Parry S, Berney S, Granger CL, et al. Electrical muscle stimulation in the intensive care setting:a systematic review. Crit Care Med. 2013;41(10):2406-18.

Collings N, Cusack R. A repeated measures, randomized cross-over trial, comparing the acute exercise response between passive and active sitting in critically ill patients. BMC Anesthesiol 2015;15:1

- 42. Gillick BT, Marshall WJ, Rheault W, Stoecker J. Mobility criteria for upright sitting with patients in the neuro/ trauma intensive care unit: an analysis of length of stay and functional outcomes. Neurohospitalist 2011; 1: 172–177.
- 43. Zafiropoulos B, Alison JA, McCarren B. Physiological responses to the early mobilisation of the intubated, ventilated abdominal surgery patient. Aust J Physiother 2004; 50: 95-100.
- Weissman C, Kemper M. Stressing the critically ill patient: the cardiopulmonary and metabolic responses to an acute increase in oxygen consumption. J Crit Care 1993; 8: 100–108.
- 46. McWilliams, D., Atkins, G., Hodson, J., Snelson, C. The Sara Combilizer as an early mobilisation aid for critically ill patients: A prospective before and after study. Australian Critical Care. 2017; 30(4): 189-195

March 2019. Only Arjo designed parts, which are designed specifically for the purpose, should be used on the equipment and products supplied by Arjo. As our policy is one of continuous development we reserve the right to modify designs and specifications without prior notice. [®] and TM are trademarks belonging to the Arjo group of companies. [©] Arjo, 2019

The Total Lift Bed is manufactured by VitalGo Systems, Inc. and are distributed under contract by Arjo. Packaging, labelling and documentation supplied with product will reflect this.

At Arjo, we are committed to improving the everyday lives of people affected by reduced mobility and age-related health challenges. With products and solutions that ensure ergonomic patient handling, personal hygiene, disinfection, diagnostics, and the effective prevention of pressure ulcers and venous thromboembolism, we help professionals across care environments to continually raise the standard of safe and dignified care. Everything we do, we do with people in mind. Arjo AB • Hans Michelsensgatan 10 • 211 20 Malmö • Sweden • +46 10 335 4500

www.arjo.com

