

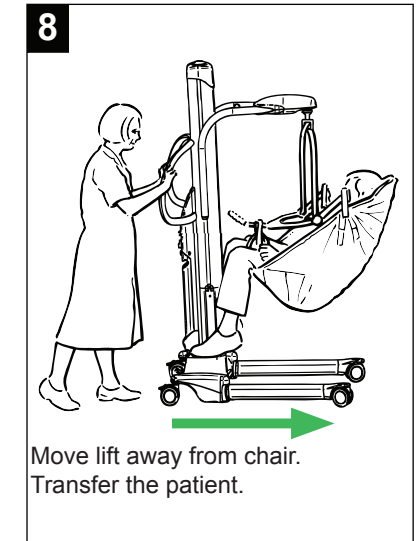
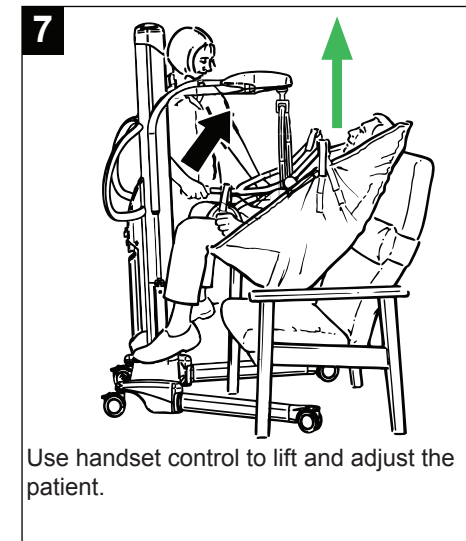
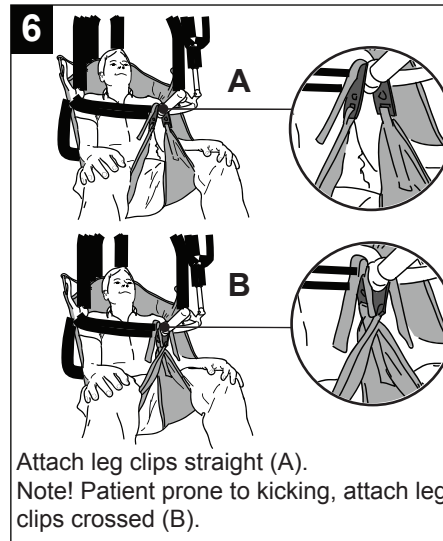
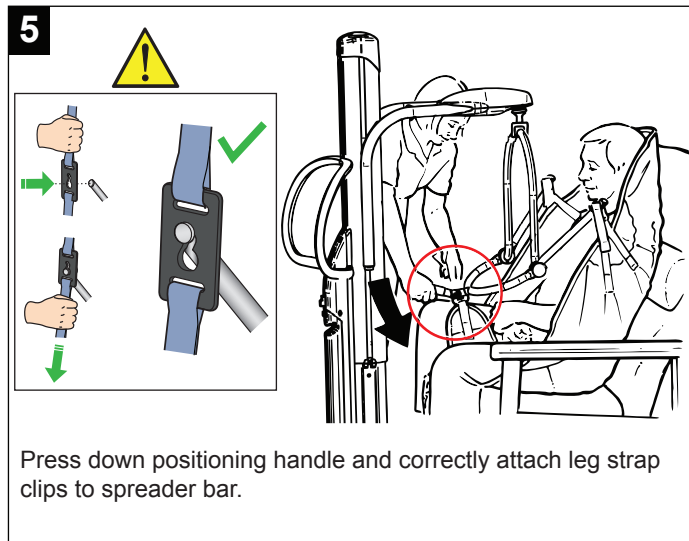
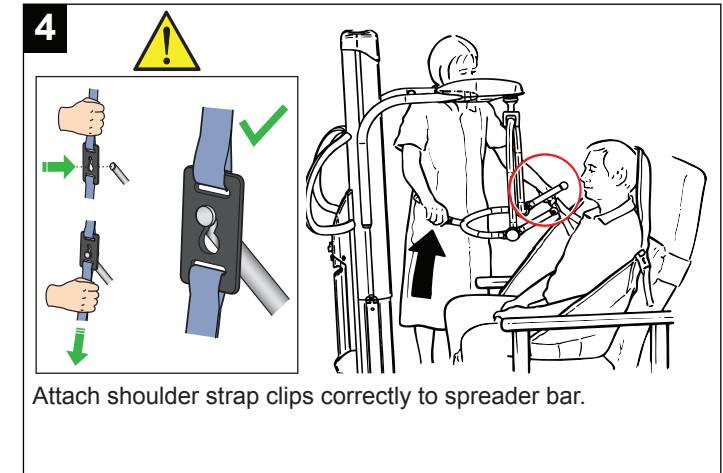
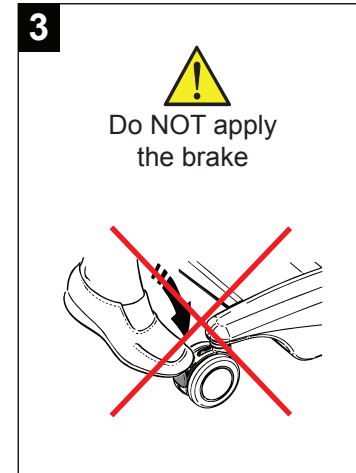
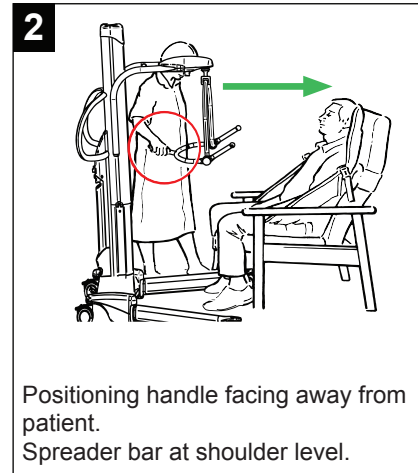


Maxi Move


From chair

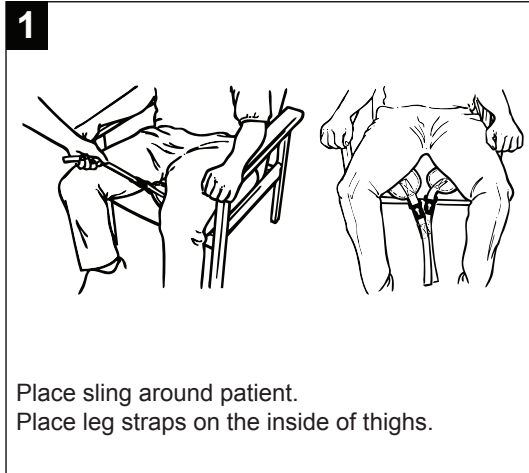

From bed

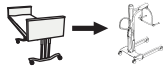


1

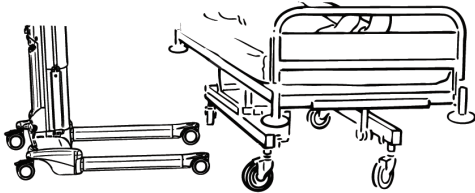


Check clearance for chassi legs under bed.

2

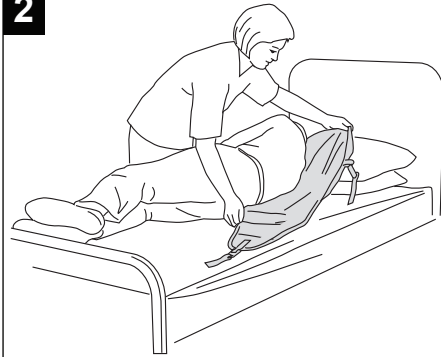


Place sling under patient.

3

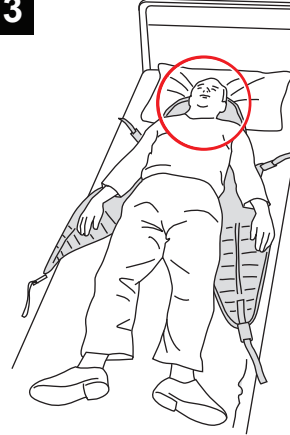


Sling head support area under patient's neck.

4

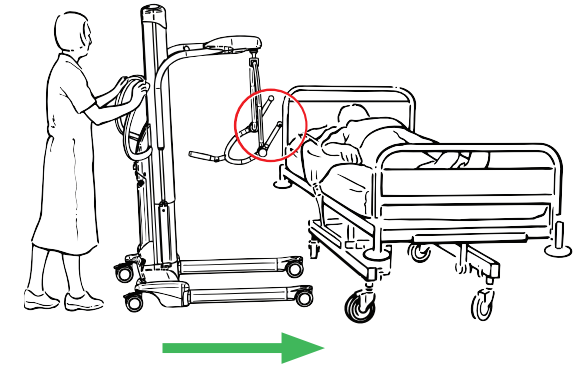


Approach bed with open side of spreader bar towards patient's head. Adjust chassis leg width if necessary.

5

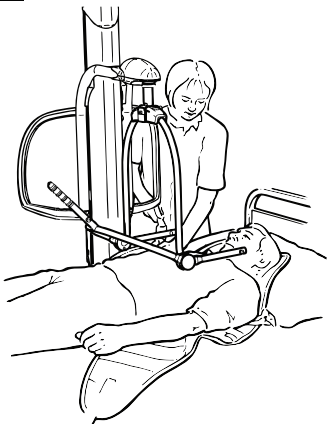


Position spreader bar above and centred over patient.

6

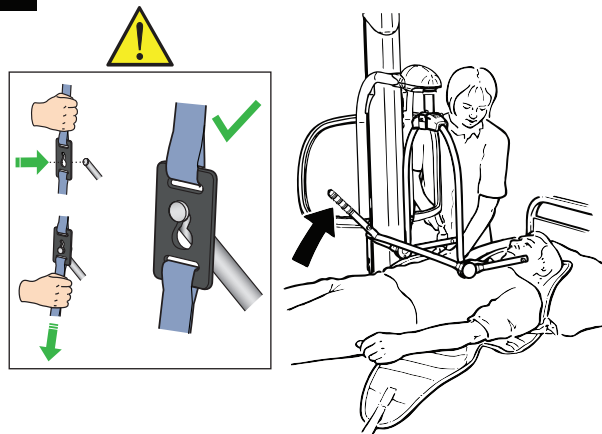


Use positioning handle to tilt spreader bar. Attach shoulder strap clips correctly to spreader bar.

7

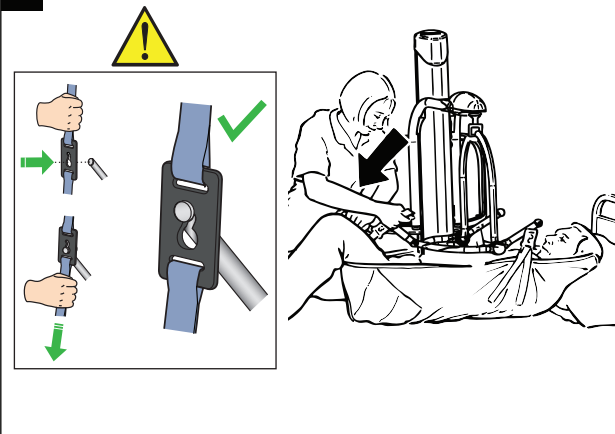

Press down positioning handle and correctly attach leg strap clips under thighs to spreader bar.

8

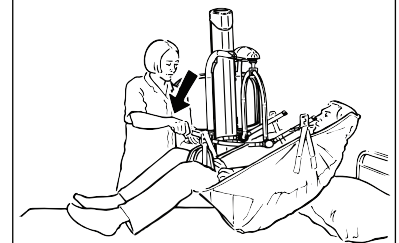

Lift and transfer the patient