
Seba

Instructions for Use

ARJOHUNTLEIGH
GETINGE GROUP



Foreword	3
Intended Use	4
Installation requirements	4
Expected service life	4
Safety Instructions	5
Use on a powered mattress or high foam mattress	5
Preparations	6
Hook Installation	7
Parts Designation	8
Product Description/Functions	9
Raise Patient in Bed	10
Lower Patient in Bed	12
Cleaning and Disinfection Instructions	14
Care and Preventive Maintenance	16
Troubleshooting	17
Approvals and List of Standards	17
Technical Specifications	18
Dimensions	20
Labels on the Seba aid	21
Spare Part	21

WARNING

To avoid injury, always read this *Instructions for Use* and accompanied documents before using the product.



Mandatory to read the *Instructions for Use*.

Design Policy and Copyright

© and ™ are trademarks belonging to the ArjoHuntleigh group of companies. © ArjoHuntleigh 2015.

As our policy is one of continuous improvement, we reserve the right to modify designs without prior notice. The content of this publication may not be copied either whole or in part without the consent of ArjoHuntleigh.

Foreword

Thank you for purchasing ArjoHuntleigh equipment.

Please contact us if you have any questions about the operation or maintenance of your ArjoHuntleigh equipment.

Please read and fully understand these *Instructions for Use (IFU)* before using your Seba™ aid.

Information in this *IFU* is necessary to perform the proper operation and maintenance of the equipment. It will help to protect your product and make sure that the equipment performs to your satisfaction. The information in this *IFU* is important for your safety and must be read and understood to help prevent possible injury. Unauthorized modifications on any ArjoHuntleigh equipment can affect safety. ArjoHuntleigh will not be held responsible for any accidents, incidents or lack of performance that occur as a result of any unauthorized modification to its products.

Support

Routine maintenance must be performed on your *Seba* aid every week to maintain safety and reliability of the equipment. See section “*Care and Preventive Maintenance*” on page 16.

If you require further information, please contact ArjoHuntleigh for comprehensive support and maintenance to maximize the long-term safety, reliability and value of the product.

Contact your local ArjoHuntleigh representative for spare parts. The telephone numbers appear on the last page of this *IFU*.

Definitions in this IFU

WARNING

Means: Safety warning. Failure to understand and obey this warning may result in injury to you or to others.

CAUTION

Means: Failure to follow these instructions may cause damage to all or parts of the system or equipment.

NOTE

Means: This is important information for the correct use of this system or equipment.



Means: The name and address of the manufacturer.

EN

Intended Use

This equipment must be used in accordance with these safety instructions. Anyone using this equipment must also have read and understood the instructions in these *Instructions for Use (IFU)*.

EN

If there is anything you do not completely understand, ask your ArjoHuntleigh representative.

The ArjoHuntleigh *Seba* seated-edge-of-bed aid is intended to be used by appropriately trained caregivers in acute and long term care environments to position adult and adolescent patients from a supine to seated-edge-of-bed position and back again.

The product should only be used for the purpose specified in these Instructions for Use. Any other use is prohibited.

Patient/resident assessment

ArjoHuntleigh recommends that facilities establish regular assessment routines. Prior to the transfer caregivers should assess each patient/resident according to the following criteria:

- Size
- Mobility Level
- Medical conditions
- Ability to understand and respond to instructions

The *Seba* aid is intended for patients/residents:

- Who are able to sit on the edge of the bed
- Who have the ability to lie on their side
- Whose weight does not exceed 185 kg

If these criteria are not met, an alternative equipment/device shall be used.

Assessment of number of caregivers

Assess how many caregivers are needed based on the patient/resident assessment.

At least two caregivers are needed if the *Seba* aid is used on:

- an inflated powered mattress
- a mattress in foam material, higher than 18 cm (7 1/8")

Installation requirements

No installation is required.

Expected service life

The expected service life of the *Seba* aid is 5 years, subject to preventive maintenance being carried out in accordance with the instructions in these Instructions for Use.

Safety Instructions

WARNING

To avoid injury, make sure that the patient is not left unattended at any time.

WARNING

To avoid injury, always assess the patient prior to use.

WARNING

To avoid injury, always follow the allowed combinations listed in this IFU. No other combinations are allowed.

EN

Use on a powered mattress or high foam mattress

WARNING

To avoid injury, pay extra attention to the patient and Seba aid when using it on an inflated powered mattress or high foam mattress. Failure to follow these instructions for use, can cause the patient to accidentally fall off the bed.

For best integration with the *Seba* aid, it is recommended to keep the total height of the mattress layers down.

Powered mattress setting


The *Seba* aid can be used on a deflated or inflated powered mattress. Always use the mattress setting mode recommended for ambulatory transfers for the specific mattress used. See respective mattress IFU. It is recommended not to use maximum pressures settings (like AutoFirm and InstaFlate) since this will increase the total height of the mattress.

Usage

When using the *Seba* aid on an inflated powered mattress or high foam mattress (higher than 18 cm (7 1/8")):

- Make sure there are at least two caregivers positioned on the transfer side.

- Never let go of the patient when the patient is positioned on the *Seba* aid.
- Be prepared to stop any unwanted sliding of the patient during placement, raising and lowering.
- Be careful not to position the patient too close to the edge of the mattress.
- To raise the patient in bed: One caregiver raises the patient, while the other caregiver supports the patient's legs.

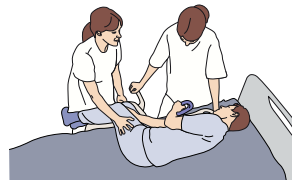

- To lower the patient in bed: One caregiver lowers the patient's upper body, while the other caregiver helps lifting the patient's legs. When lowered, make sure the patient is NOT too close to the edge of the bed.



Preparations

Actions Before First Use (7 steps)

1. Visually check the package for damage. If the product looks damaged due to freight, make a complaint to the transport agency.
2. The packaging should be recycled according to local regulations.
3. Check that all parts of the product are supplied. Compare with section “*Parts Designation*” on page 8. If any part is missing or damaged - do NOT use the product.
4. Read this *IFU*.
5. Install the hook according to section “*Hook Installation*” on page 7.
6. Disinfect the product according to section “*Cleaning and Disinfection Instructions*” on page 14.
7. Choose a designated area, where the *IFU* and Quick Guide should be kept and is easily accessible at all times.

Actions Before Every Use (3 steps)

1. Visually inspect the *Seba* aid. If any part is missing or damaged - do NOT use the product.
2. Assess the patient. See section “*Intended Use*” on page 4.
3. Assess if one or two caregivers are needed. See section “*Assessment of number of caregivers*” on page 4.

In between Use (2 steps)

1. To store the *Seba* aid:
 - use the hook to hang the *Seba* aid by the positioning grip “*Parts Designation*” on page 8, or
 - stand the *Seba* aid on the floor.
2. Disinfect the product according to section “*Cleaning and Disinfection Instructions*” on page 14.


General information

The Quick Guide is equipped with a Quick Response (QR) code. Scan it to get to the *Seba* aid web page.



Hook Installation

Fig. 1

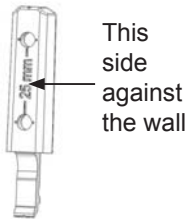

The hook kit includes:

- 2 brackets
- 2 hooks


Not included:

- Flat head screws (4 mm)

Before installing the hook, make sure the wall properties and the surface are suitable to hold the weight of the *Seba* aid.

EN

Fig. 2

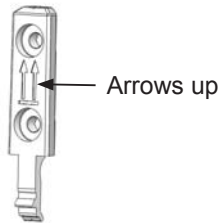

Attach the hook kit (4 steps)

1. Place the bracket at a height of 140 cm (4 feet 7 inches) or higher, measured from the lower screw hole. (See Fig. 1)
2. Mount the bracket so that the arrows are visible and pointing up. (See Fig. 2)
3. Slide the hook over the bracket until it snaps into place. (See Fig. 3)
4. The hook is now installed


Fig. 3



Detach the hook from the bracket (2 steps)


1. Push in the tip of the bracket towards the wall. (See Fig. 4)
2. While pushing, pull the hook up.

Fig. 4

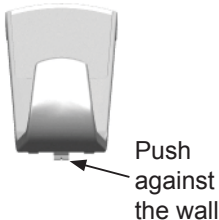

Parts Designation

EN

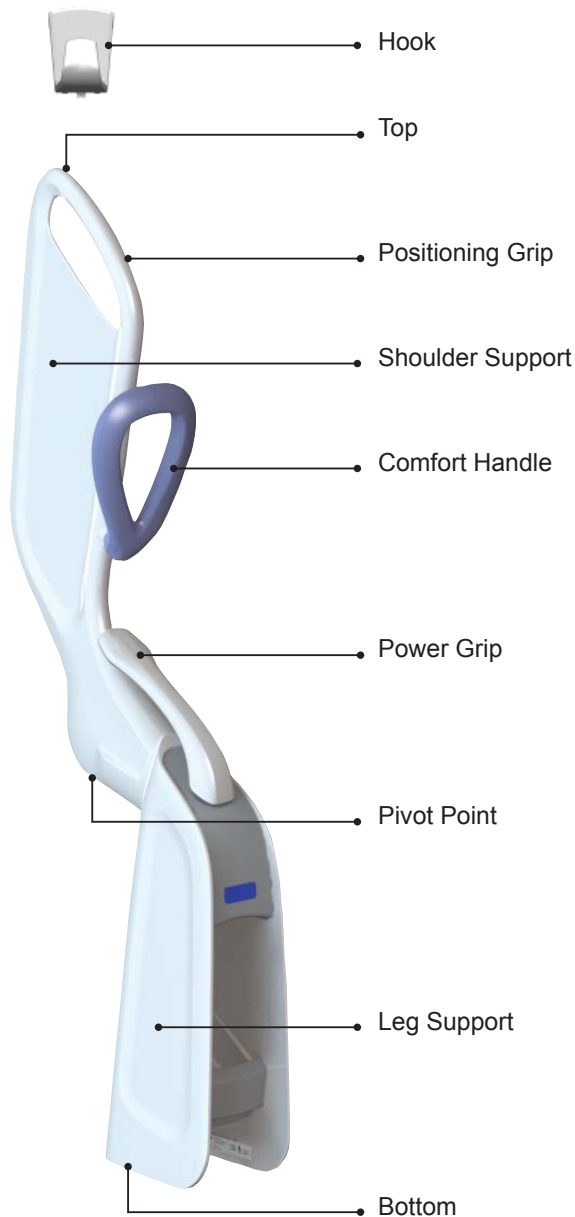

Product Description/Functions

Fig. 5



Fig. 6



Fig. 7



Fig. 8



Directions

- The *Seba* aid can be used on both the left and right side of the bed.

Positioning grip

- The positioning grip can help maximise patient stability. (See Fig. 5)
- Reduces the primary load for the caregiver during raising and lowering in bed.

Shoulder support

- Thin profile for easy positioning.
- Supports the patient.

Comfort handle

This comfort handle is for patient use when using the *Seba* aid and can: (See Fig. 6)

- give the patient reassurance.
- promote patient participation.
- rotate for use on both sides of the bed.

Power Grip

Used to raise the patient from bed for optimal force assistance. (See Fig. 7)

- Provides ergonomic efficiency.
- Rotates for use on both sides of the bed.

Pivot Point

- Eases and controls the lifting and manoeuvring.
- It aligns with the patient's hip for optimised efficiency.

Leg support

- Keeps lower body free of stress and strain.

How to carry the *Seba* aid

Carry the *Seba* aid by holding it under the pivot point. (See Fig. 8)

EN

Raise Patient in Bed

Fig. 9


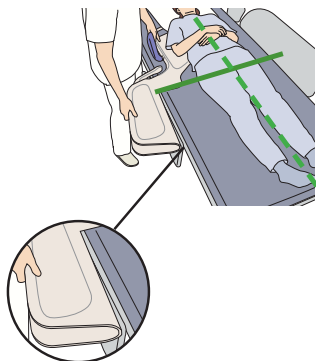


Fig. 10



Fig. 11

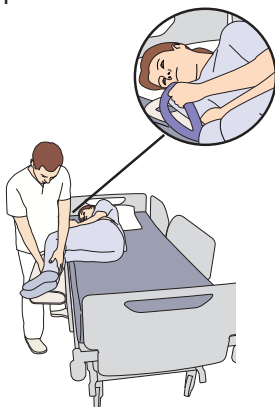

(20 steps)

Preparations

1. If any other equipment will be used, place it nearby.
2. Apply the brakes on the bed.
3. Adjust the bed to an ergonomic height for the caregiver.
4. Fold down the side rail (if any) on the transfer side.
5. Make sure the patient is positioned centrally in bed or towards the opposite side of the transfer side. ((See Fig. 9)

If not, use local technique to reposition.

Placement

6. Place the *Seba* aid underneath the patient's shoulder.
7. Align the pivot point with the patient's hip joint. (See Fig. 9)
8. Make sure the leg support is placed just outside and parallel to the bed. ((See Fig. 9)

Roll patient onto the *Seba* aid

9. Help the patient to bend their legs.
10. Roll the patient onto the *Seba* aid and ask the patient to reach for the comfort handle. (See Fig. 10)
11. Help the patient to place their legs on the leg support and make sure they are outside of the bed. (See Fig. 11)
12. Ask the patient to hold on to the comfort handle. (See Fig. 11)

Continue with the steps on the next page

Fig. 12

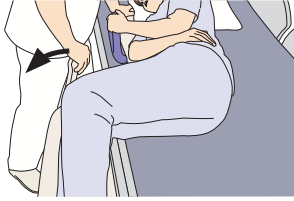


Fig. 13

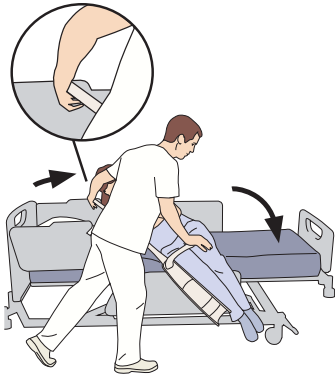


Fig. 14

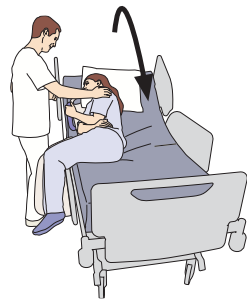


Fig. 15



Raise the patient using the power grip

13. Fold the power grip up and out to a vertical position. (See Fig. 12)
14. Take a firm hold of the positioning grip with one hand and the power grip with the other.
15. Turn towards the foot end of the bed.
16. Raise the *Seba* aid and the patient by pushing the power grip and pulling the positioning grip at the same time. (See Fig. 13)

CAUTION

Do NOT apply force to the comfort handle during raising. The comfort handle is intended for the patient.

Alternative methods

Bed back raise (See Fig. 14)

- Raise the head end of the bed while keeping one hand on the patient's shoulder.

Hip Swing (See Fig. 15)

- Grab the positioning grip with one hand and put the other hand on the patient's hip. Swing the patient up to a sitting position.

Stabilizing

17. Support the patient, now sitting on the edge of the bed.
18. Adjust the bed height for the patient.
19. Carefully, remove the *Seba* aid.

Disinfect

20. Disinfect the *Seba* aid according to section "*Cleaning and Disinfection Instructions*" on page 14.

Lower Patient in Bed

Fig. 16



EN

Fig. 17

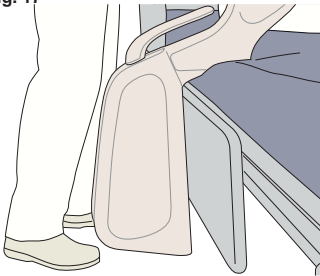


Fig. 18



(16 steps)

Preparations

1. Place the *Seba* aid close by.
2. Apply the brakes on the bed.
3. Adjust the bed height for the patient.
4. Fold down the side rail (if any) on the transfer side.
5. Sit the patient down on the bed, make sure that at least half of the patient's thighs are on the bed.

Placement

6. Place the *Seba* aid next to the patient towards the head end of the bed.
7. Make sure the *Seba* aid is (**See Fig. 16**)
 - on top of the mattress,
 - close to the patient and
 - with the back of the leg support close to the bed side. (**See Fig. 17**)
8. Ask the patient to hold on to the comfort handle.
9. Help the patient get positioned. Make sure to:
 - lean the patient forward towards the comfort handle and
 - place the shoulder on the shoulder support. (**See Fig. 18**)
10. Adjust the bed to an ergonomic height for the caregiver.

Continue with the steps on the next page

Fig. 19



Fig. 20


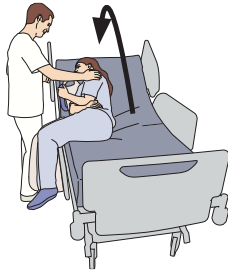


Fig. 21



Fig. 22

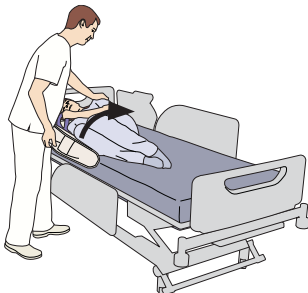

Lower the patient using the shoulder grip

11. Lower the patient gently on the bed as follows:

- take a firm hold of the positioning grip with one hand,
- place the other hand on the patient's shoulder and
- pay attention to the patient's legs during the movement so that they don't get stuck or scratched against the bed. (See Fig. 19)

EN

Alternative method

Bed back lowering (See Fig. 20)

- Lower the back rest of the bed from its most upright position while keeping one hand on the patient's shoulder.

Roll the patient off the Seba aid

12. Lift the patient's legs onto the bed.

13. Make sure to align the bent legs in a centre line along the patient's upper body. (See Fig. 21)

14. Use the *Seba* aid to roll the patient onto their back. (See Fig. 22)

15. Remove the *Seba* aid.

Disinfect

16. Disinfect the *Seba* aid according to section "*Cleaning and Disinfection Instructions*" on page 14.

Cleaning and Disinfection Instructions

If you have any questions regarding disinfecting the equipment, or wish to order disinfectant, contact ArjoHuntleigh Customer Service.

The following procedure should be carried out after every use.

EN

WARNING



To avoid eye and skin damage, always use protective glasses and protective gloves.

If contact occurs, rinse with plenty of water. If eyes or skin become irritated, consult a physician. Always read the Instructions for Use and Material Safety Data Sheet of the disinfectant.

WARNING

To prevent cross-contamination, always follow the disinfection instructions in this Instruction for Use.

Allowed Disinfectants

- 70% alcohol,
- Chlorine solution of 1000 ppm
- Arjo Clean
- Arjo Disinfectant Cleanser IV
- Arjo All Purpose Disinfectant
- Arjo CenKleen IV
- Arjo Plus V

Accessories for Disinfecting

- Protective gloves
- Protective glasses
- Spray bottle/shower handle with water
- Spray bottle/shower handle with disinfectant
- Cloths
- Soft bristle brush

Fig. 23



Fig. 24



Fig. 25



Fig. 26



Fig. 27



Always follow these steps for Cleaning and Disinfection

The Seba aid (12 steps)

1. Put on protective gloves and glasses.
2. Make sure the disinfectant is mixed according to the instructions on the disinfectant bottle label.
3. Remove visible residue.
4. Rinse the *Seba aid* with lukewarm water 20-35 °C (68-95 °F).
5. Spray detergent on the *Seba aid*. (See Fig. 23)
6. Use a soft bristle brush soaked in disinfectant. Scrub the:
 - support areas in contact with patient,
 - positioning and power grip and
 - comfort handle.
7. Soak a cloth in disinfectant and wipe all areas of the *Seba aid*. (See Fig. 25)
8. Allow a disinfection time according to the instructions on the disinfectant bottle label.
9. Rinse the *Seba aid* thoroughly with lukewarm water 20- 35 °C (68-95 °F) until all the disinfectant has been removed. (See Fig. 26)
10. Wet a new cloth with water and wipe off all traces of disinfectant left in hidden areas.
11. If the disinfectant is tough to remove, spray water on the affected area and wipe it off with a disposable towel. Repeat until all of the disinfectant has been removed.
12. Let all parts dry.

The hook (4 steps)

1. Wipe down the hook with a wet cloth. (See Fig. 27)
2. Soak a cloth in disinfectant and wipe down the hook.
3. Wet a new cloth with water and wipe off all traces of disinfectant left on the hook.
4. Let it dry.

Care and Preventive Maintenance

SCHEDULE Caregiver Obligations Action/Check	After Every USE	Every WEEK
Clean/Disinfect	X	
Visually check all parts		X
Visually check mechanical attachments		X

EN

Caregiver Obligations

Caregiver obligations shall be carried out by personnel with sufficient *Seba* aid knowledge following the instructions in this *IFU*.

After every use

The *Seba* aid has to be cleaned and disinfected immediately after every use, see section “*Cleaning and Disinfection Instructions*” on page 14.

Every week

Visually check all parts.

- Check where contact is made by either patient or the caregiver.
- Make sure no cracks or sharp edges have developed that could cause the patient or caregiver injury or that has become unhygienic causing cross-contamination.
- If any part is damaged, contact qualified personnel.


Check mechanical attachments.

- Check the attachment of the comfort handle and the power grip.

Troubleshooting

If something is wrong with the *Seba* aid, contact qualified personnel.

Approvals and List of Standards

CERTIFICATE	STANDARDS
EN12182:2012	Technical aids for disabled persons. General requirements and test methods.
Bio Compatibility	Passed the tests as defined in EN ISO 10993-1:2009.
	Declaration of Conformity CE marking in accordance with the Medical Devices Directive 93/42/EEC. The equipment has been tested and fulfils all essential requirements.

EN

Technical Specifications

General	
Maximum patient weight	185 kg (408 lbs)
Weight of the <i>Seba</i> aid	4.5 kg (9.9 lbs)
Material	PP (polypropylene), ABS plastic (Acrylonitrile Butadiene Styrene), Aluminium Zinc aluminium alloy

EN

Operating conditions	
Ambient temperature	+5°C to 40°C (41°F to 104°F)
Relative humidity	Not applicable
Atmospheric pressure	Not applicable

Transport and Storage	
Ambient temperature	-20°C to 70°C (-4°F to 158°F)
Relative humidity	Not applicable
Atmospheric pressure	Not applicable

Recycling	
Package	Corrugated card board and PE foam recyclable.
The product	Metal and plastic parts shall be separated and recycled according to local regulations.


Allowed Combinations	
Hook	<ul style="list-style-type: none"> Hook kit: 8664999
Medical Beds	<ul style="list-style-type: none"> Enterprise 5000 Model: 5000X, 5001X, E5000X Enterprise 8000 Model: 8000X, 8001X, 8002X, 8003X, E8000X Enterprise 9000 Model: 9000X, E9000X Minuet 2 Model: 160, 161 Citadel Bed Frame System

Allowed Combinations	
Medical Beds	<p>The <i>Seba</i> aid is compatible with a broad range of beds in acute care and long term care settings.</p> <p>The <i>Seba</i> aid is compatible with beds that fulfil the following requests:</p> <ul style="list-style-type: none"> • with a mattress platform • where the mattress platform is able to be raised minimum 45 cm (17 3/4") above the floor • where the width of bed and mattress are minimum 80 cm (31 1/2") • where the combined width from the outer surface of the bed to the edge of the mattress does not exceed 8,5 cm (3 3/8") • where elevations (if any) for keeping the mattress in place does not hinder or obstruct the <i>Seba</i> aid movement.
Mattresses	<ul style="list-style-type: none"> • Evolve • Pentaflex Range: 150SLXXX/XX, 250SLXXX/XXX, 120PMXXX/XXX, 252PMXXX/XXX, POXXX/XXX • Nimbus 3 • Nimbus 4 • Nimbus Professional • Autologic 110, 175, 200: PXA001XXX, PXB005XXX, PXB001XXX • Alpha Response • Alpha Active 3 • Alpha Active 4 • Alpha Trancell Deluxe: TRA04, ATD002DAR, ATD001DAR • First Step Select: First Step Classic, First Step AirRail • First Step All-in-one • TheraKair Visio • Citadel Patient Therapy System <p>The <i>Seba</i> aid is compatible with mattresses:</p> <ul style="list-style-type: none"> • made of foam material with a maximum height of 21 cm (8 1/4") • that are powered replacement mattress systems, with a maximum inflated height of 22 cm (8 5/8") • that are powered overlay mattress systems, with a maximum inflated overlay height of 12.5 cm (4 7/8"), combined with a foam base with a maximum height of 18 cm (7 1/8")
No other combinations are allowed	


Dimensions

Measurements in mm, and inches (")


Front View

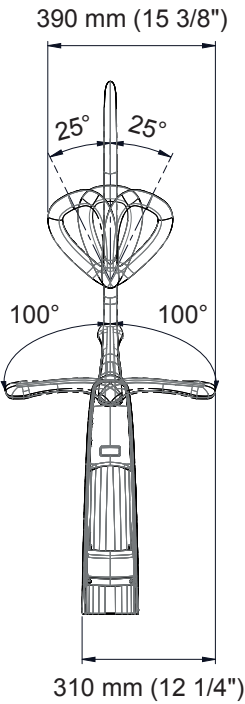

Side View

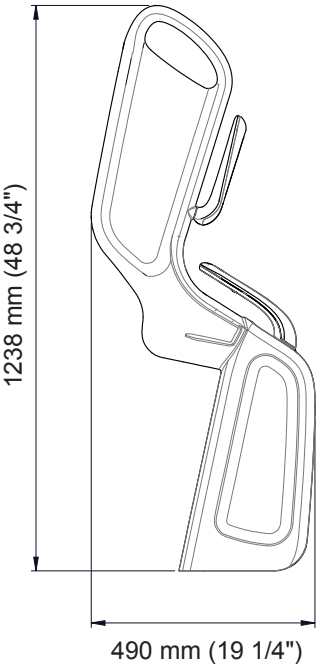



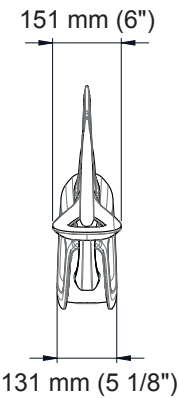




Top View



Labels on the Seba aid

On the label you will find the following information:

Symbol explanation	
SEBXX-XXXX	Model number
	Reference number
	Serial number
	CE marking in accordance with The Medical Devices Directive 93/42/EEC.
	Manufacturer name and address

	Date of manufacture
	Read the <i>IFU</i> before use
	Maximum patient weight

EN

Spare Part



Intentionally left blank

AUSTRALIA

ArjoHuntleigh Pty Ltd
78, Forsyth Street
O'Connor
AU-6163 Western Australia
Tel: +61 89337 4111
Free: +1 800 072 040
Fax: +61 89337 9077

BELGIQUE / BELGIË

ArjoHuntleigh NV/SA
Evenbroekveld 16
BE-9420 ERPE-MERE
Tél/Tel: +32 (0) 53 60 73 80
Fax: +32 (0) 53 60 73 81
E-mail: info@arjohuntleigh.be

BRASIL

Maquet do Brasil
Equipamentos Médicos Ltda
Rua Tenente Alberto Spicciati, 200
Barra Funda, 01140-130
SÃO PAULO, SP - BRASIL
Fone: +55 (11) 2608-7400
Fax: +55 (11) 2608-7410

CANADA

ArjoHuntleigh
90 Matheson Boulevard West
Suite 300
CA-MISSISSAUGA, ON, L5R 3R3
Tel/Tél: +1 905 238 7880
Free: +1 800 665 4831 Institutional
Free: +1 800 868 0441 Home Care
Fax: +1 905 238 7881
E-mail: info.canada@arjohuntleigh.com

ČESKÁ REPUBLIKA

ArjoHuntleigh s.r.o.
Hlinky 118
CZ-603 00 BRNO
Tel: +420 549 254 252
Fax: +420 541 213 550

DANMARK

ArjoHuntleigh A/S
Vassingerødvej 52
DK-3540 LYNGE
Tel: +45 49 13 84 86
Fax: +45 49 13 84 87
E-mail:
dk_kundeservice@arjohuntleigh.com

DEUTSCHLAND

ArjoHuntleigh GmbH
Peter-Sander-Strasse 10
DE-55252 MAINZ-KASTEL
Tel: +49 (0) 6134 186 0
Fax: +49 (0) 6134 186 160
E-mail: info-de@arjohuntleigh.com

ESPAÑA

ArjoHuntleigh Ibérica S.L.
Ctra. de Rubí, 88 1ª planta - A1
08173 Sant Cugat del Vallés
ES- BARCELONA 08173
Tel: +34 93 583 11 20
Fax: +34 93 583 11 22
E-mail: info.es@arjohuntleigh.com

FRANCE

ArjoHuntleigh SAS
2 Avenue Alcide de Gasperi
CS 70133
FR-59436 RONCQ CEDEX
Tél: +33 (0) 3 20 28 13 13
Fax: +33 (0) 3 20 28 13 14
E-mail: info.france@arjohuntleigh.com

HONG KONG

ArjoHuntleigh (Hong Kong) Ltd
1510-17, 15/F, Tower 2
Kowloon Commerce Centre
51 Kwai Cheong Road
Kwai Chung
HONG KONG
Tel: +852 2207 6363
Fax: +852 2207 6368

INTERNATIONAL

ArjoHuntleigh International Ltd
ArjoHuntleigh House
Houghton Hall Park
Houghton Regis
UK-DUNSTABLE LU5 5XF
Tel: +44 (0) 1582 745 800
Fax: +44 (0) 1582 745 866
E-mail:
international@arjohuntleigh.com

ITALIA

ArjoHuntleigh S.p.A.
Via Giacomo Peroni 400-402
IT-00131 ROMA
Tel: +39 (0) 6 87426211
Fax: +39 (0) 6 87426222
E-mail: Italy.promo@arjohuntleigh.com

NEDERLAND

ArjoHuntleigh Nederland BV
Biezenwei 21
4004 MB TIEL
Postbus 6116
4000 HC TIEL
Tel: +31 (0) 344 64 08 00
Fax: +31 (0) 344 64 08 85
E-mail: info.nl@arjohuntleigh.com

NEW ZEALAND

ArjoHuntleigh Ltd
41 Vestey Drive
Mount Wellington
NZ-AUCKLAND 1060
Tel: +64 (0) 9 573 5344
Free Call: 0800 000 151
Fax: +64 (0) 9 573 5384
E-mail: nz.info@ArjoHuntleigh.com

NORGE

ArjoHuntleigh Norway AS
Olaf Helsetts vei 5
N-0694 OSLO
Tel: +47 22 08 00 50
Faks: +47 22 08 00 51
E-mail:
no.kundeservice@arjohuntleigh.com

ÖSTERREICH

ArjoHuntleigh GmbH
Dörrstrasse 85
AT-6020 INNSBRUCK
Tel: +43 (0) 512 204 160 0
Fax: +43 (0) 512 204 160 75
POLSKA
ArjoHuntleigh Polska Sp. z o.o.
ul. Ks Piotra Wawrzyniaka 2
PL-62-052 KOMORNIKI (Poznan)
Tel: +48 61 662 15 50
Fax: +48 61 662 15 90
E-mail: arjo@arjohuntleigh.com

PORTUGAL

ArjoHuntleigh em Portugal
MAQUET Portugal, Lda. (Distribuidor
Exclusivo)
Rua Poeta Bocage n.º 2 - 2G
PT-1600-233 Lisboa
Tel: +351 214 189 815
Fax: +351 214 177 413
E-mail: Portugal@arjohuntleigh.com

SUISSE / SCHWEIZ

ArjoHuntleigh AG

Fabrikstrasse 8
Postfach
CH-4614 HÄGENDORF
Tél/Tel: +41 (0) 61 337 97 77
Fax: +41 (0) 61 311 97 42

SUOMI

Oy Vestek AB
Martinkuja 4
FI-02270 ESPOO
Puh: +358 9 8870 120
E-mail: info@vestek.fi

SVERIGE

ARJO Scandinavia AB
Hans Michelsensgatan 10
SE-211 20 MALMÖ
Tel: +46 (0) 10 494 7760
Fax: +46 (0) 10 494 7761
E-mail: kundservice@arjohuntleigh.com

UNITED KINGDOM

ArjoHuntleigh UK
ArjoHuntleigh House
Houghton Hall Park
Houghton Regis
UK-DUNSTABLE LU5 5XF
Tel: +44 (0) 1582 745 700
Fax: +44 (0) 1582 745 745
E-mail: sales.admin@arjohuntleigh.com

USA

ArjoHuntleigh Inc.
2349 W Lake Street Suite 250
US-Addison, IL 60101
Tel: +1 630 307 2756
Free: +1 800 323 1245 Institutional
Free: +1 800 868 0441 Home Care
Fax: +1 630 307 6195
E-mail: us.info@arjohuntleigh.com

REV 15: 02/2015

GETINGE GROUP

Getinge Group is a leading global provider of products and systems that contribute to quality enhancement and cost efficiency within healthcare and life sciences. We operate under the three brands of **ArjoHuntleigh**, **Getinge** and **Maquet**. **Getinge** provides solutions for infection control within healthcare and contamination prevention within life sciences. **Maquet** specializes in solutions, therapies and products for surgical interventions, interventional cardiology and intensive care.

ARJOHUNTLEIGH

GETINGE GROUP

ArjoHuntleigh focuses on patient handling and hygiene, disinfection, DVT prevention, medical beds, therapeutic surfaces and diagnostics.



ArjoHuntleigh AB
Hans Michelsensgatan 10
211 20 Malmö, Sweden
www.arjohuntleigh.com



04SE01.INT1