Overbed Table

Instructions for Use

ARJOHUNTLEIGH
GETINGE GROUP

Brugsvejledning • Bedienungsanleitung • Instrucciones de uso
Notice d'utilisation • Istruzioni per l'uso • Gebruiksaanwijzing
Bruksanvisning • Instrukcja obsługi • Instruções de Uso
Instruktioner för användning

EN

DA

DE

ES

FR

IT

NL

NO

PL

PT-BR

SV

Design Policy and Copyright

® and ™ are trademarks belonging to the ArjoHuntleigh group of companies unless otherwise stated.

© ArjoHuntleigh 2016

As our policy is one of continuous improvement, we reserve the right to modify designs without prior notice. The content of this publication may not be copied either wholly or in part without the consent of ArjoHuntleigh.

EN

Contents

1.	Introduction.....	2
2.	Components Diagram	3
3.	Assembly.....	4
4.	Operation.....	11
5.	Cleaning and Maintenance	13
6.	Warranty and Service	14
7.	Specifications / Configurations.....	15
8.	Symbols Used	17

Warnings, Cautions and Notes

WARNINGS given in this manual identify possible hazards in procedures or conditions, which if not correctly followed, could result in death, injury or other serious adverse reactions.

Cautions given in this manual identify possible hazards in procedures or conditions, which if not correctly followed, could result in equipment damage or failure.

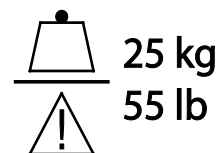
★ This symbol indicates optional items or features.

WARNING

The carer should ensure the Overbed Table is correctly positioned and adjusted to allow the patient to use it safely and comfortably.

Do not lean or sit on the Overbed Table.

Do not exceed the Safe Working Load of 25kg evenly distributed.

Caution

Take care when moving the Overbed Table over rough or uneven surfaces. Lift the Overbed Table over large ridges and wide gaps.

1. Introduction

The **ArjoHuntleigh Overbed Table** is an overbed / overchair table that provides patients with a firm, flat surface suitable for eating, writing, reading, etc. Single-column construction and adjustable height makes it suitable for use with most types of beds and chairs.

- Model OBT0 has a fixed one-piece top surface.
- Models OBT1 and OBT2 have a two-piece top surface with adjustable tilt.
- Features:
 - Wipe-clean top
 - Base design allows the Overbed Table to be used over either a bed or a chair
 - Gas spring assisted height adjustment
 - Castors for ease of mobility
 - Range of colours and finishes
 - Two-piece top surface with adjustable tilt ★
 - Bookrest (on tilt version only) ★
 - Optional perimeter rails (on tilt version only) ★

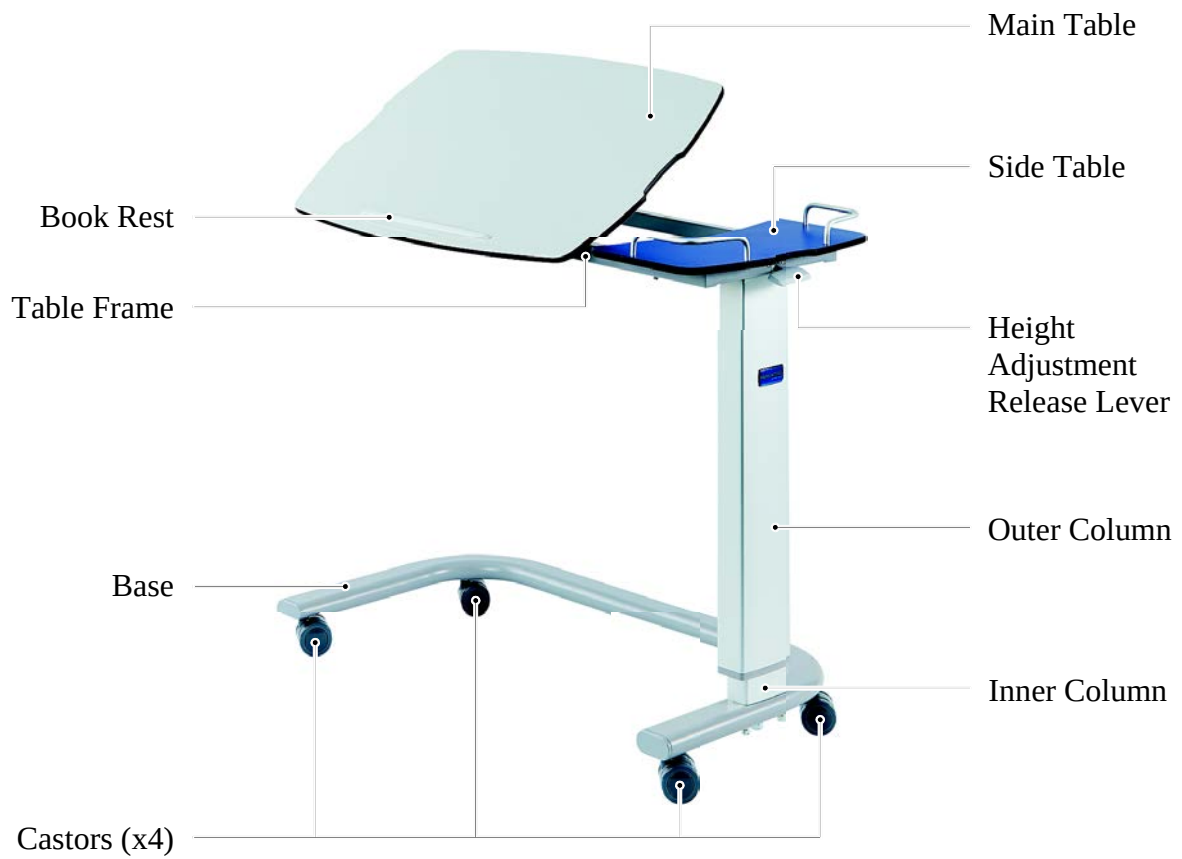
Model OBT0 with fixed top

Model OBT2 with tilting top
Shown with optional rails

2. Components Diagram


3. Assembly

3.1 Assembly Instructions


Tools needed:

- 17mm Spanner
- 14mm Spanner
- Phillips Screwdriver
- 4mm Hex Key
- 5mm Hex Key
- 6mm Hex Key

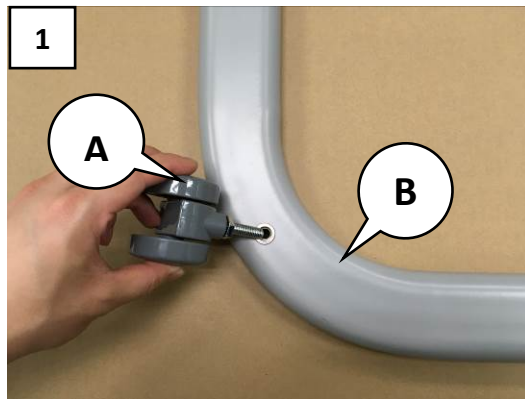
1. Use 17mm spanner to fully tighten the four castors (A) into the threaded holes in the base (B).

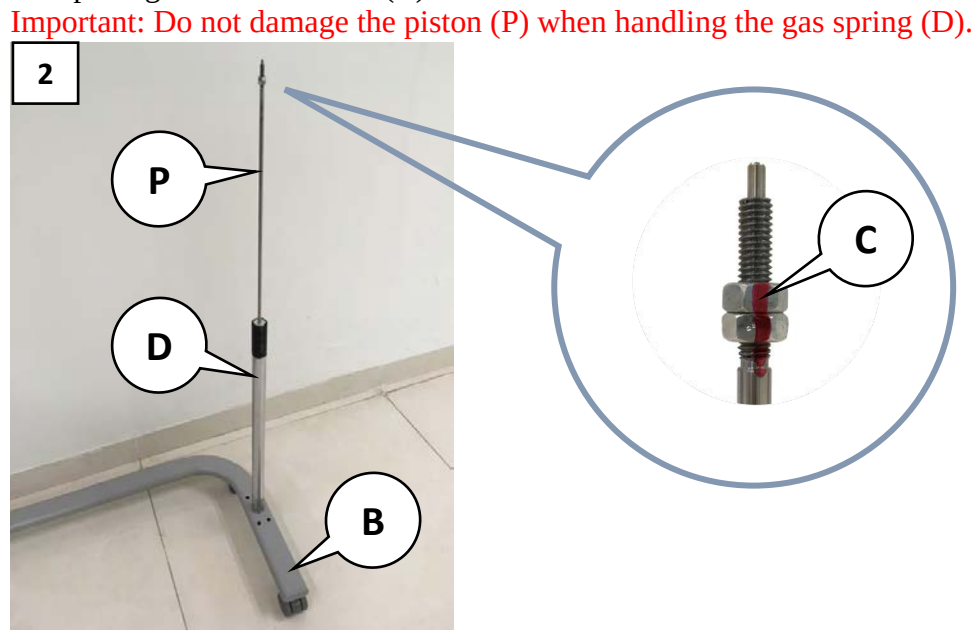
2. Ensure that the red-lined nuts (C) on the gas spring (D) are aligned as shown in the inset, and then position the threaded end of the gas spring (D) through the opening hole on the base (B).

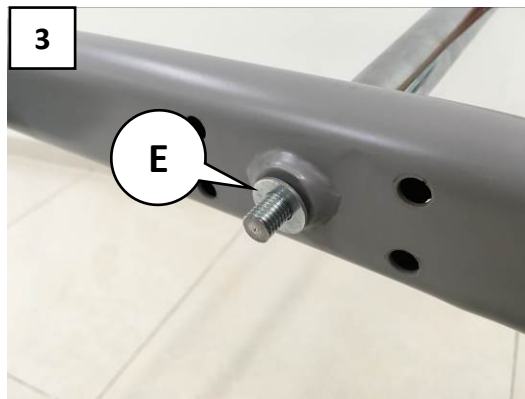3. At the bottom of the gas spring, place flat washer (E) over the thread.

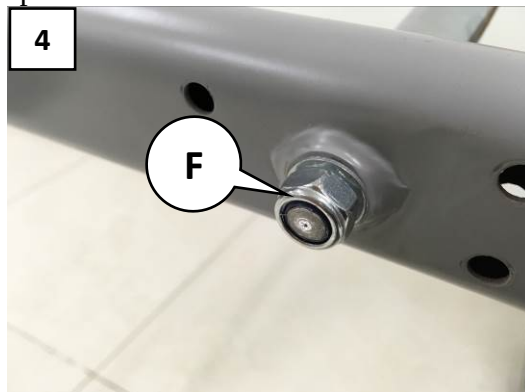
4. Place locknut (F) over the thread and flat washer and tighten with 14mm spanner.

5. Slide the column assembly (G) over the gas spring.

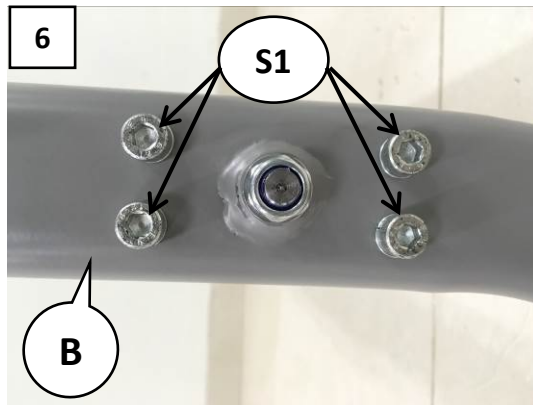
6. Secure the inner column assembly to the base (B) with four spring washers and M8x40mm hex bolts (S1); tighten the bolts firmly.


7. Attach the support plate (H) on top of the gas spring, turning until it stops at the red-lined nut, ensuring the keying feature (Q) aligns facing out from the column as indicated with red arrow.

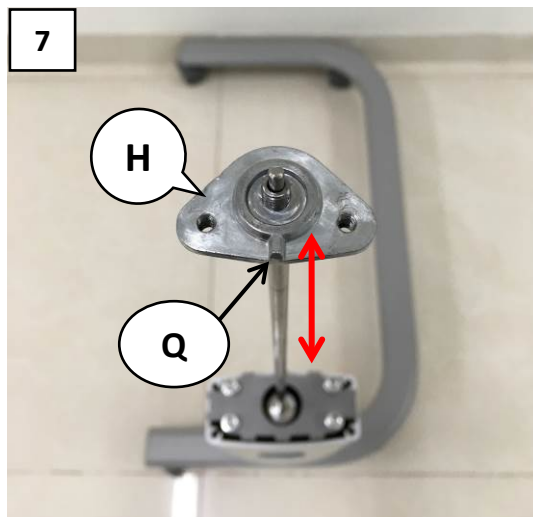Important: Ensure the red-lined nut remains in position.


8. On the underside of the table top assembly, remove four M6x35mm hex bolts (S2).

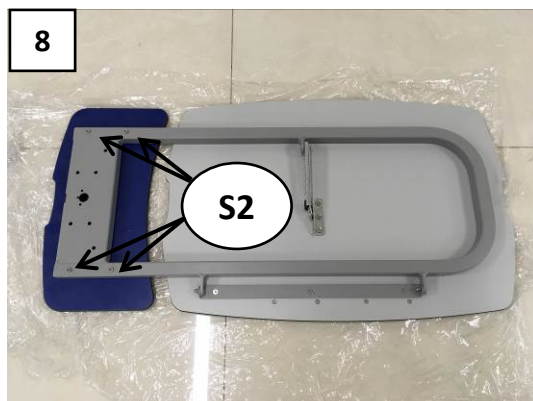
NOTE: The removed wrapping film from the table top assembly may be used to protect the table top during this step.

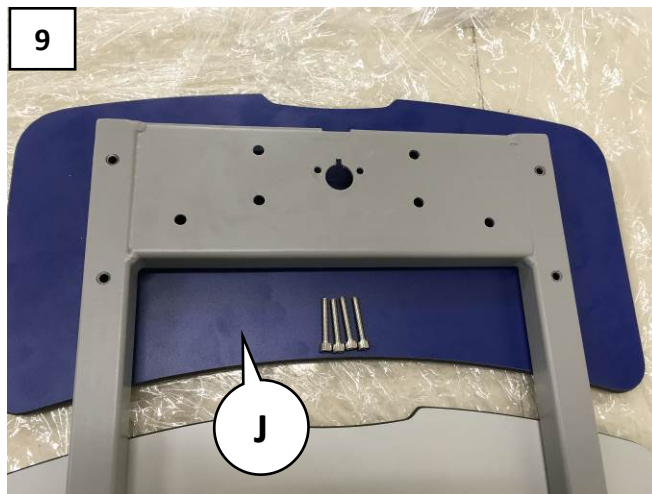9. Detach the side table (J).
NOTE: Model OBT0 will have a single fixed large table top.

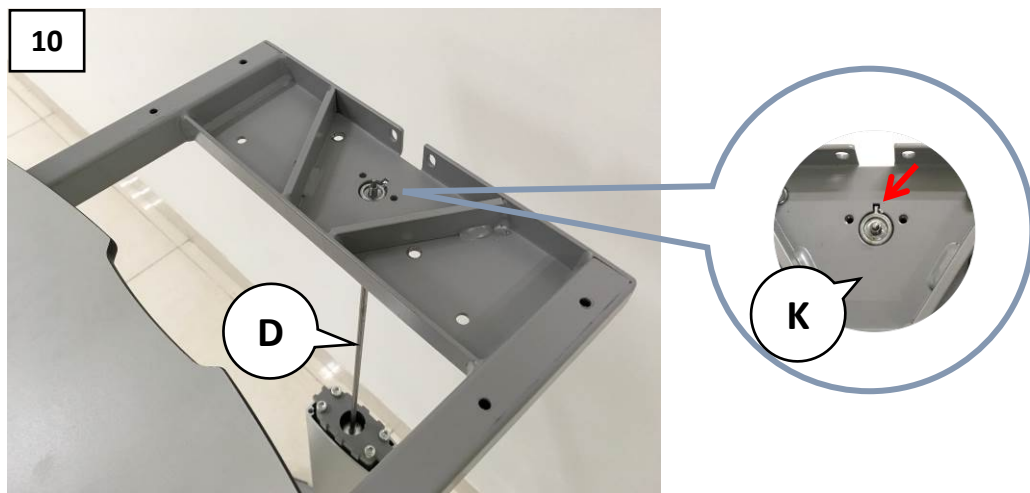
10. Align the top of the gas spring (D) to the key hole on table frame (K).


Important: To avoid damage to gas spring, a second person should hold the table frame in position to align with the support plate.

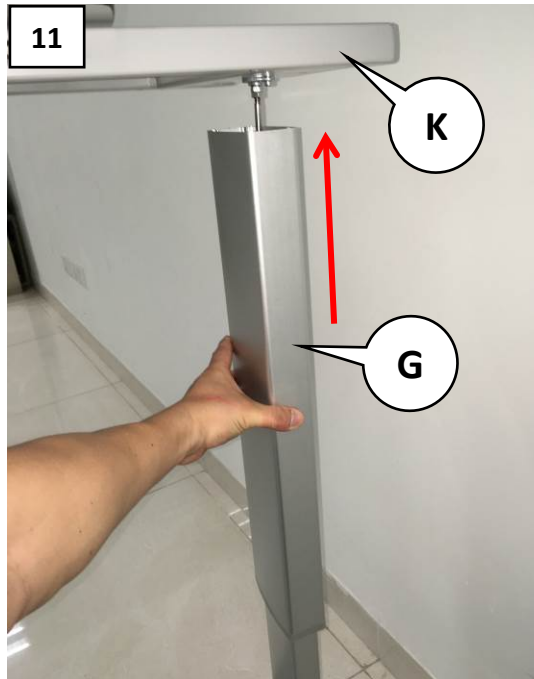
11. Slide the outer column housing (G) upwards to the table frame (K).

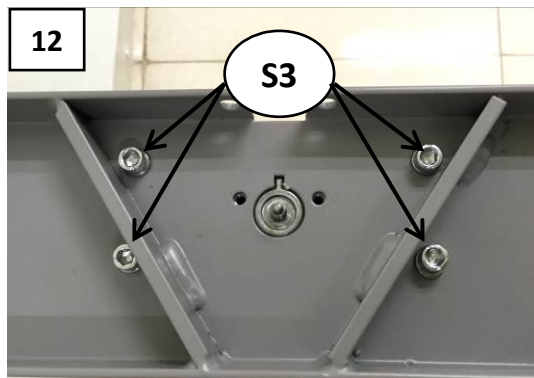
12. Secure the outer column housing to the table frame with four spring washers and M6x35 hex bolts (S3); tighten the bolts firmly

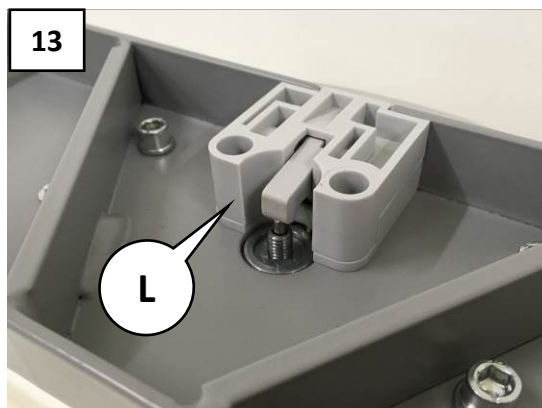
13. Position the release lever assembly (L) over the top of gas spring.

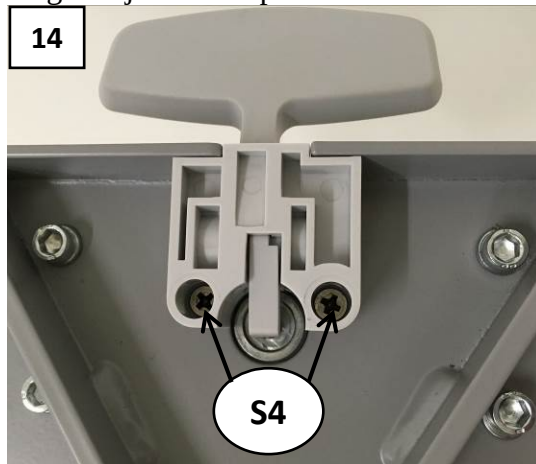14. Secure the release lever assembly (L) with two M4x20mm screws (S4). Check height adjustment operation with release lever.

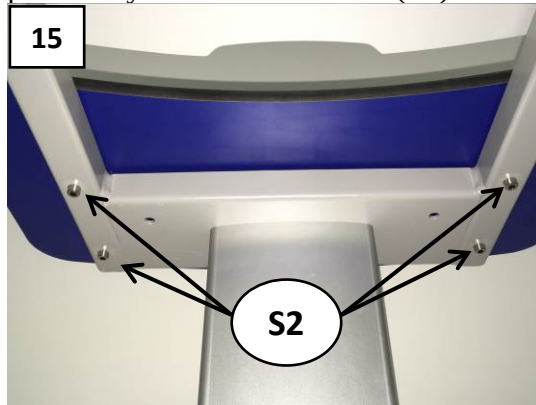
15. Re-attach the side table onto the top of the table frame and secure with previously removed hex bolts (S2).

16. Fully assembled product.

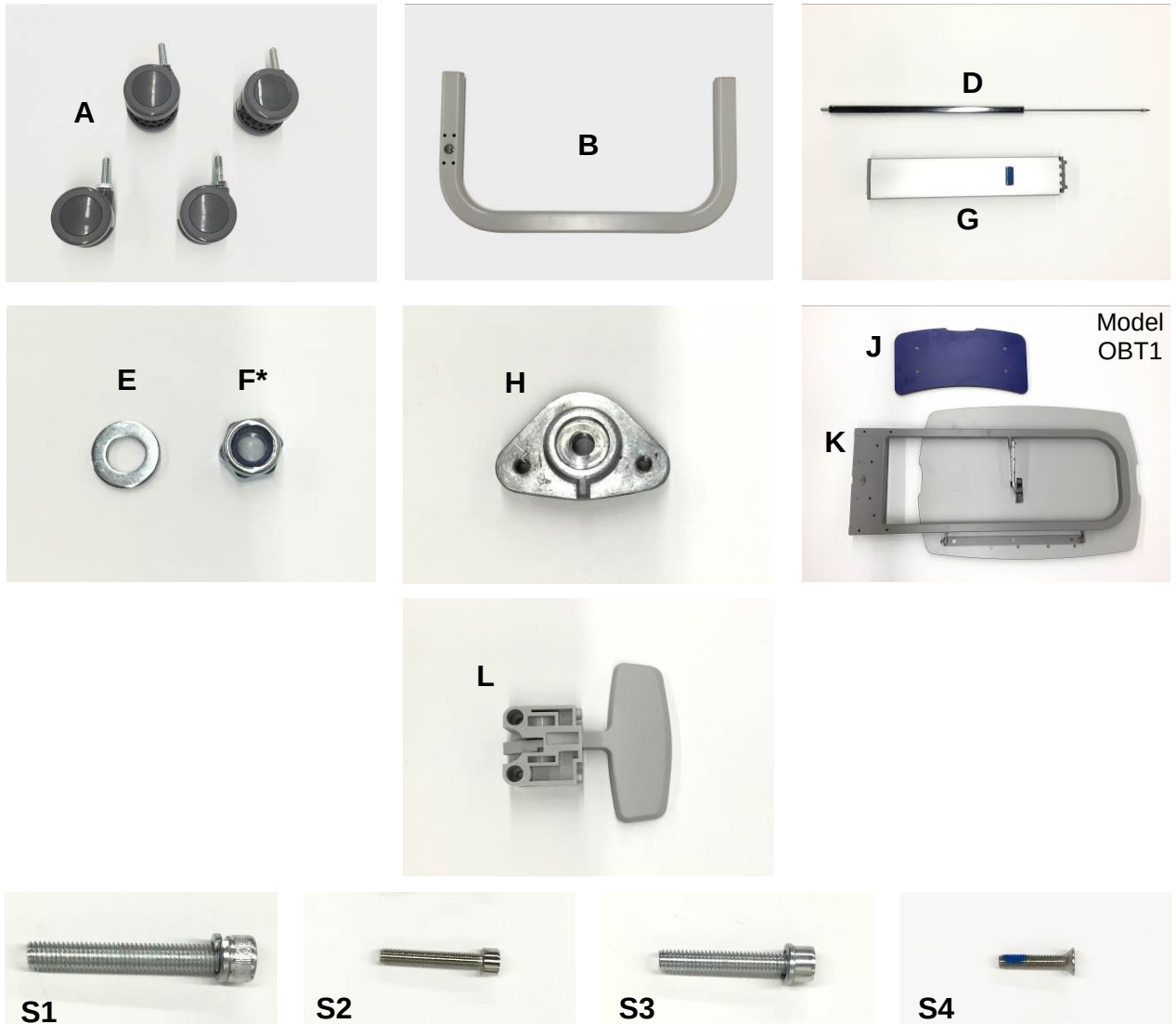3.2 Component identification

* The nut **F** may be supplied fitted to the gas spring **D**

WARNING

Do not adjust or dismantle any of the pre-assembled components except where shown in these instructions.

Before using the Overbed Table, check that all fixings (screws, nuts, etc.) are secure.

4. Operation

Height Adjustment

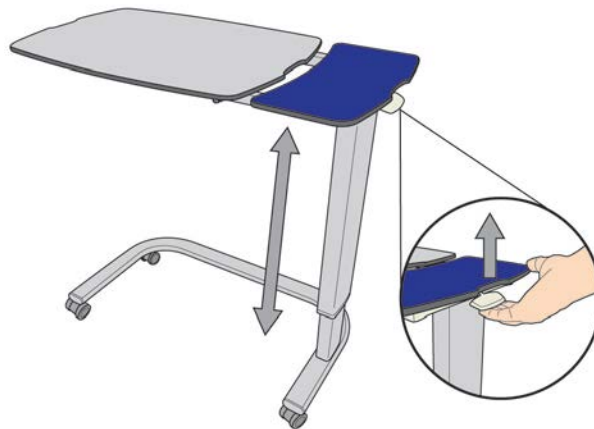
To lower the Overbed Table height:

Lift the height adjustment release lever and push the table top down.

NOTE: To avoid entrapment, the Overbed Table is designed to allow it to rise if pressure is applied from underneath.

To raise the Overbed Table height:

Lift the table top to rise under the action of the internal gas spring.

Height Adjustment

**Tilt
Adjustment
(model OBT1
with tilting
top only)**

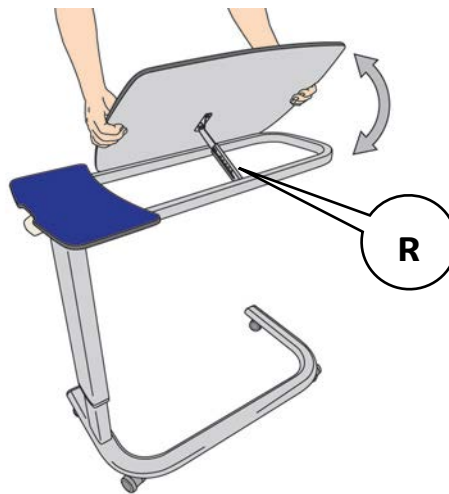
The tilt angle of the main table may be adjusted to one of six positions.

To increase the tilt angle:

Hold the edge of the main table as shown. Lift the main table until the ratchet (R) engages at the required position.

To reduce the tilt angle:

Hold the edge of the main table as shown. Tilt the main table as far as it will go, then lower it back to the flat position.

Tilt Adjustment

WARNING

Take care not to trap hands or fingers between the fixed and movable parts of the table 

Do not place hot drinks on the tilting section.

5. Cleaning and Maintenance

5.1 Cleaning

Wearing suitable protective clothing, clean all surfaces with a disposable cloth soaked in a neutral detergent and hand hot water.

Start by cleaning the upper sections of the Overbed Table and work along all horizontal surfaces. Work methodically towards the lower sections of the Overbed Table and clean the castors last. Take extra care to clean areas that may trap dust or dirt.

Rinse with clean water and dry with disposable paper towels.

5.2 Disinfecting

After cleaning the Overbed Table as described above, wipe all surfaces with sodium dichloroisocyanurate (NaDCC) at a concentration of 1,000 parts per million (0.1%) of available chlorine.

In the case of pooling body fluids, e.g. blood, the concentration of NaDCC should be increased to 10,000 parts per million (1%) of available chlorine.

Caution

Avoid excessive wetting of the product, particularly the table top

Do not use abrasive compounds or pads

Do not use phenol-based disinfectants

5.3 Preventive Maintenance

The following preventive maintenance procedures should be carried out at intervals of 12 months. All service or repair activities must be carried out by properly qualified and trained persons approved by ArjoHuntleigh.

Check that the height adjustment mechanism functions smoothly and that the main table locks securely in position over the full range of Overbed Table height.

Make sure all four castors are securely fixed to the base and that they swivel and rotate freely. Remove any debris that may have been picked up by the wheels.

Models OBT1 & OBT2 only: Check that the main table locks securely at each tilt position.


6. Warranty and Service

ArjoHuntleigh's standard terms and conditions apply to all sales. A copy is available on request. These contain full details of warranty terms and do not limit the statutory rights of the consumer.

For service, maintenance and any questions regarding this, or any other ArjoHuntleigh product, please refer to the local contact information provided on the inside back cover of this Instructions for Use document.

7. Specifications / Configurations

OBT0, OBT1, and OBT2 Overbed / Overchair Tables

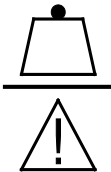Table top dimensions	92 x 47 cm max.	
Height adjustment range	73.5 to 110 cm	
Tilt angles (approx.)	0°/18°/25°/32°/38°/43°	
Product weight (approx.)	17 kg	
Castor size	50mm	
Safe working load	25 kg evenly distributed  25 kg 55 lb	
Flame rating	Classification M1 according to Standard NF P 92-507:2004	
Material	Table tops	High Pressure Compact Laminate (HPCL)
	Column	Aluminum Extrusion Anodized Finish
	Base	Tubular Steel Powder-Coated Finish
Material properties of HPCL	Resistance to scratches and impact from daily use	
	Resistance to moisture, steam, and heat	
Available Configurations		
Color	AB – AH Blue LL – Lilac LB – Light Blue LG – Light Grey WO – White Oak AT – Ash Tree NW – North Walnut	
Options	OBT0 – Fixed table top, no tilt OBT1 – Split table top (no rail) OBT2 – Split table top with perimeter rail	

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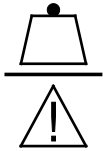
NORGE

ArjoHuntleigh Norway AS
Olaf Helsets vei 5
N-0694 OSLO
Tel: +47 22 08 00 50
Faks: +47 22 08 00 51
E-mail: no.kundeservice@arjohuntleigh.com

ÖSTERREICH

ArjoHuntleigh GmbH
Dörrstrasse 85
AT-6020 INNSBRUCK
Tel: +43 (0) 512 204 160 0
Fax: +43 (0) 512 204 160 75

POLSKA

ArjoHuntleigh Polska Sp. z o.o.
ul. Ks Piotra Wawrzyniaka 2
PL-62-052 KOMORNIKI (Poznań)
Tel: +48 61 662 15 50
Fax: +48 61 662 15 90
E-mail: arjo@arjohuntleigh.com

PORTUGAL

ArjoHuntleigh em Portugal
MAQUET Portugal, Lda. (Distribuidor Exclusivo)
Rua Poeta Bocage n.º 2 - 2G
PT-1600-233 Lisboa
Tel: +351 214 189 815
Fax: +351 214 177 413
E-mail: Portugal@arjohuntleigh.com

SUISSE / SCHWEIZ

ArjoHuntleigh AG
Fabrikstrasse 8
Postfach
CH-4614 HÄGENDORF
Tél/Tel: +41 (0) 61 337 97 77
Fax: +41 (0) 61 311 97 42

SUOMI

Oy Vestek AB
Martinkuja 4
FI-02270 ESPOO
Puh: +358 9 8870 120
E-mail: info@vestek.fi

SVERIGE

ARJO Scandinavia AB
Hans Michelsensgatan 10
SE-211 20 MALMÖ
Tel: +46 (0) 10 494 7760
Fax: +46 (0) 10 494 7761
E-mail: kundservice@arjohuntleigh.com

UNITED KINGDOM

ArjoHuntleigh UK
ArjoHuntleigh House
Houghton Hall Park
Houghton Regis
UK-DUNSTABLE LU5 5XF
Tel: +44 (0) 1582 745 700
Fax: +44 (0) 1582 745 745
E-mail: sales.admin@arjohuntleigh.com

USA

ArjoHuntleigh Inc.
2349 W Lake Street Suite 250
US-Addison, IL 60101
Tel: +1 630 307 2756
Free: +1 800 323 1245 Institutional
Free: +1 800 868 0441 Home Care
Fax: +1 630 307 6195
E-mail: us.info@arjohuntleigh.com

REV 17: 03/2016

GETINGE GROUP

Getinge Group is a leading global provider of products and systems that contribute to quality enhancement and cost efficiency within healthcare and life sciences. We operate under the three brands of **ArjoHuntleigh**, **Getinge** and **Maquet**. **Getinge** provides solutions for infection control within healthcare and contamination prevention within life sciences. **Maquet** specializes in solutions, therapies and products for surgical interventions, interventional cardiology and intensive care.

ARJOHUNTLEIGH

GETINGE GROUP

ArjoHuntleigh focuses on patient handling and hygiene, disinfection, DVT prevention, medical beds, therapeutic surfaces and diagnostics.

ArjoHuntleigh AB
Hans Michelsensgatan 10
211 20 Malmö, Sweden
www.arjohuntleigh.com